


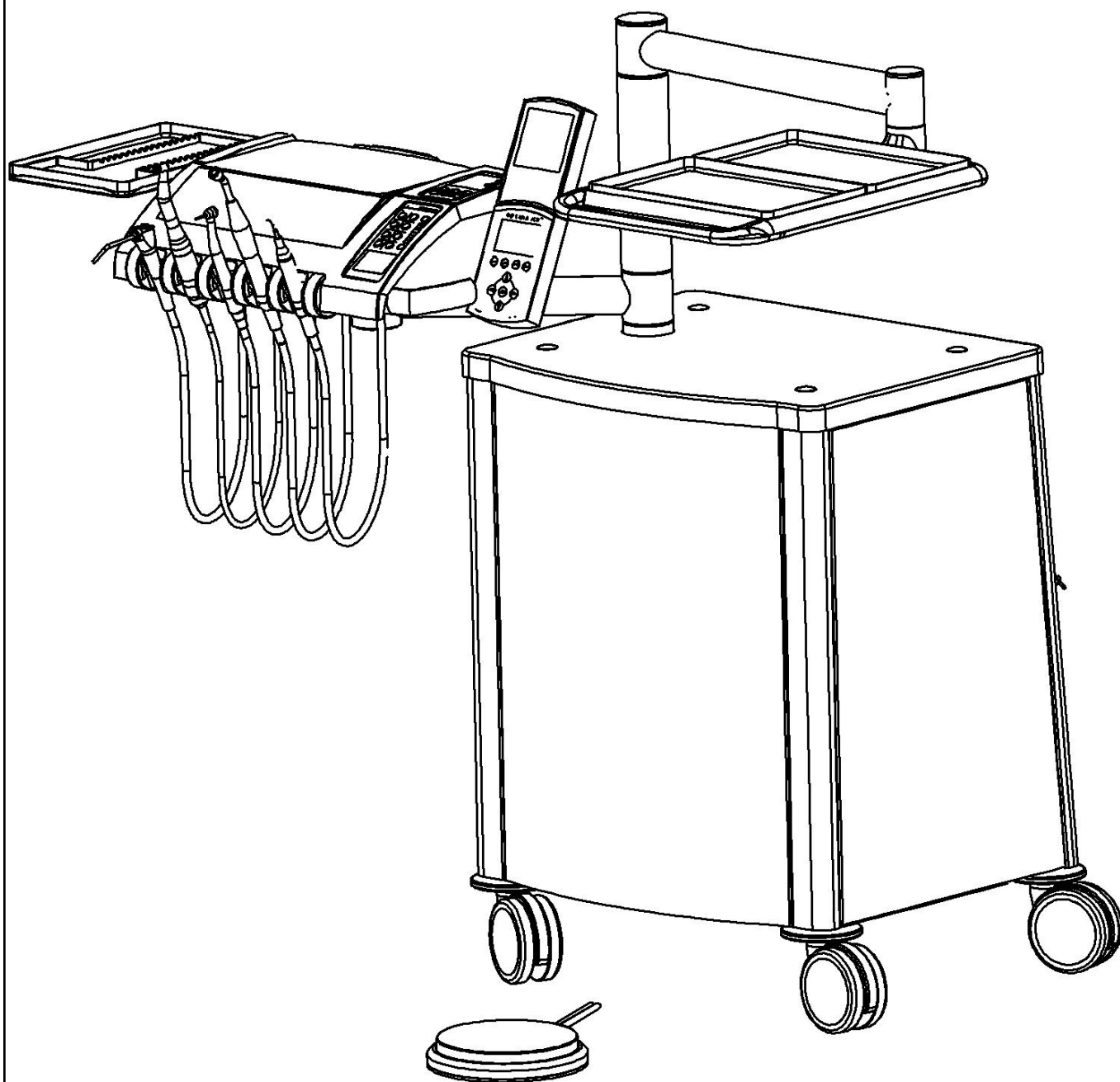


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

*Gallant Cart with
compressor
List 3*

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
3.1	001	Doctor table		
3.1	002	Table	AEGA127.06.00.000	
3.1	003-01	Foot controler	FCF 251-01 (US Medlink)	
3.1	003-02	Foot controler P2L1	224900200 (FARO)	
3.1	003-03	Foot controler	AEGA 4.320.001	

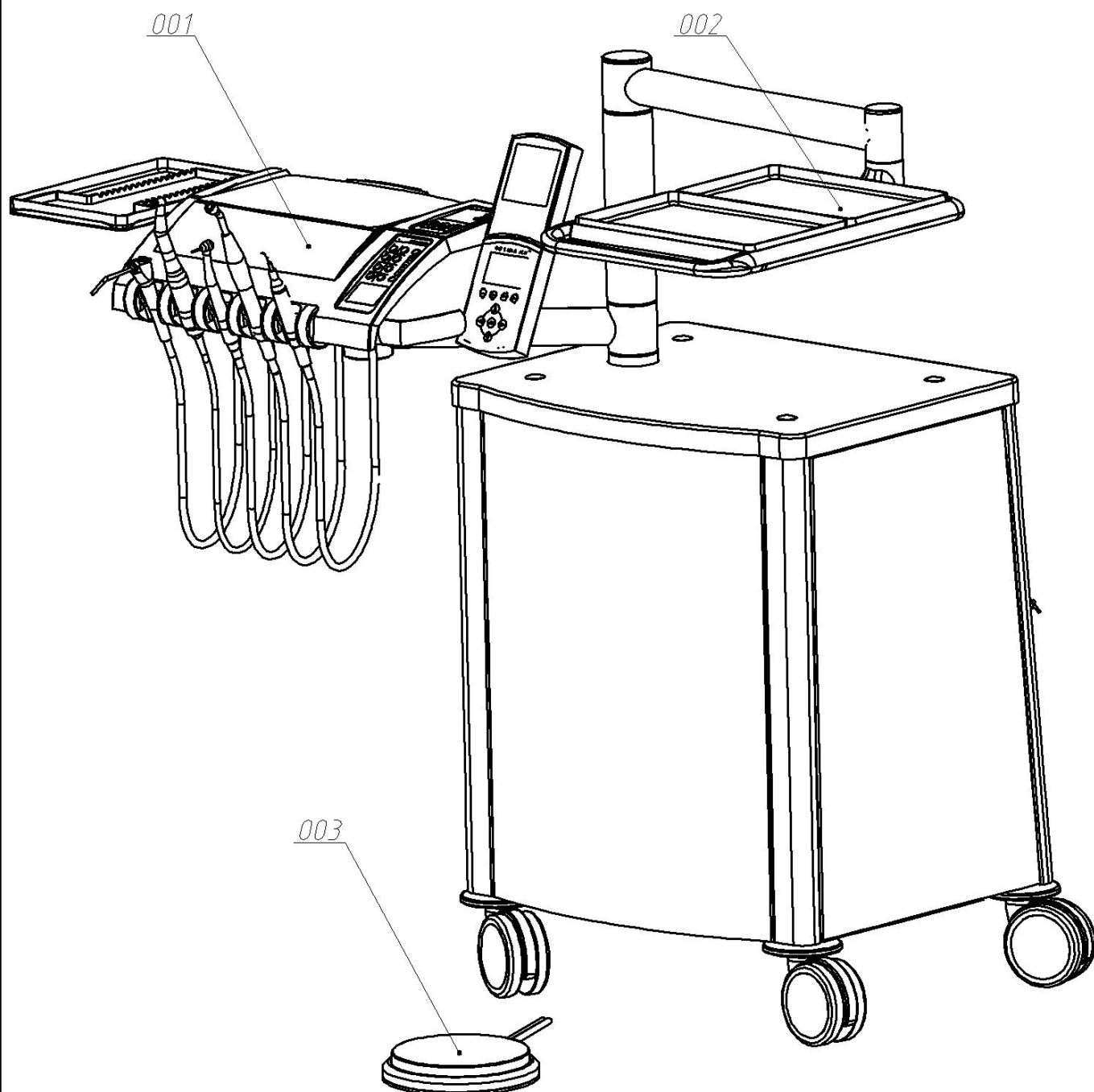


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

*Gallant Cart with
compressor
List 3.1*

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
3.2	001	Arm	GAL 999.05.00.000	
3.2	002	Plug	GAL 999.00.00.003	
3.2	003	Slider bearing	GAL 999.04.00.002	
3.2	004	O-Ring	GAL 999.00.00.002	
3.2	005	Extension arm	GAL 999.06.00.000	
3.2	006	O-Ring	GAL 999.00.00.004	
3.2	007	Arm	GAL 999.04.00.000	
3.2	008	Stand	GAL 999.02.00.000	
3.2	009	Holder	GAL 999.03.00.00	
3.2	010	Firm bottle	2I ACF-543-02	
3.2	011	Swich	AE-1353BNAB	
3.2	012	Screw	A.M6x8 DIN 915	

Connection arms to uprise

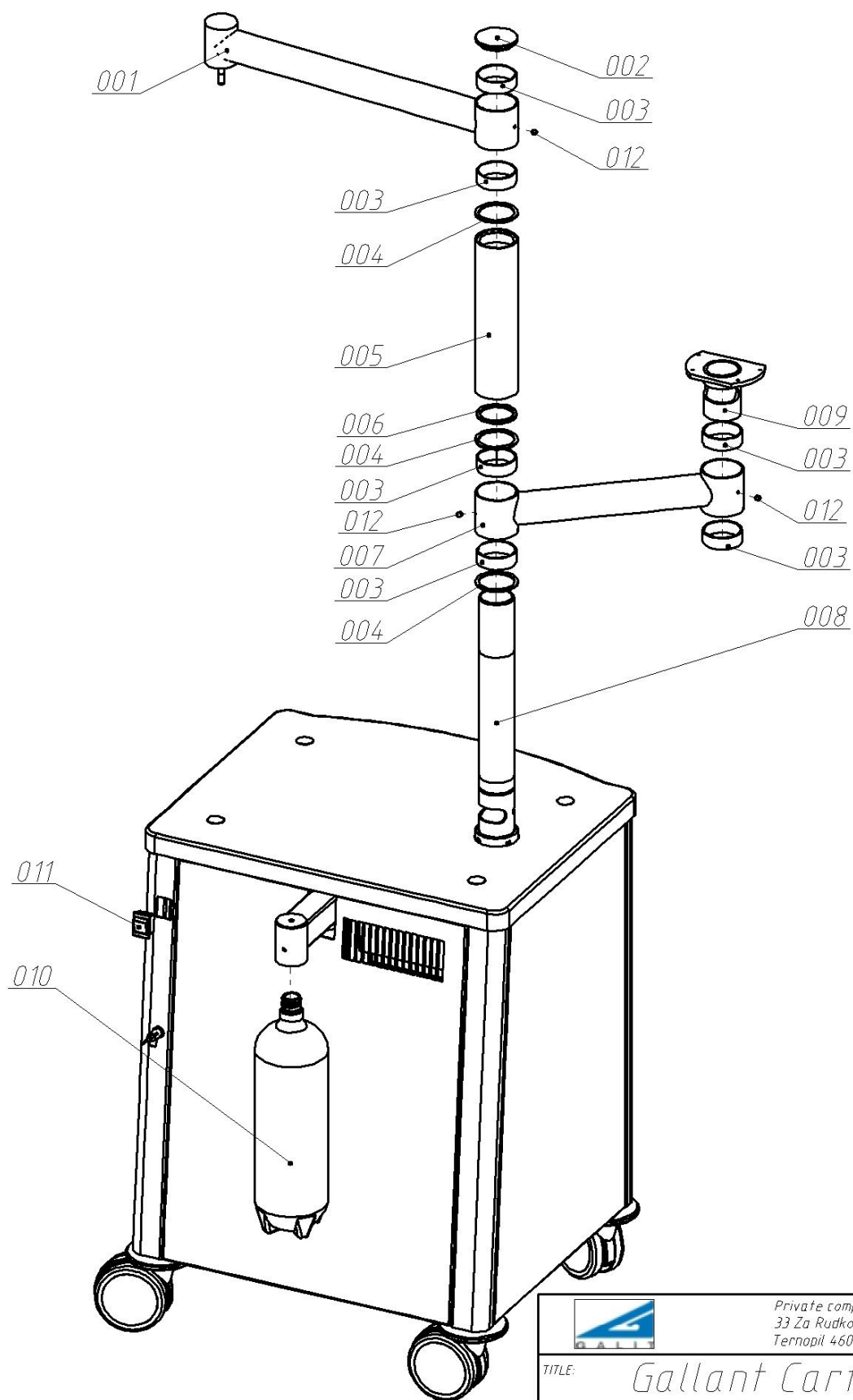


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

Gallant Cart with
compressor
List 3.2

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
3.3	001	Panel	GAL 999.08.00.000	
3.3	002	Panel	GAL 999.00.00.005-01	
3.3	003	Panel	GAL 999.00.00.005	
3.3	004	Panel	GAL 999.00.00.006	
3.3	005	Holder	GAL 999.07.00.000	
3.3	006	Upper Plate	GAL 999.00.00.001	
3.3	007	Bolt	GAL 999.00.00.009	
3.3	008	Stand	GAL 999.02.00.000	
3.3	009	Hexagon nut	DIN 934 - M10	
3.3	010	Screw	DIN 912 M6 x 16	
3.3	011	Screw	DIN 7985 - M4 x 8	

Uprise

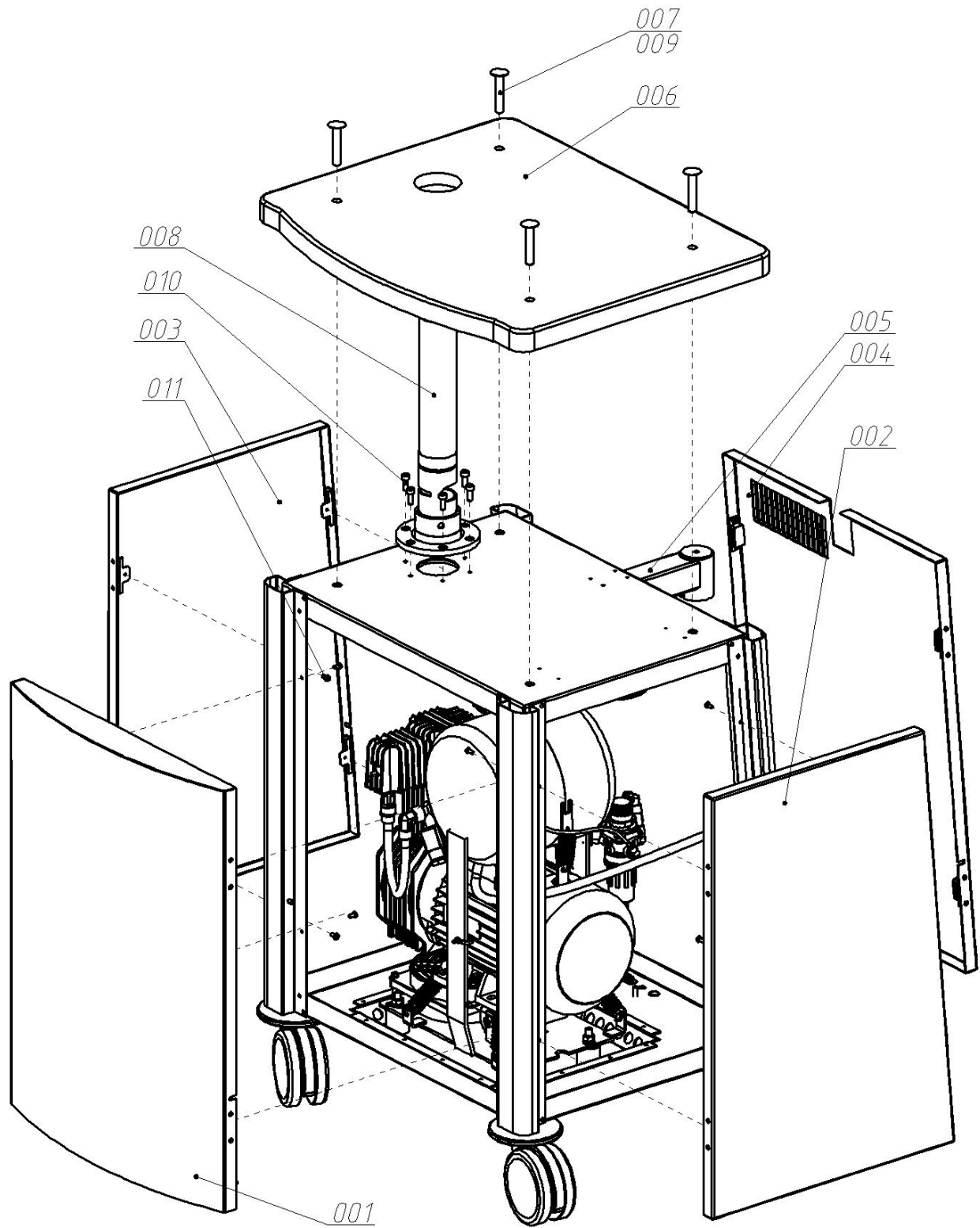


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

Gallant Cart with
compressor
List 3.3

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
3.4	001	Panel	GAL 999.00.00.005	
3.4	002	Bracket	GAL 999.00.00.007	
3.4	003	Screw	DIN 965/963- M3 x 8	

Panel

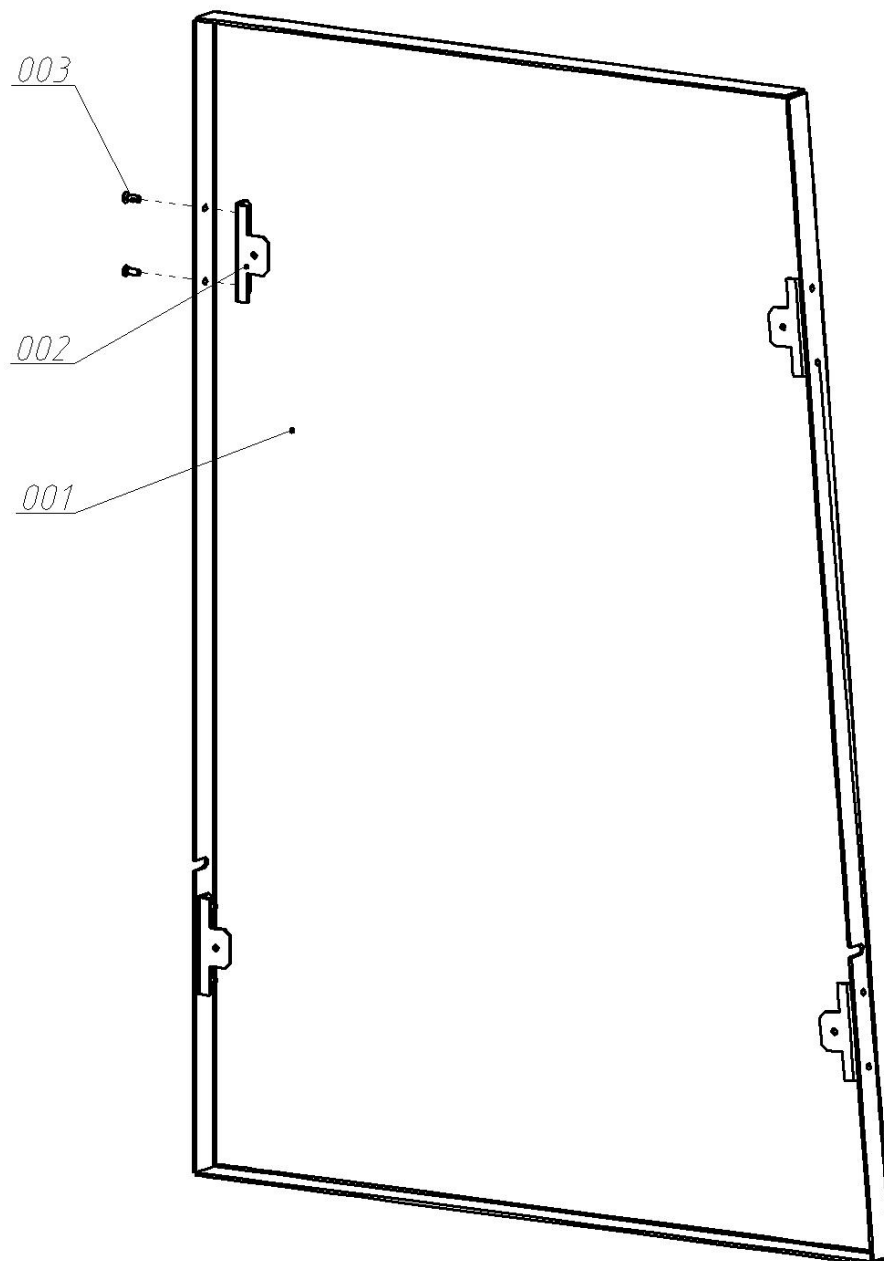


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

Gallant Cart with
compressor
List 3.4

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
3.5	001	Panel	GAL 999.00.00.006	
3.5	002	Magnet clip	246.26.741 HAFELE	
3.5	003	Screw	DIN 965/963- M3 x 10	
3.5	004	Hexagon nut	DIN 934 - M3	

Panel

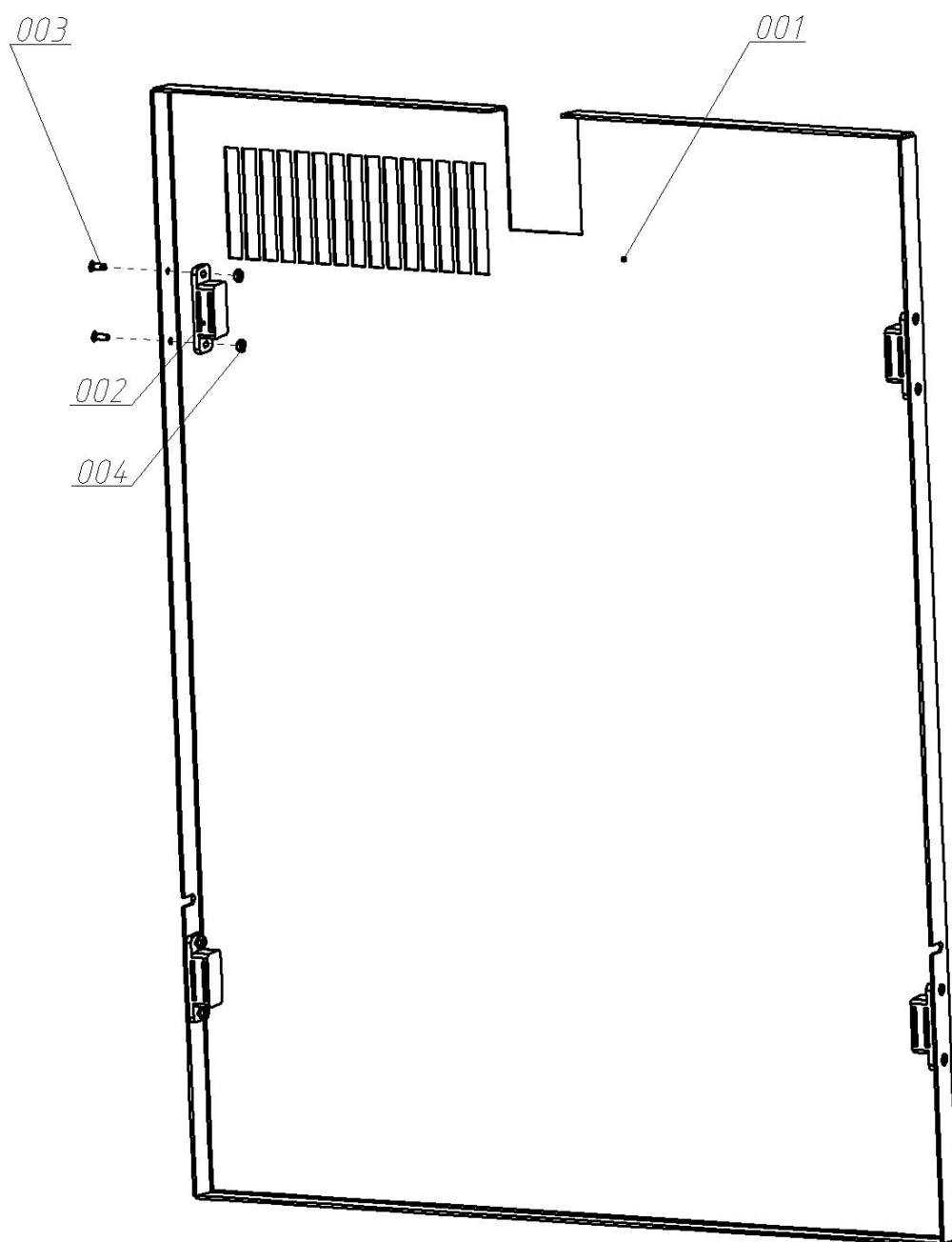


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

Gallant Cart with
compressor
List 3.5

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
3.6	001	Uprise	GAL 999.01.00.000	
3.6	002	Holder	GAL 999.07.00.000	
3.6	003	Upise	GAL 999.09.00.000	
3.6	004	Compressor	DK50i	
3.6	005	castor GUY-raymond		
3.6	006	Amot	30x30 Ekom	
3.6	007	Hexagon Nut	M8 GOST 5927-70(DIN 934)	
3.6	008	Spring Washer	8 GOST 6402-70(DIN 7980)-7	
3.6	009	Washer	A.8 ГОСТ 11371-78(DIN 125)	
3.6	010	Finger	AEGA 116.00.00.066	
3.6	011	Hexagon nut	DIN 934 - M4	
3.6	012	Filter Block		
3.6	013	Screw	DIN 7985 - M4 x 8	
3.6	014	Bolt	M5x45 GOST 7805-70 (DIN 933)	
3.6	015	Screw	DIN 965/963- M5 x 10	

Uprise

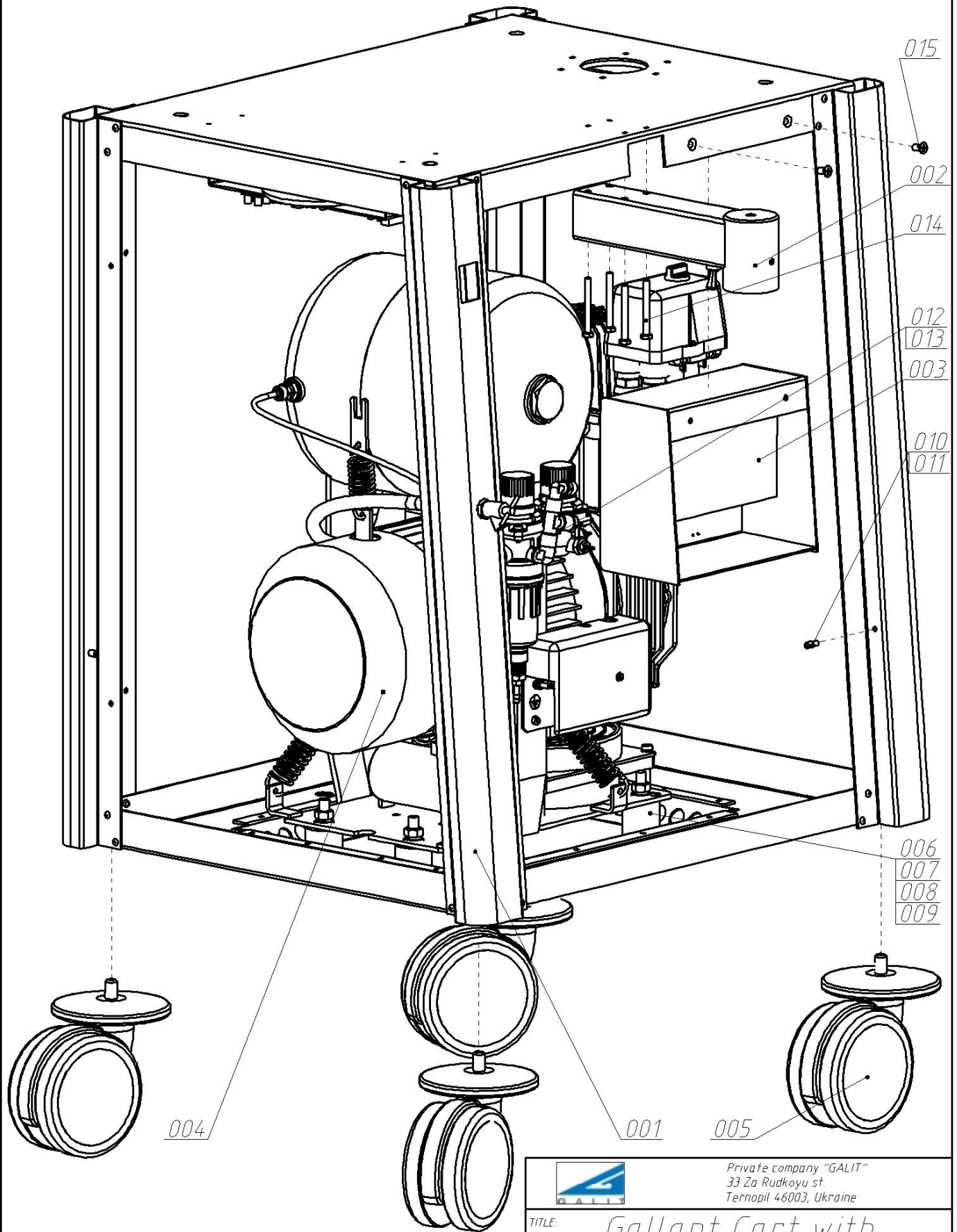


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

Gallant Cart with
compressor
List 3.6

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
3.7	001	Holder	GAL 999.07.00.000	
3.7	002	Plug	AEGA 016.02.00.001-01	
3.7	003	Fitting	2601 Ø2 M5	
3.7	004	Washer	Ø5 mm FIF 607-01	
3.7	005	Fitting	1052 1/16 10-32 FIF 617-01	
3.7	006	Fitting	2601 Ø4.5 M5	
3.7	007	Pneumatic switch	VAF-451-03	
3.7	008	Screw	M6x8 GOST 8878-93 (DIN 914)	
3.7	009	Coupling	FIF 605-01	
3.7	010	Coupling	FIF 605-02	

Attachment of water supply elements

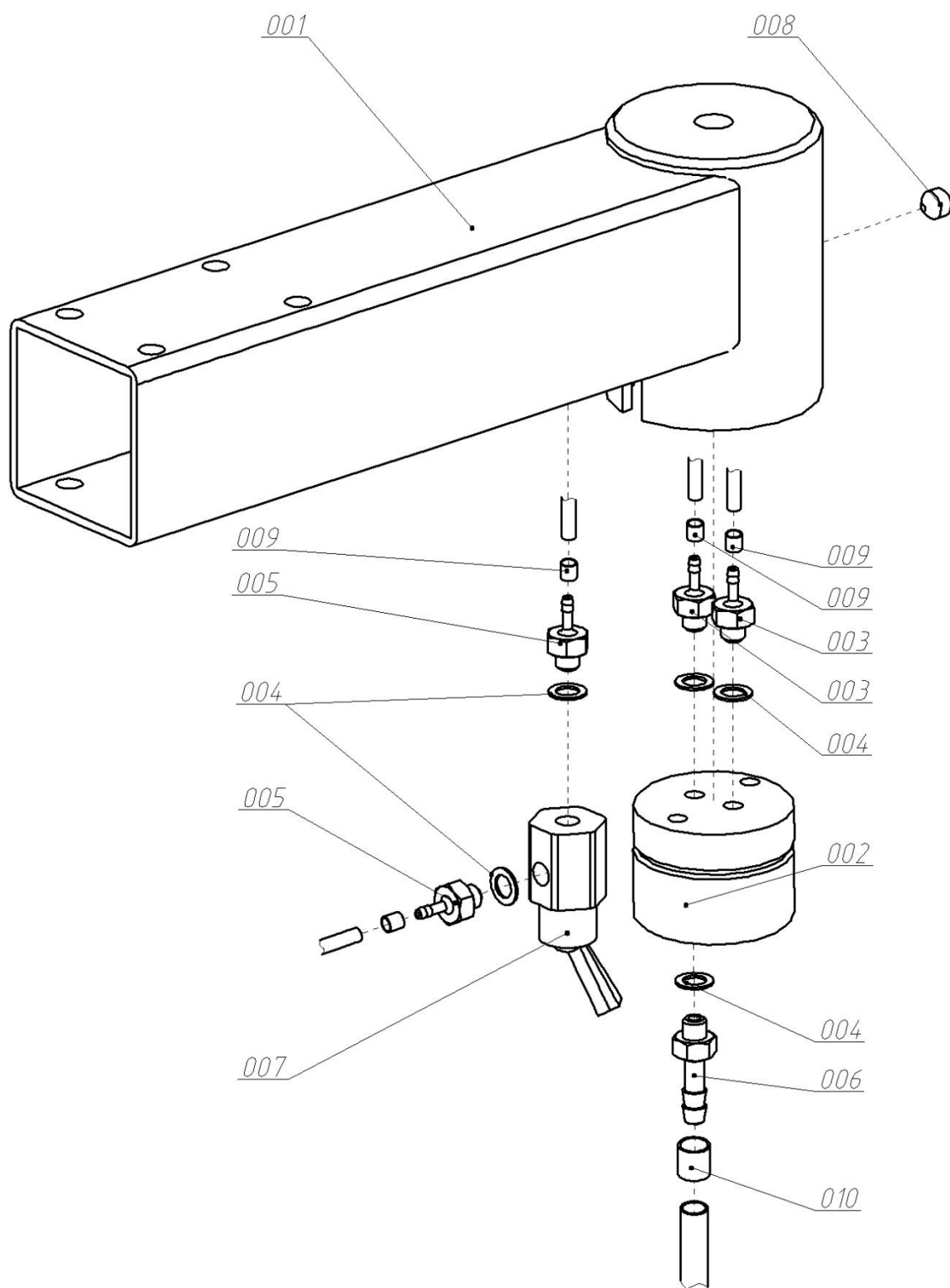


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

Gallant Cart with
compressor
List 3.7

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
3.8	001	Bracket	AEGA 116.00.00.070	
3.8	002	Filter-regulator	N108 D00	
3.8	003	Micro-regulator	M008 P00	
3.8	004	Fitting	1052 1/16 10-32 FIF 617-01	
3.8	005	Fitting	P1053 1/8 10-32 FIF 618-01	
3.8	006	Washer	Ø5 mm FIF 607-01	
3.8	007	Fitting	2500 1/8	
3.8	008	Fitting	S6520 8-1/8	
3.8	009	Fitting	1500 8/6-1/8	
3.8	010	Fitting	2531 1/8-M5	
3.8	011	Fitting	2070 1/8-1/8	
3.8	012	Fitting	2020 1/8-1/8	
3.8	013	Coupling	FIF 605-01	
3.8	014	Coupling	FIF 605-02	

Filter Block

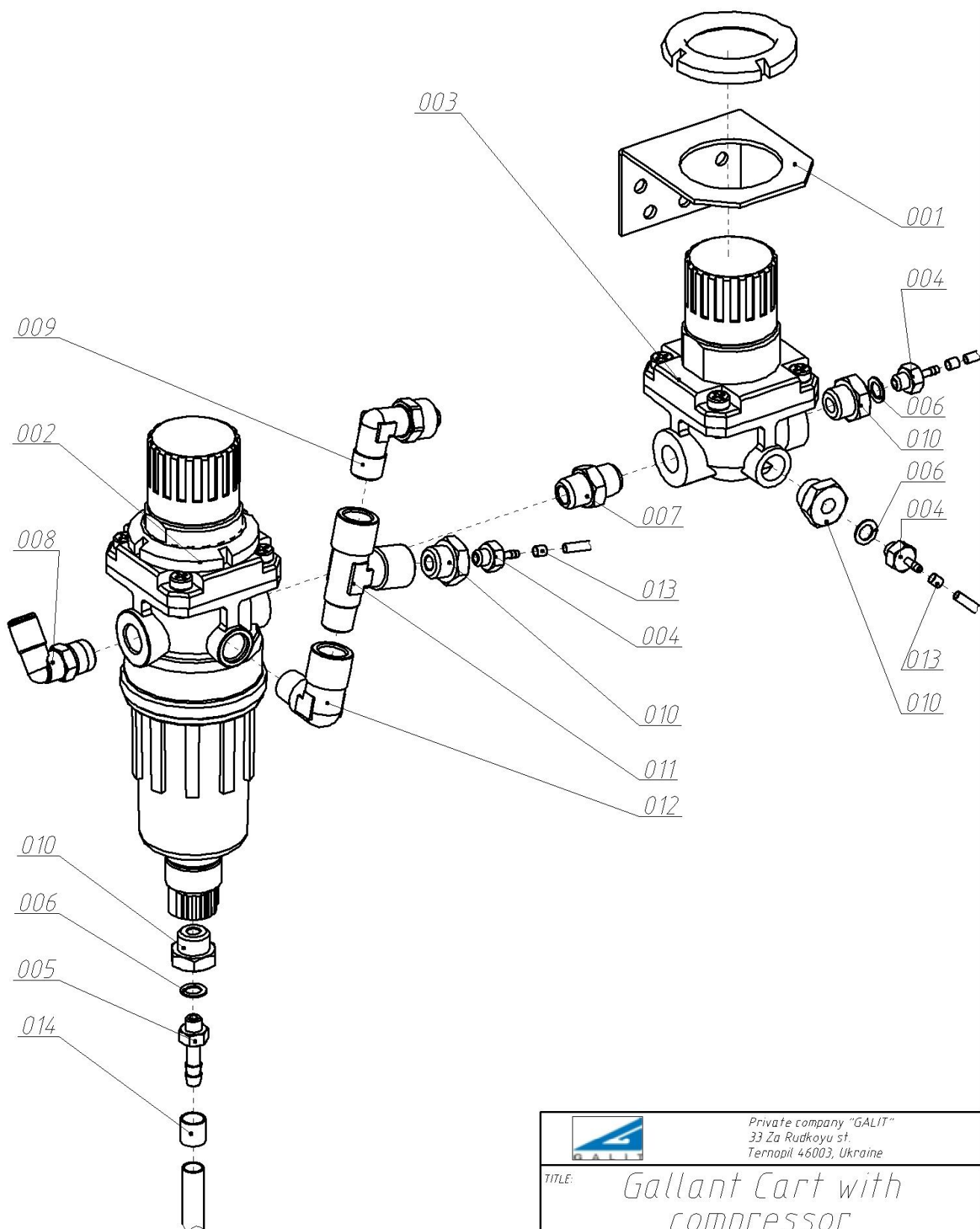


Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

Gallant Cart with
compressor
List 3.8

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
3.9	001	Bracket	GAL 999.00.00.008	
3.9	002	Fuse-holder	TLF-100-3	
3.9	003	Trans	TST100/013	
3.9	004	Bolt	M5x45 GOST 7805-70(DIN 933)	
3.9	005	Screw	DIN 7985- M3 x 20	
3.9	006	Screw	DIN 967 - M4 x 6	
3.9	007	Washer	A.5 GOST 11371-78(DIN 125)	
3.9	008	Hexagon nut	M5 GOST 5927-70(DIN 934)	

Electric parts

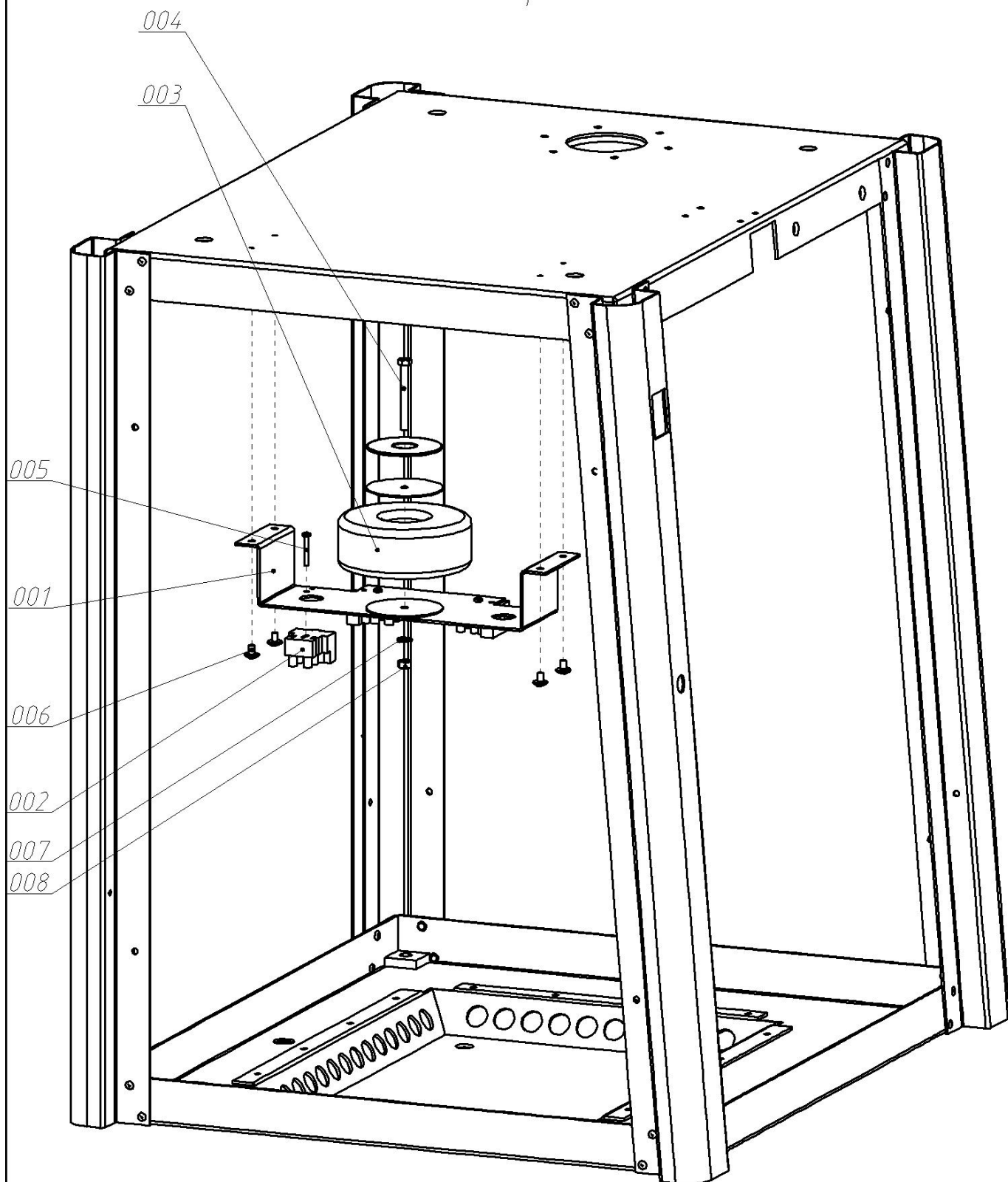

Private company "GALIT"
33 Za Rudkoyu st.
Ternopil 46003, Ukraine

TITLE:

Gallant Cart with
compressor
List 3.9