

MEtherm® 60 – Rinse aid



Manufacturer

MELAG Medizintechnik oHG
Geneststraße 6 - 10
D-10829 Berlin

1. Overview of product advantages

- For the stain-free drying of instruments
 - Avoids the formation of deposits
 - Improves drying results
-

2. Product description

MEtherm 60 is a liquid rinse aid (concentrate) used for the automatic reprocessing of medical / dental instruments and equipment such as dental instruments, surgical instruments MIC/micro instruments, ophthalmological instruments, fixed endoscopes etc. in MELAtherm 10. MEtherm 60 optimizes the drying results after the last rinse cycle in MELAtherm 10.

MEtherm 60 is not to be used for reprocessing ophthalmological instruments.

MEtherm 60 is pH neutral, has an especially high level of material-friendliness and avoids the development of film on instruments and the MELAtherm 10 over the long-term.

3. Application usage advice

Application concentration:

The application concentration of MEtherm 60 amounts to 0.2-0.6 ml/l (0.02-0.06 %). The metering is set on the works side and is performed in a fully-automatic fashion via the integrated metering module of the MELAtherm 10.

Application usage device:

Should it become necessary to replace the container, a corresponding message will be issued on the display of MELAtherm 10. Do not mix MEtherm 60 with other process agents.

4. Technical data

Composition:

5-15 % non-ionic surfactants

Density: approx. 1.037 g/cm³ / 20 °C

Colour: almost none

Form: liquid

Flash point: > 57 °C / method: DIN 51755 part 1

pH: approx. 4 / 20 °C / concentrate

5. Other data

Keep the container sealed closed. Store in original packaging. Protect against frost, heat and direct sunlight.

6. Ordering information

Article:		Art. no.:
MEtherm 60 - Rinse aid	940 ml Bottle	11623

7. Related products

- MEtherm 50 - Mildly-alkaline, enzymatic cleaning agent
 - MEtherm 55 - Neutralizer C (citric acid-based)
 - MEtherm 56 - Neutralizer P (phosphoric acid-based)
 - MEtherm 61 - Rinse aid
-