


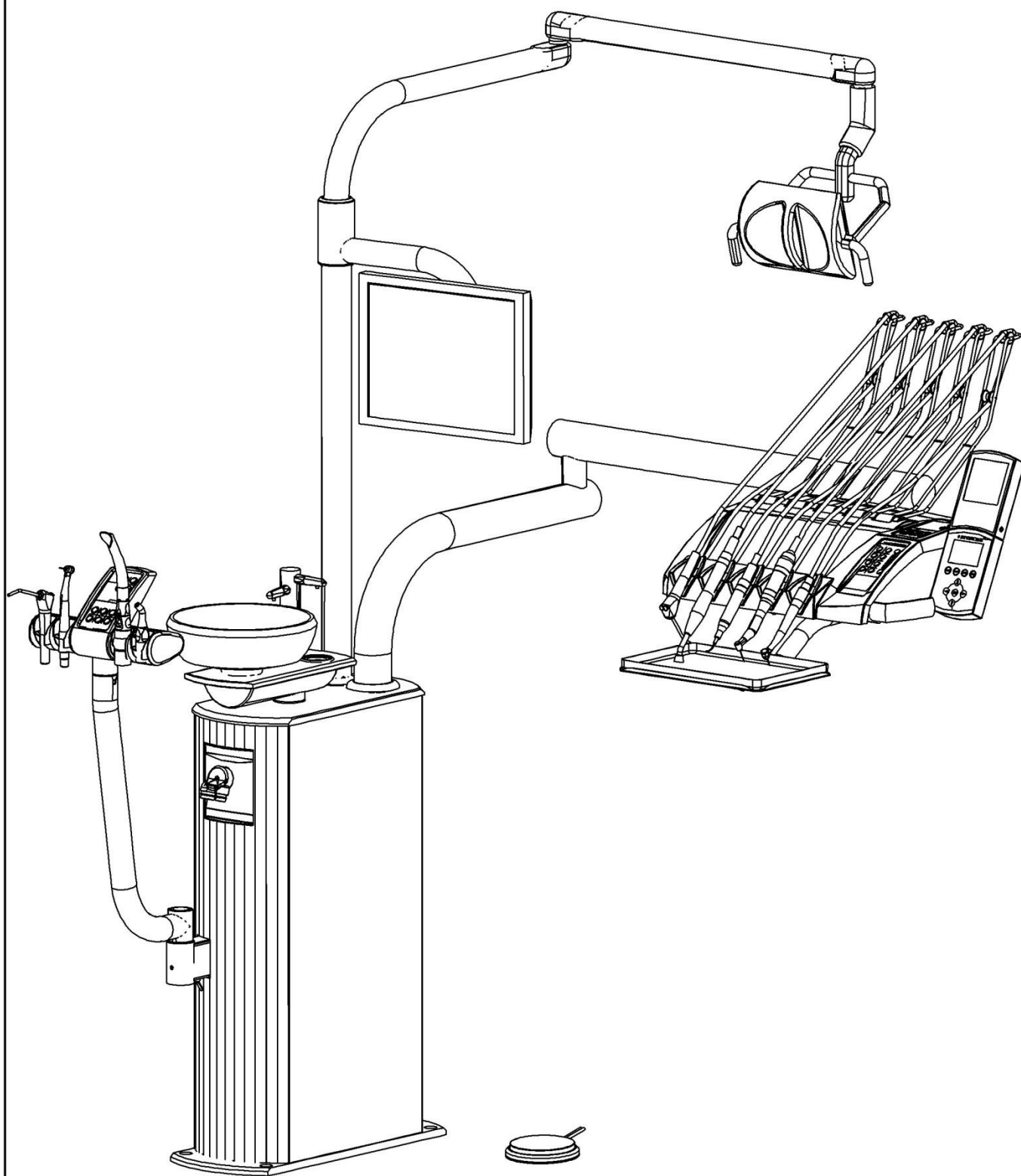


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.1	001	Water Unit		
2.1	002	Assistant table	AEGA 116.93.00.000	
2.1	003	Pantograph arm	AEGA 016.65.00.000	
2.1	004-01	Doctor table U	AEGA 127.00.00.000	
2.1	004-02	Doctor table L	AEGA 137.00.00.000	
2.1	005	Monitor holder	AEGA 043.00.00.000	
2.1	006	Light Post	AEGA 116.76.01.000	
2.1	007	Monitor	Neovo DR-17 (Standard)	
2.1	008-01	Light on pantograph arm EDI	306010300 (FARO)	
2.1	008-02	Light on pantograph arm ALYA	312000000 (FARO)	
2.1	008-03	Light on pantograph arm MAYA	320010300 (FARO)	
2.1	009-01	Foot controller	FCF 251-01 (US Medlink)	
2.1	009-02	Foot controller P2L1	224900200 (FARO)	
2.1	009-03	Foot controller	AEGA 4.320.001	

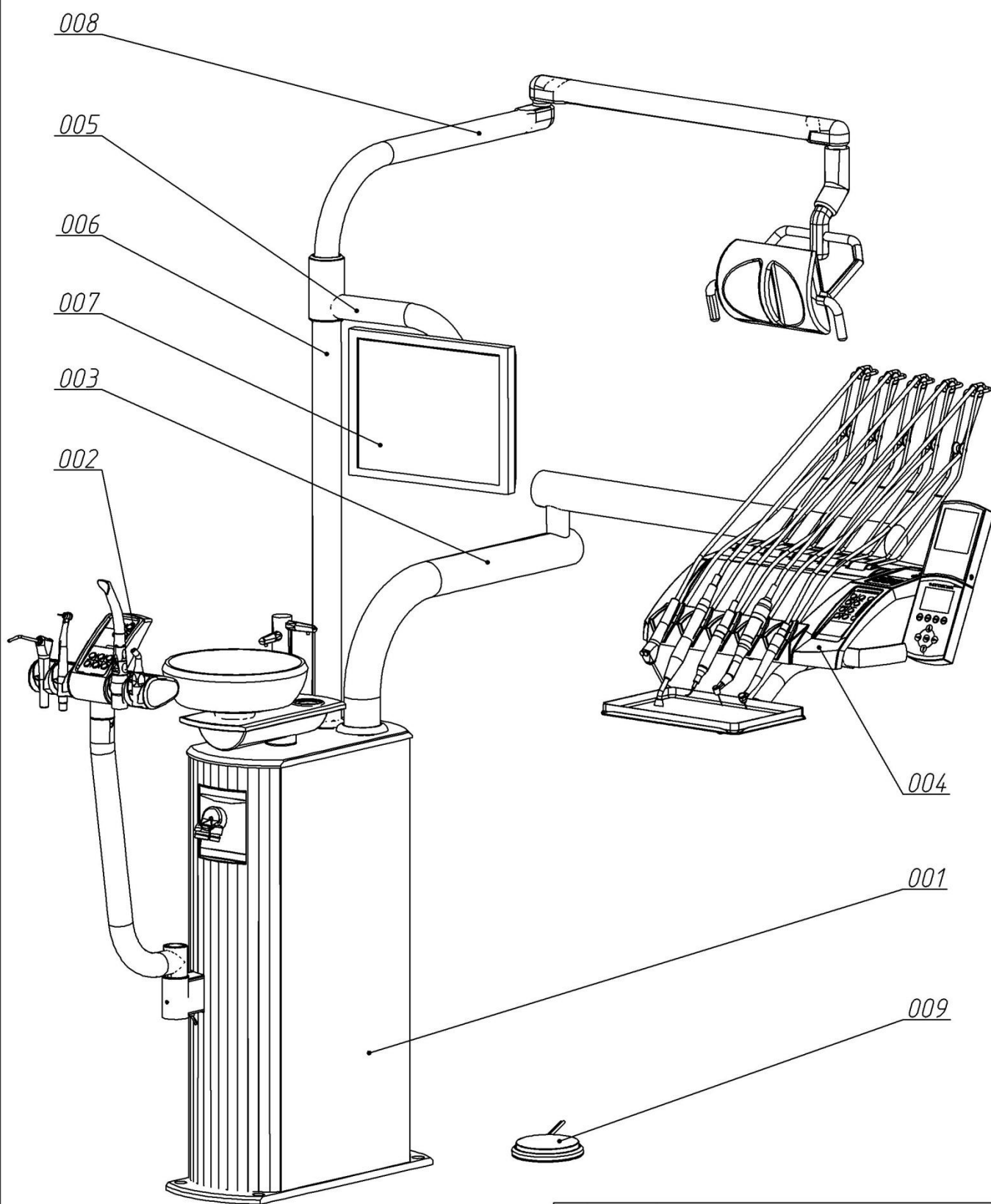


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:


Dental Unit "GALLANT F"
List 2.1

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.2	001	Stand	AEGA 216.01.00.000	
2.2	002	Post	AEGA 216.00.00.013	
2.2	003	Screw DIN 933 M6x12	5M66012-2	
2.2	004	Power supply	AEGA 116.95.00.000	
2.2	005	Panel	AEGA 216.00.00.001	
2.2	006	Finger	AEGA 116.00.00.066	
2.2	007	Screw DIN 912 M6x10	5156010-2	
2.2	008	Washer DIN 125 6	7O06-2	
2.2	009	Cramp	AEGA 216.00.00.004	
2.2	010	Nut DIN 934 M10	6010-2	
2.2	011	Washer DIN 125 10	7O10-2	
2.2	012	Washer DIN 7980 10	7P10-2	
2.2	013	Nut DIN 439B M10	6Z10-2	
2.2	014	Stud	AEGA 016.00.00.005	
2.2	015	Lower plate	AEGA 216.00.00.003	
2.2	016	Upper plate	AEGA 216.00.00.018	
2.2	017	Nut DIN 934 M4	6004-2	
2.2	018	Washer DIN 125 4	7O04-2	

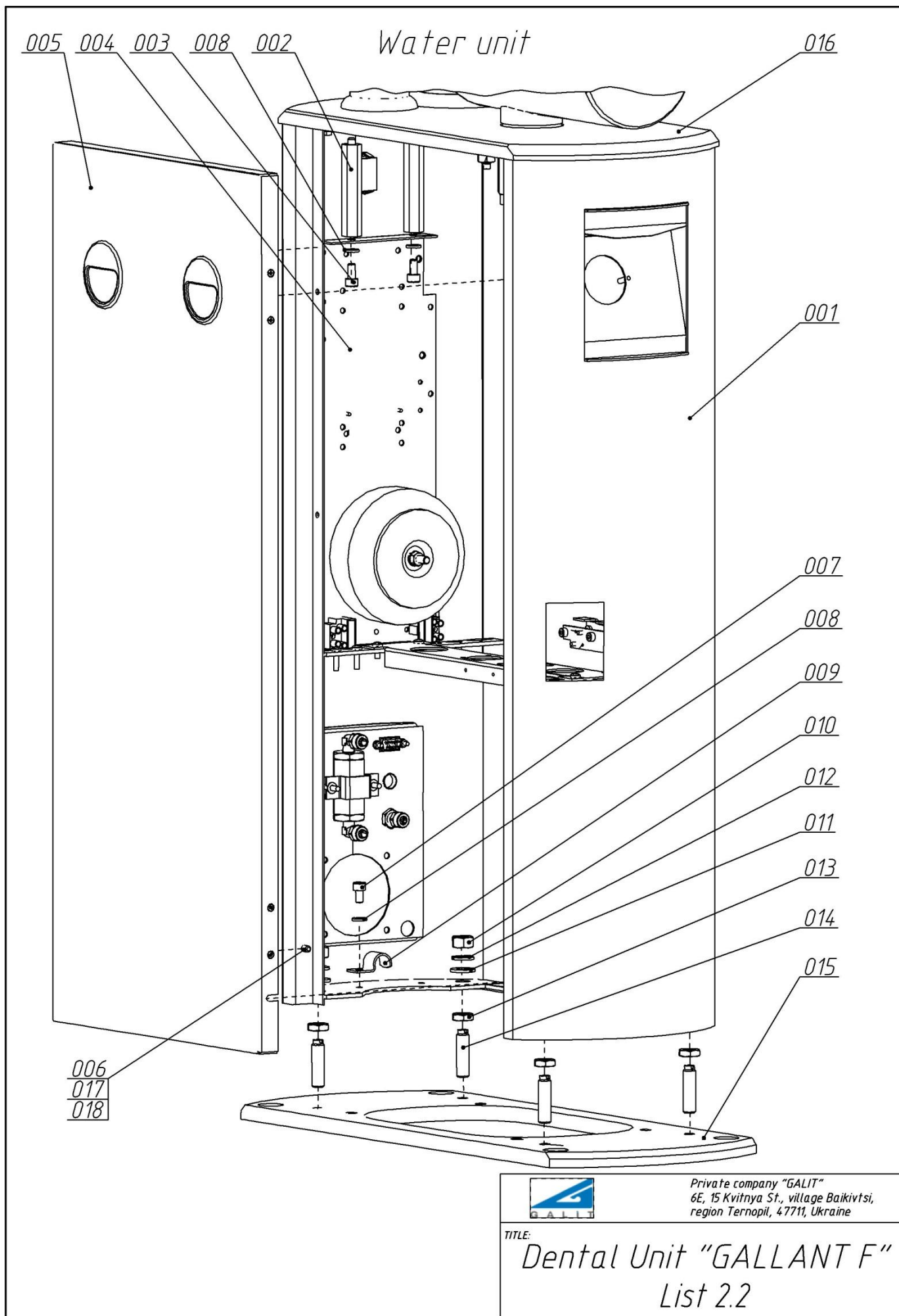

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.3	001	Panel	AEGA 216.00.00.001	
2.3	002	Ring	AEGA 216.00.00.010	
2.3	003	Magnet clip	246.26.741 HAFELE	
2.3	004	White rabbetted round knob		
2.3	005	Nut DIN 934 M3	6003-2	
2.3	006	Screw DIN 965/963 M3x10	5M13012-2	
2.3	007	Washer DIN 125 3	7O03-2	

Panel

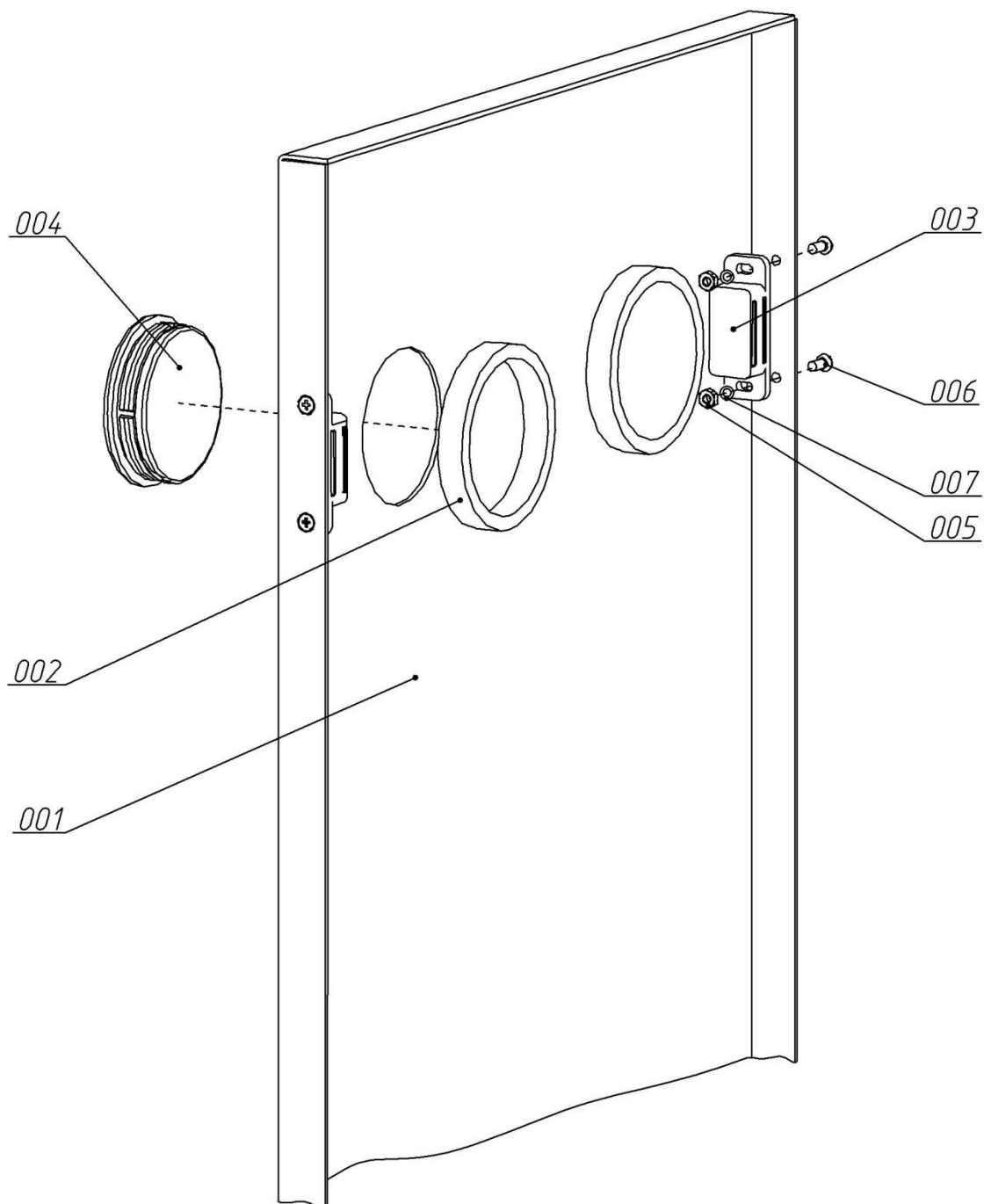


Private company "GALLIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.3

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.4	001	Arm	AEGA 216.04.00.000	
2.4	002	Ring	AEGA 216.00.00.007	
2.4	003	Holder	AEGA 216.03.00.000	
2.4	004	Bushing	AEGA 216.00.00.006	
2.4	005	Cap	AEGA 116.83.00.001	
2.4	006	Screw DIN 915 M6x6	986006-2	
2.4	007	Screw DIN 912 M6x85	5156090-2	
2.4	008	Washer DIN 125 6	7O06-2	
2.4	009	Washer DIN 7980 6	7P06-2	
2.4	010	Nut DIN 934 M6	6006-2	

Connection of assistant table to framework

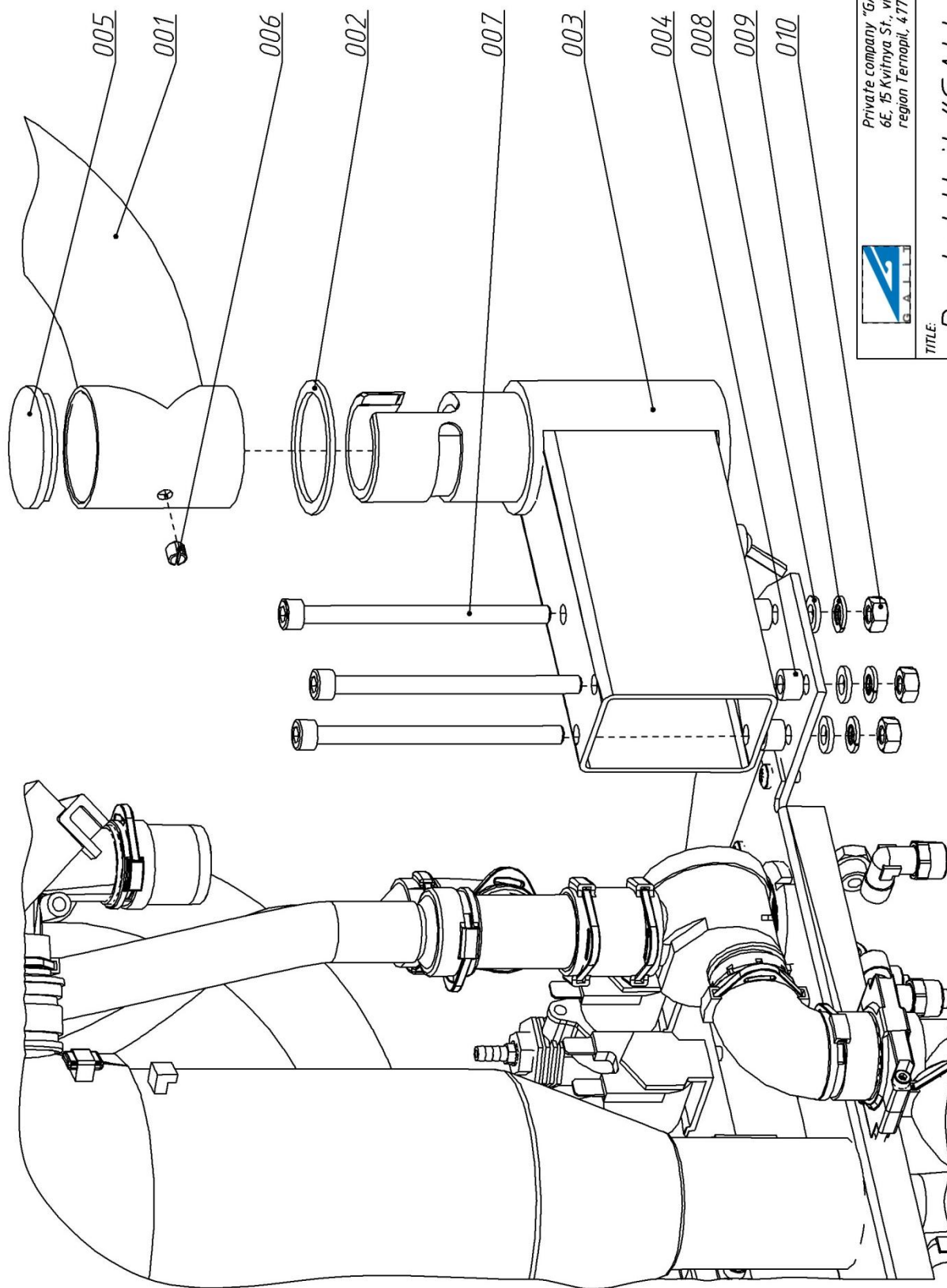

Private company "GALLIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine


TITLE:

Dental Unit "GALLANT F"

List 2.4

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
2.5	001	Ring	AEGA 116.00.00.061-01	
2.5	002	Ring	AEGA 116.00.00.061	
2.5	003	Cuspidor Unit	AEGA 916.00.00.000	
2.5	004	Upper Plate	AEGA 216.00.00.018	
2.5	005	Case	AEGA 916.04.00.000	
2.5	006	Stud	AEGA 216.00.00.011	
2.5	007	Cam	AEGA 916.00.00.011	
2.5	008	Cramp	AEGA 916.00.00.013	
2.5	009	Cup	AEGA 116.75.00.000	
2.5	010	Bearing	HK 40x16	
2.5	011	O-ring A47 GOST 13941-68	95PK2047	
2.5	012	Screw DIN 7985 M4x6	5M54006-2	
2.5	013	Case	AEGA 116.77.00.000	
2.5	014	Screw DIN 914 M6x6	986006-H	
2.5	015	Screw DIN 965/963 M6x8	5M13008-2	
2.5	016	Washer DIN 7980 8	7P08-2	
2.5	017	Nut DIN 934 M6	6008-2	
2.5	018	Ring	AEGA 116.77.00.001	

Connection of cuspidor unit

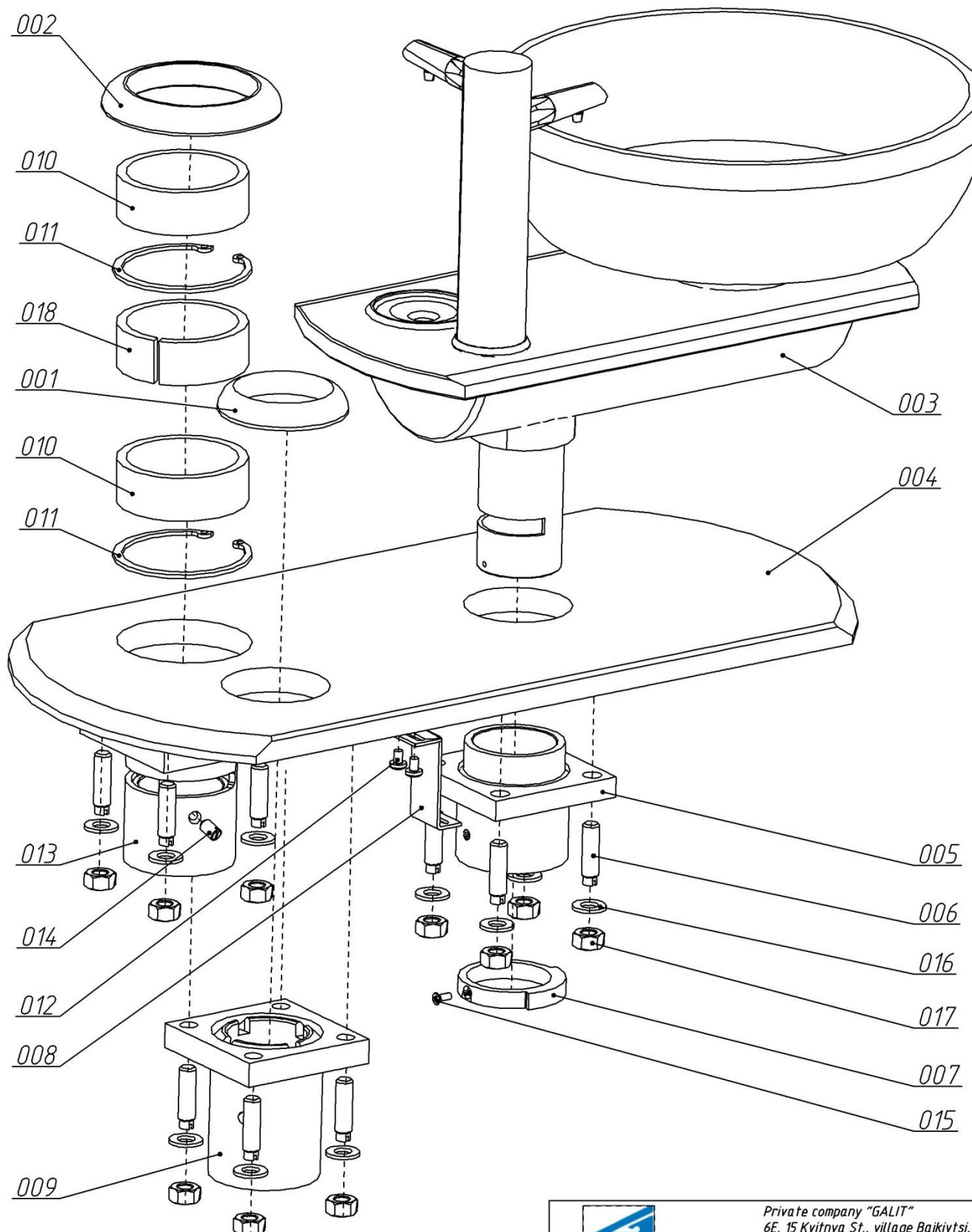


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.5

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.6	001	Insert	AEGA 916.00.00.005	
2.6	002	Glass Bowl	Tecnocer c.r.e	
2.6	003	Nut	AEGA 916.00.00.002	
2.6	004	Panel	AEGA 916.00.00.007	
2.6	005	Insert	AEGA 916.00.00.005-01	
2.6	006	Spout	AEGA 916.00.00.014	
2.6	007	Cup	AEGA 916.00.00.006	
2.6	008	Uprise	AEGA 916.01.00.000	
2.6	009	Ring	AEGA 916.00.00.012	
2.6	010	Nipple	AEGA 916.00.00.009	
2.6	011	Cup Holder	AEGA 916.00.00.003	
2.6	012	Nipple	AEGA 016.09.00.002	
2.6	013	Case	AEGA 916.00.00.001	
2.6	014	Enclosure	AEGA 916.00.00.004	
2.6	015	Mouth	AEGA 916.00.00.010	
2.6	016	O-ring	008-012-25 GOST 9833-73	
2.6	017	O-ring	045-050-30 GOST 9833-73	
2.6	018	Panel	AEGA 916.02.00.000	
2.6	019	Screw DIN 912 M5x8	5155008-2	
2.6	020	Screw DIN 7985 M4x10	5M54010-2	
2.6	021	Screw DIN 913 M4X8	984008-P	
2.6	022-01		RI07051	
2.6	022-02	Cuspidor screen with gray lid	VCF-353-02	

Cuspidor unit

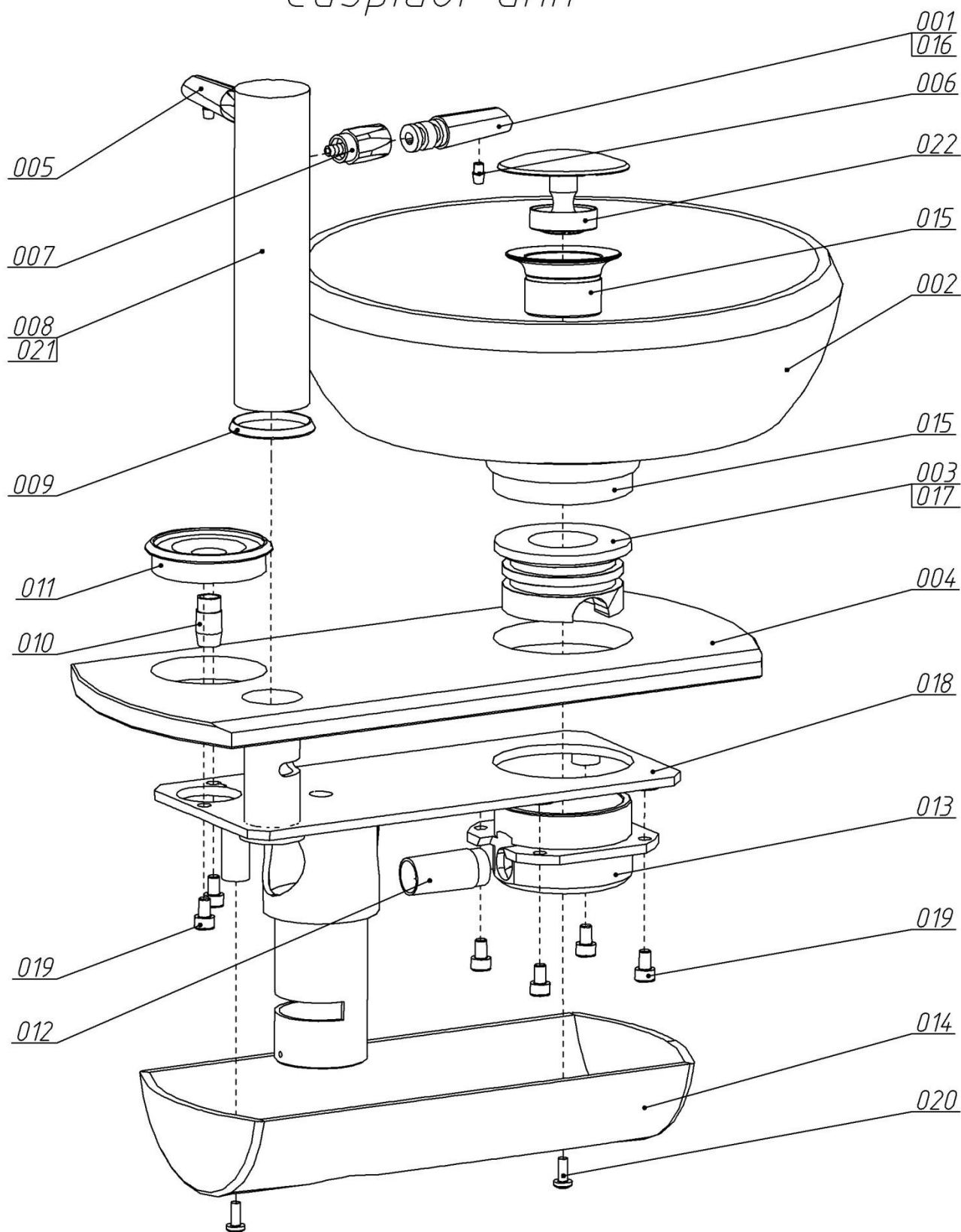


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.6

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.7	001	Pressing	AEGA 016.85.00.007	
2.7	002	Amp connector	DB-9M	
2.7	003	Jackplug	DB-9F	
2.7	004	Fitting	1500 8/6-1/8	
2.7	005	Fitting	2531 1/2-1/8	
2.7	006	Minifaucet	1/2" B3	
2.7	007	Fitting	1590 6/4	
2.7	008	Bushing	AEGA 8.227.001	
2.7	009	Screw	AEGA 216.00.00.009	
2.7	010	Fitting	1500 6/4-1/8	
2.7	011	Nut DIN 934 M3	6003-2	
2.7	012	Washer DIN 125 4	7004-2	
2.7	013	Nut DIN 934 M4	6004-2	

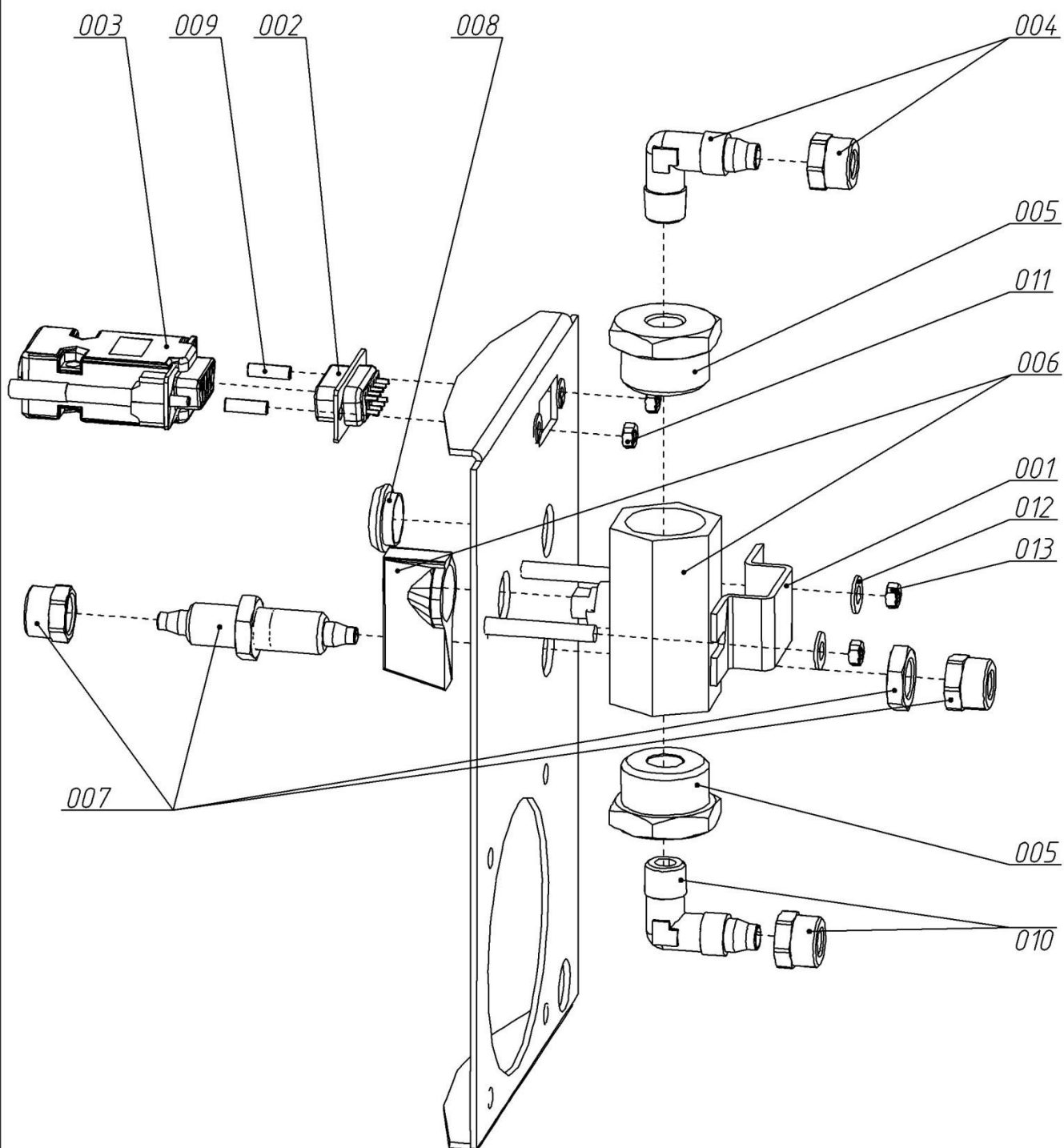


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:


Dental Unit "GALLANT F"
List 2.7

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.8	001	Bracket	AEΓA 016.00.00.063	
2.8	002	Holder	AEGA 116.86.00.000	
2.8	003	Strip	AEGA 116.00.00.076	
2.8	004	Fitting	FIF-617-01 1/16	
2.8	005	Fitting	FIF-618-01 1/8	
2.8	006	Washer Ø5mm	FIF-607-01	
2.8	007	Electric valve RPE	2105NC24dc	
2.8	008	Fitting	2531 1/8-M5	
2.8	009	Micro solenoid valve	V165V01-Z031A M5 Ø2 24V DC	
2.8	010	Screw DIN 912 M3x8	5153008-2	
2.8	011	Screw DIN 912 M4x25	5154025-2	
2.8	012	Screw DIN 912 M4x10	5154010-2	
2.8	013	Nut DIN 934 M4	6004-2	
2.8	014	Coupling	FIF-605-01	
2.8	015	Coupling	FIF-605-02	

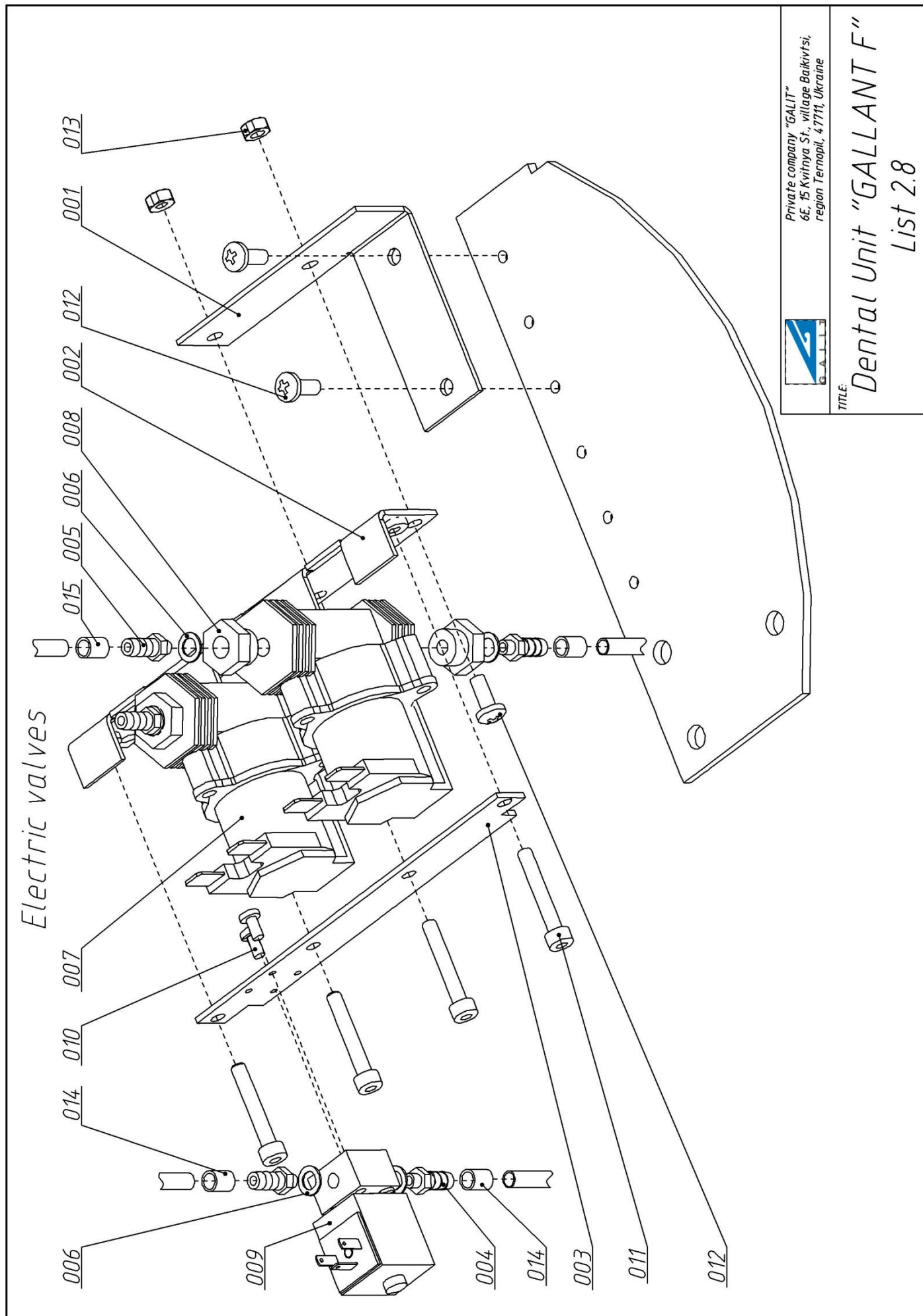

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.11	001	Bracket	AEGA 116.95.00.001	
2.11	002	Rubber washer	AEGA 116.95.00.002	
2.11	003	Washer	AEGA 016.00.00.030	
2.11	004	Transformer	TST 300/041	
2.11	005	Fuse-holder	TLF-100-3	
2.11	006	Interplanar Strut	RI-LCBS-TFM4-8	
2.11	007	Screw DIN 7985 M4x8	5M54008-2	
2.11	008	Bolt DIN 933 M6x80	5M66080-2	
2.11	009	Screw DIN 7985 M3x16	5M53016-2	
2.11	010	Controller	A120M	
2.11	011	Washer DIN 125 6	7O06-2	
2.11	012	Nut DIN 934 M6	6006-2	
2.11	013	Fuse		

Transformer and electric parts

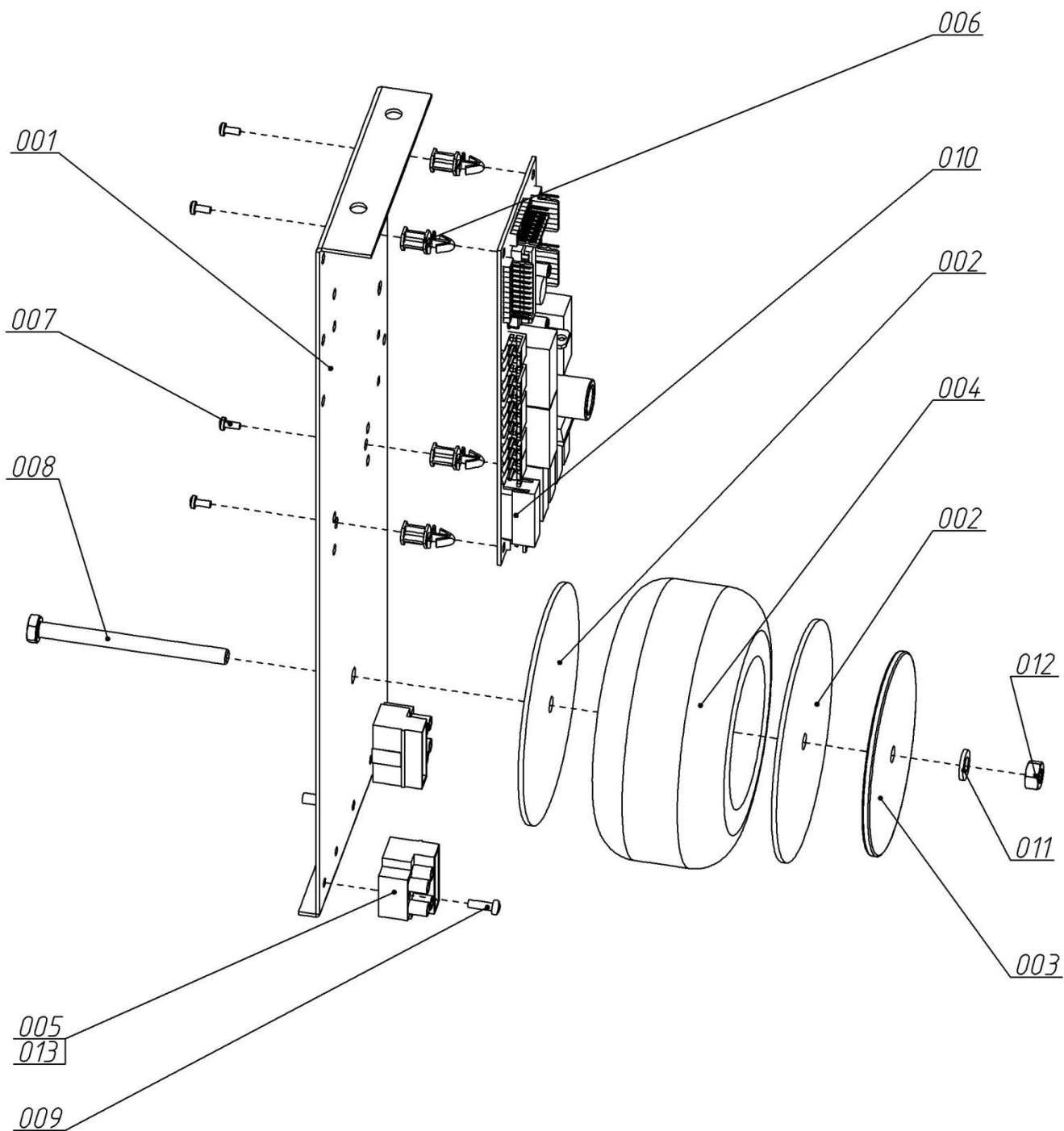


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.11


LIST	№	DESCRIPTION	CODE KD	CODE GALIT
2.12	001	Arm	GAL 043.01.00.000	
2.12	002	Axis	GAL 043.00.00.001	
2.12	003	Axis	GAL 043.00.00.002	
2.12	004	Ring	GAL 043.00.00.003	
2.12	005	Ball socket adjuster	GAL 043.00.00.004	
2.12	006	Holder	GAL 043.00.00.005	
2.12	007	Axis	GAL 043.00.00.006	
2.12	008	Nut	GAL 043.00.00.007	
2.12	009	plate	GAL 043.00.00.008	
2.12	010	Washer	GAL 043.00.00.009	
2.12	011	Washer	GAL 043.00.00.010	
2.12	012	Ring	GAL 043.00.00.011	
2.12	013	Blank	GAL 043.00.00.012	
2.12	014	Blank	GAL 043.00.00.012-01	
2.12	015	Screw DIN 914 M8x10	988010-H	
2.12	016	Screw DIN 915 M6x6	986006-C	
2.12	017	Screw DIN 915 M6x10	986010-C	
2.12	018	Screw DIN 965/963 M5x12	5M15012-2	

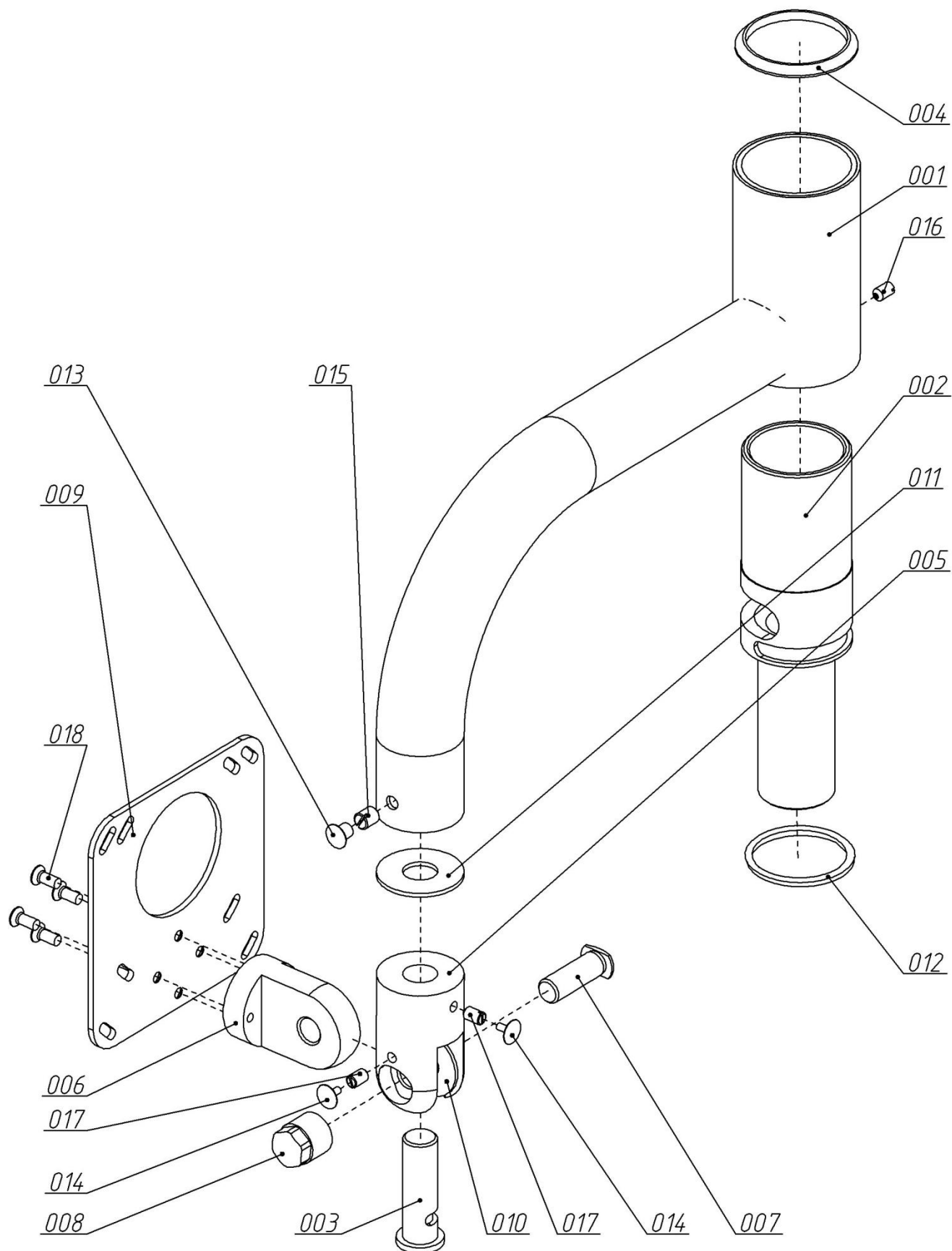

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:
Dental Unit "GALLANT H"
List 2.12

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
2.13	001	Bracket	AEGA 146.01.00.000	
2.13	002	Pipe	AEGA 146.00.00.001	
2.13	003	Bushing	AEGA 146.00.00.002	
2.13	004	Bushing	AEGA 146.00.00.003	
2.13	005	Cover	AEGA 146.00.00.004	
2.13	006	Cover	AEGA 146.00.00.005	
2.13	007	Axis	AEGA 146.00.00.007	
2.13	008	Screw DIN 915 M6x10	986010-C	
2.13	009	Screw DIN 965/963 M4x16	5M14016-2	
2.13	010	Stop ring A24 GOST 13940-68	95PK1024	

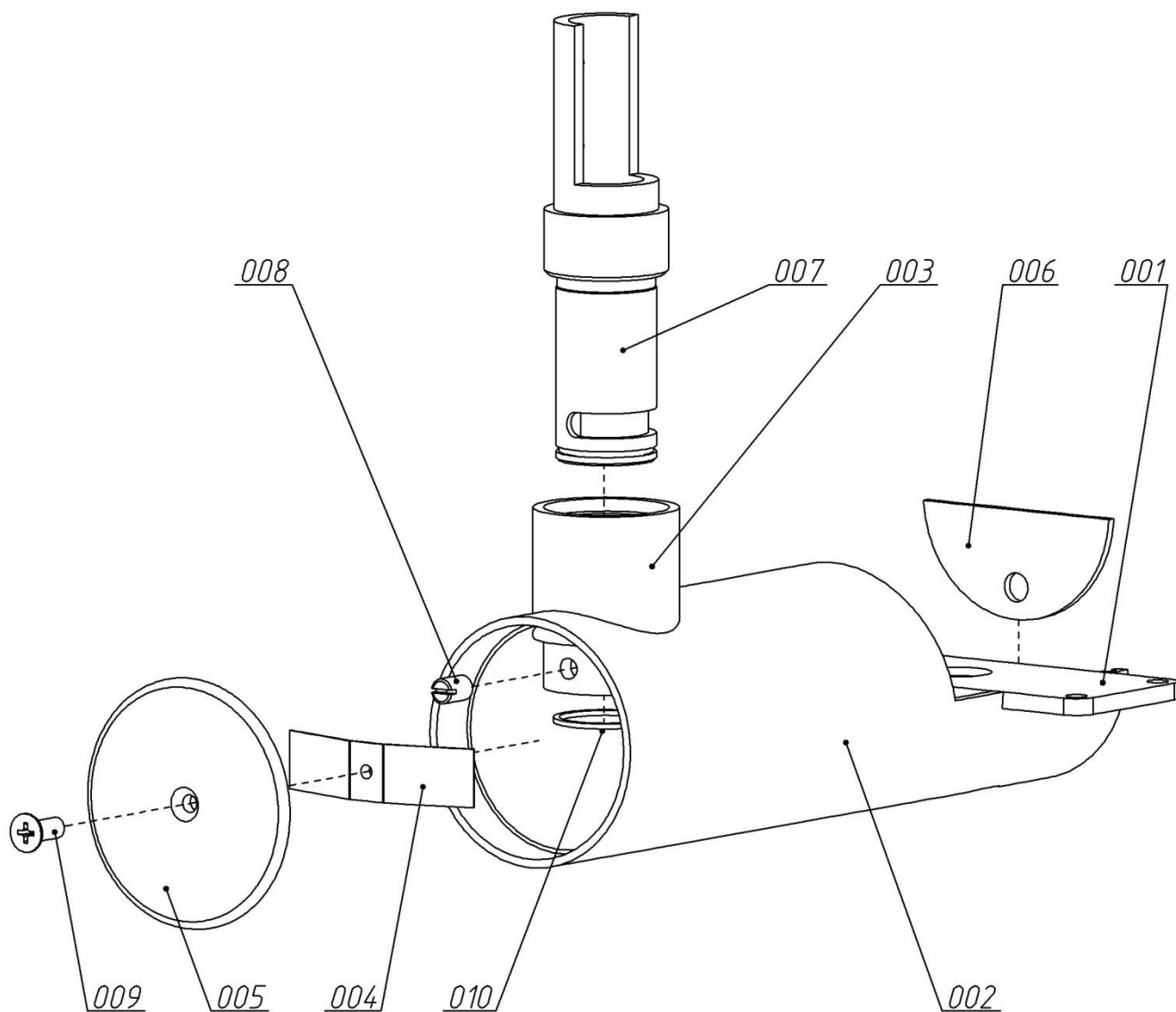


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:
Dental Unit "GALLANT H"
List 2.13

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
2.14	001	Bracket	AEGA 216.00.00.021	
2.14	002	Bleed	AEGA 016.03.00.001-02	
2.14	003	Suction	AEGA 016.03.00.002	
2.14	004	Case	AEGA 016.03.00.003	
2.14	005	Trap	AEGA 016.03.00.004	
2.14	006	Sleeve	FOOD 10+metallic fiber	
2.14	007	O-Ring	2661 M5	
2.14	008	O-Ring	2661 M6	
2.14	009	O-Ring	2661 1/8	
2.14	010	Buckle	10-16 ГОСТ 28191-89	
2.14	011	Fitting	2013 1/8	
2.14	012	Screw DIN 7985 M4x8	5M54008-2	

Water bleed

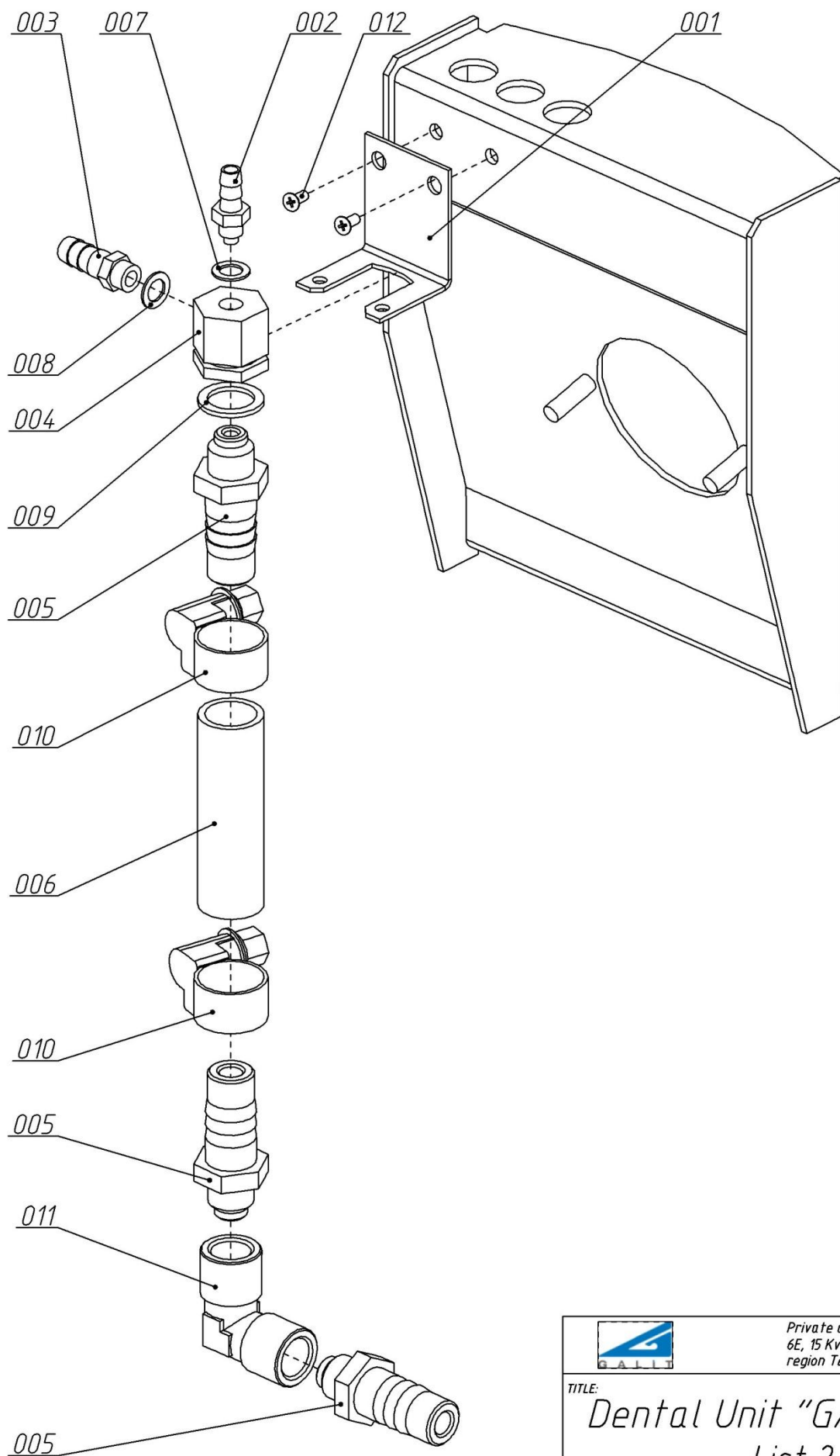


Private company "GALLIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:
Dental Unit "GALLANT F"
List 2.14

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.15	001	Bracket	AEGA 216.00.00.021	
2.15	002	Bleed	AEGA 016.03.00.001-01	
2.15	003	Suction	AEGA 016.03.00.002	
2.15	004	Case	AEGA 016.03.00.003	
2.15	005	Trap	AEGA 016.03.00.004	
2.15	006	O-Ring	2661 M6	
2.15	007	O-Ring	2661 M5	
2.15	008	O-Ring	2661 1/8	
2.15	009	Screw DIN 967 M4x8	5M54008-2	

Air bleed

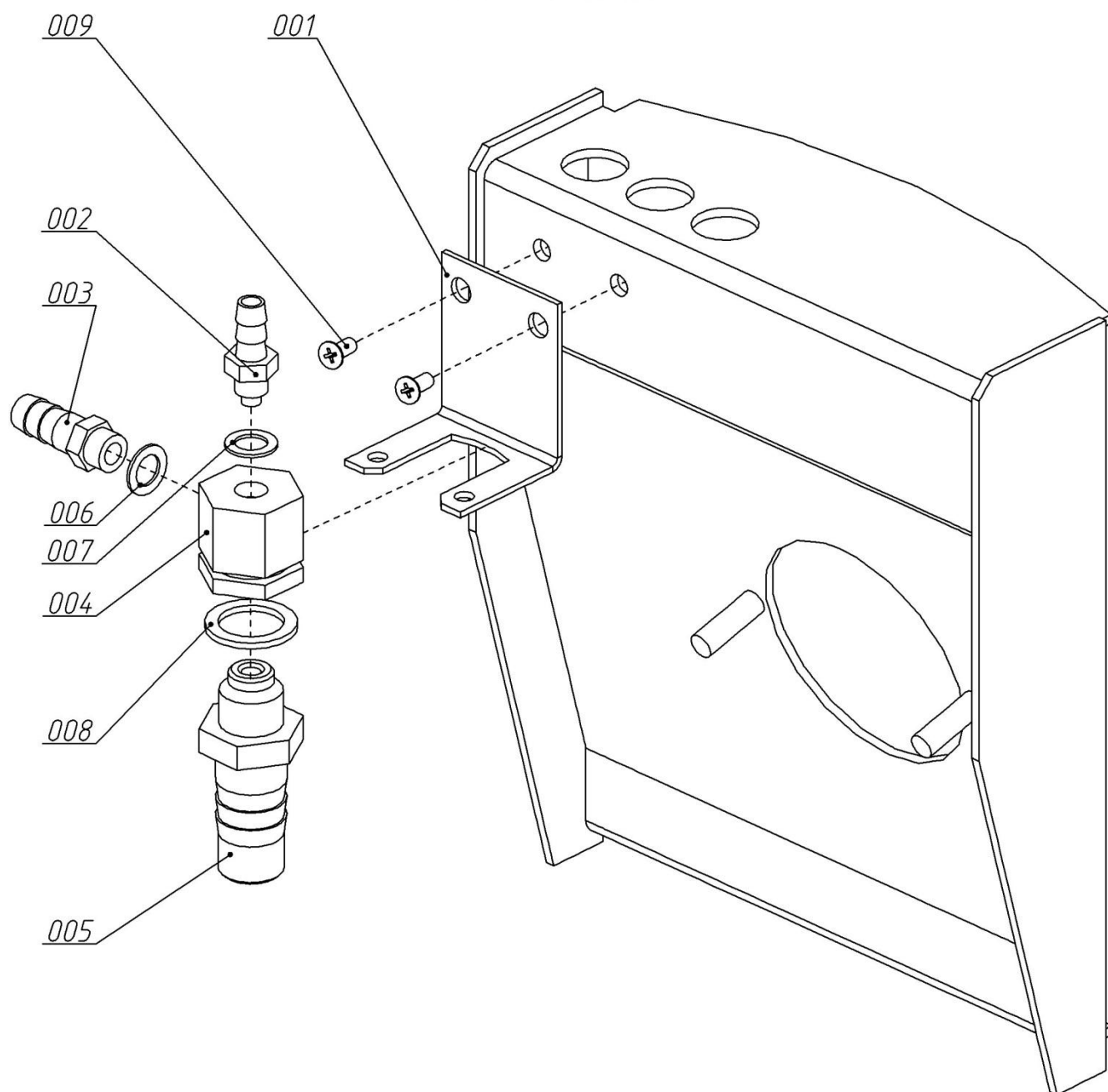


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.15

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.16	001	Bracket	AEGA 016.00.00.024	
2.16	002	Filter-regulator	N108-D00	
2.16	003	Fitting	1500 6/4-1/8	
2.16	004	Fitting	2500 1/8	
2.16	005	Fitting	2070 1/8-1/8	
2.16	006	Fitting	1500 8/6-1/8	
2.16	007	Fitting	FIF-617-01 1/16	
2.16	008	Fitting	2531 1/8-M5	
2.16	009	Fitting	FIF-618-01 1/8	
2.16	010	Filter-regulator	N108-D01-C01	
2.16	011	Washer Ø5mm	FIF-607-01	
2.16	012	Micro-regulator	M008-R00	
2.16	013	Fitting	2611 1/8	
2.16	014	Coupling	FIF-605-01	
2.16	015	Coupling	FIF-605-02	

Filters-Version with Cattani Separator add Basic version

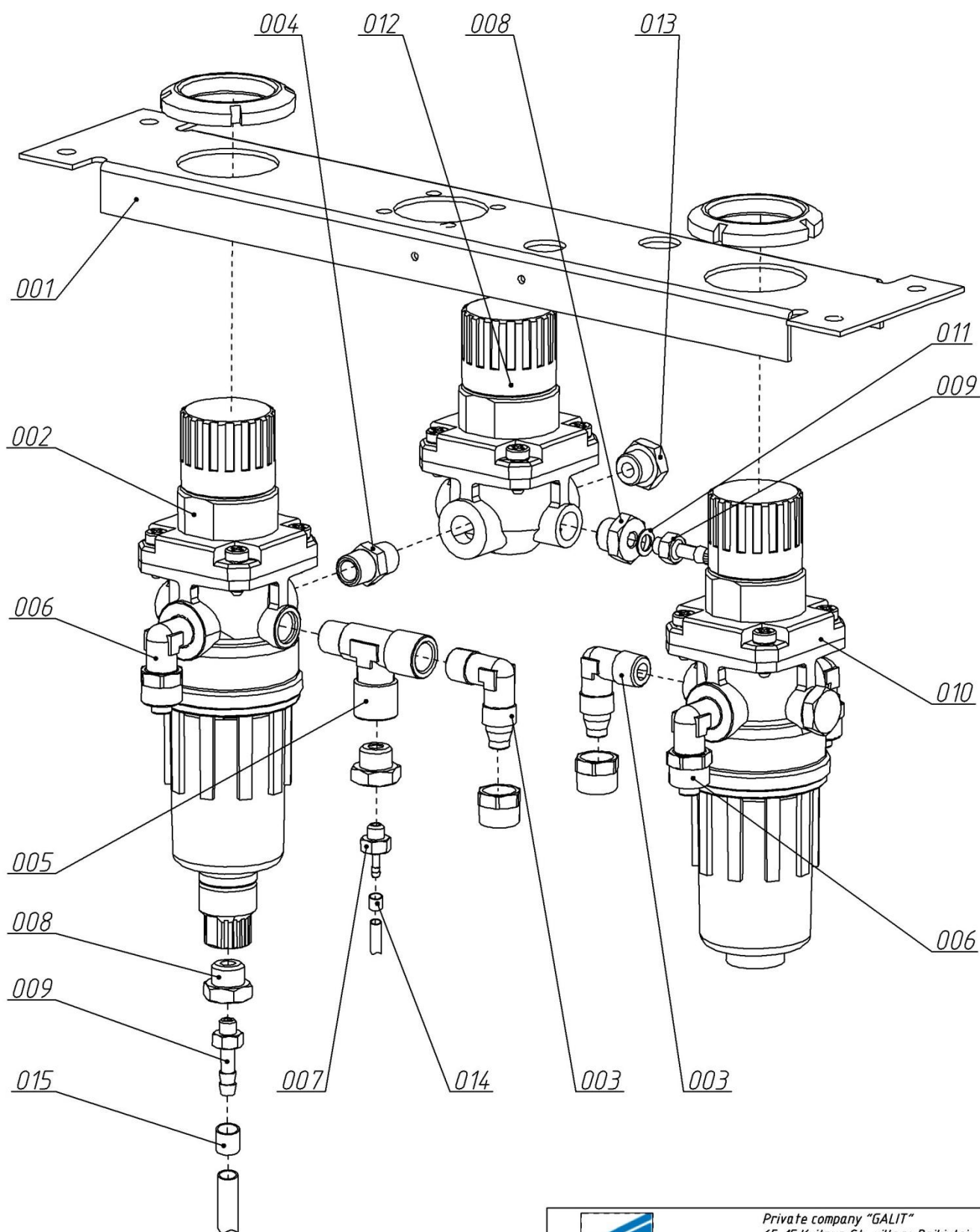


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.16

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.17	001	Bracket	AEGA 216.00.00.020	
2.17	002	Micro-regulator	M008-R00	
2.17	003	Filter-regulator	N108-D00	
2.17	004	Filter-regulator	N108-D01-C01	
2.17	005	Screw DIN 967 M4x8	5M54008-2	
2.17	006	Nut DIN 934 M4	6004-2	

Filters-Version with CAS-1, CS-1 and spittoon bowl valve

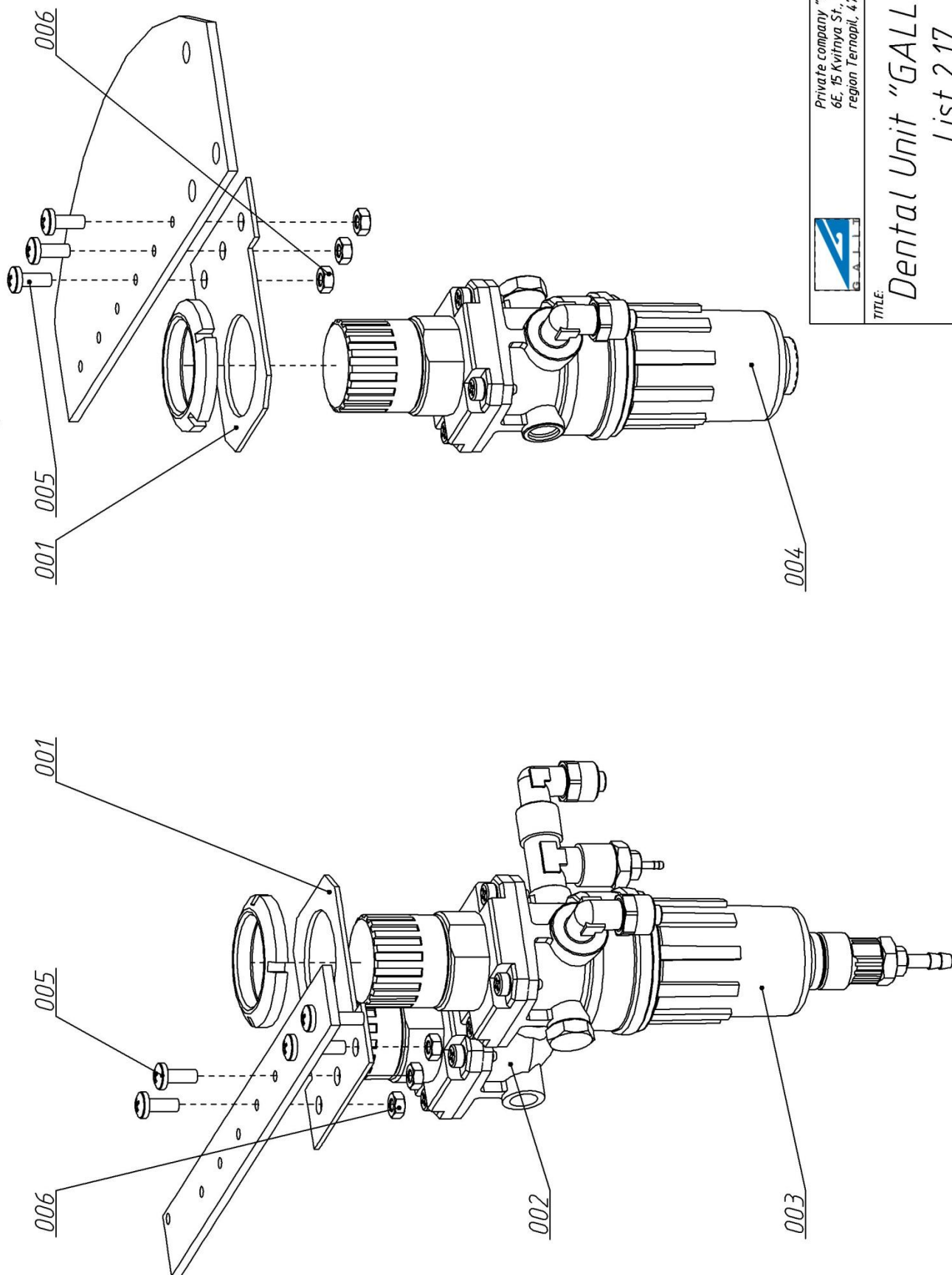

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine


TITLE:

Dental Unit "GALLANT F"

List 2.17

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.18	001	Bracket	AEGA 016.00.00.024	
2.18	002	Socket	0700-720-18	
2.18	003	Socket	0700-720-10	
2.18	004	Elbow 90°	0700-700-04	
2.18	005	T-Joint	0700-700-02	
2.18	006	Securing ring	0700-700-01	
2.18	007	Securing ring	0700-700-09	
2.18	008	Double socket	0700-700-14	
2.18	009	Coarse filter	0700-702-52	
2.18	010	Extension piece	0700-700-17	
2.18	011	Clamp	0700-700-54	
2.18	012	T-Joint	0700-700-08	
2.18	013	Blank	0700-700-10	
2.18	014	Socket	0700-720-17	
2.18	015	Screw DIN 912 M3x30	5153030-2	

Basic version

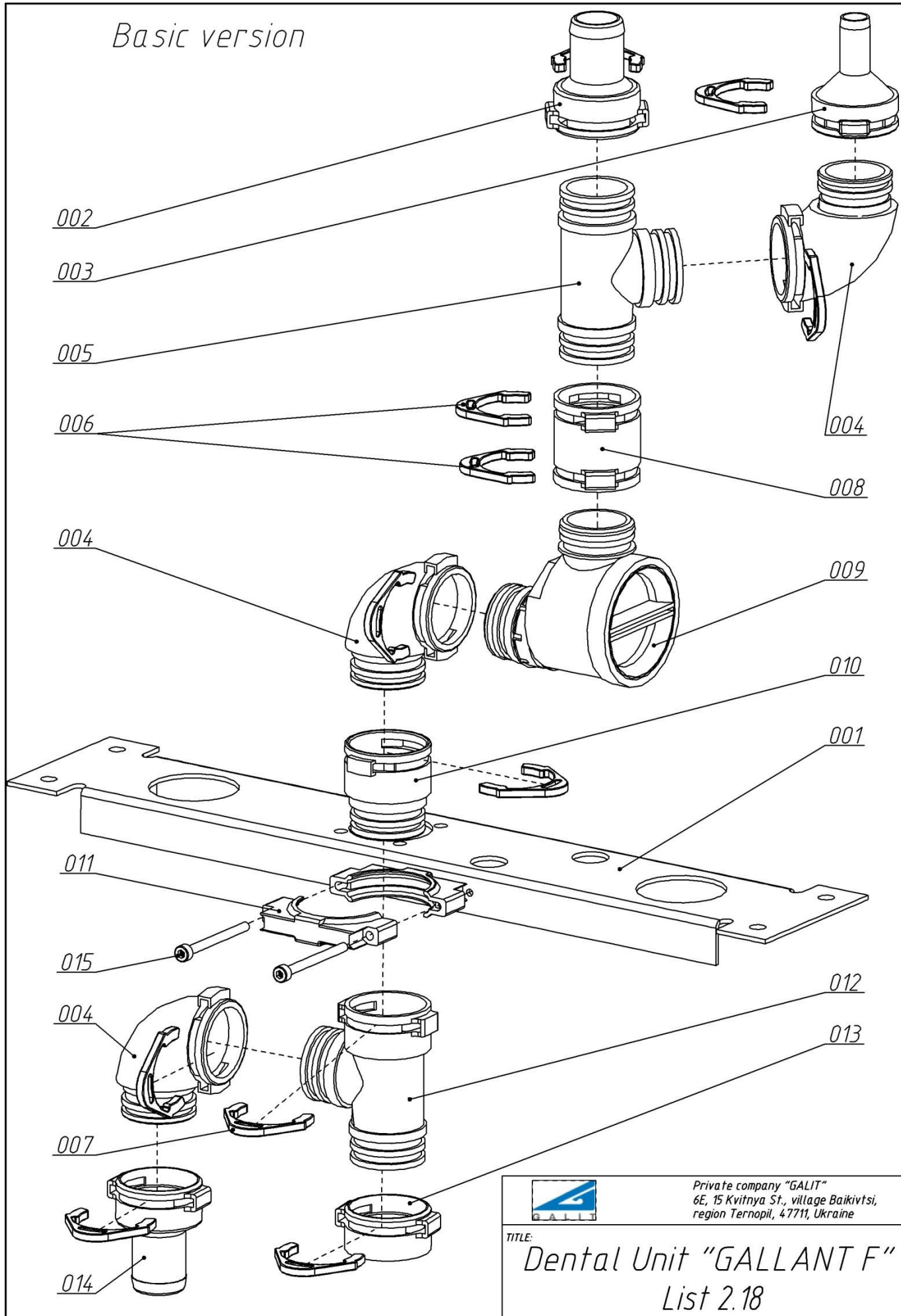


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.18

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.19	001	Bracket	AEGA 016.00.00.024	
2.19	002	Nipple	AEGA 116.00.00.060	
2.19	003	Case	AEGA 116.96.00.000	
2.19	004	Separator Cattani	020973	
2.19	005	Elbow 90°	0700-700-04	
2.19	006	Securing ring	0700-700-01	
2.19	007	Securing ring	0700-700-09	
2.19	008	Socket	0700-720-18	
2.19	009	T-Joint	0700-700-02	
2.19	010	Double socket	0700-700-14	
2.19	011	Coarse filter	0700-702-52	
2.19	012	Extension piece	0700-700-17	
2.19	013	T-Joint	0700-700-08	
2.19	014	Blank	0700-700-10	
2.19	015	Clamp	0700-700-54	
2.19	016	Grey Pipe	Ø17.5 Cattani	
2.19	017	Buckle	16-25 GOST 28191-89	
2.19	018	Screw DIN 912 M3x30	5153030-2	
2.19	019	Socket	0700-720-10	
2.19	020	Socket	0700-720-17	

Version with Cattani Separator

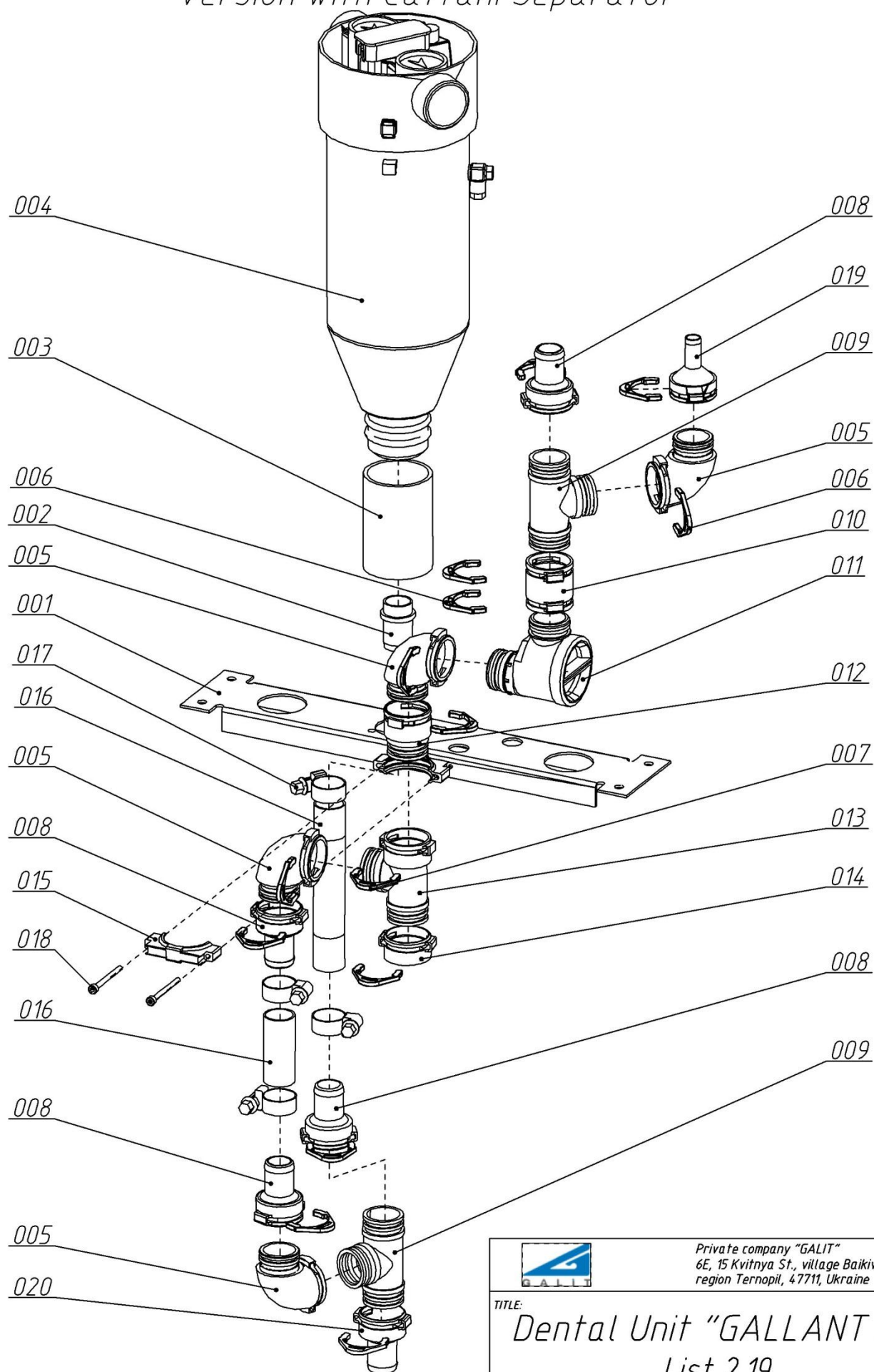

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine


TITLE:

Dental Unit "GALLANT F"
List 2.19

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.20	001	Separator Cattani	020973	
2.20	002	Socket*	3923A	
2.20	003	Sleeve lignum Ø30*		
2.20	004	Socket*	0700-720-30	
2.20	005	Secretion filter	0725-040-50	
2.20	006	Bushing	DFG 7.860.004	
2.20	007	Securing ring	0700-700-01	
2.20	008	Blank	0700-700-10	
2.20	009	Plug	0700-700-21	
2.20	010	Double connector	0725-041-05	
2.20	011	Place selector valve	7560-500-60	

* - It is possible to use Socket 0700-720-25. socket 3923B and Sleeve lignum Ø25.

Version with Separator Cattani

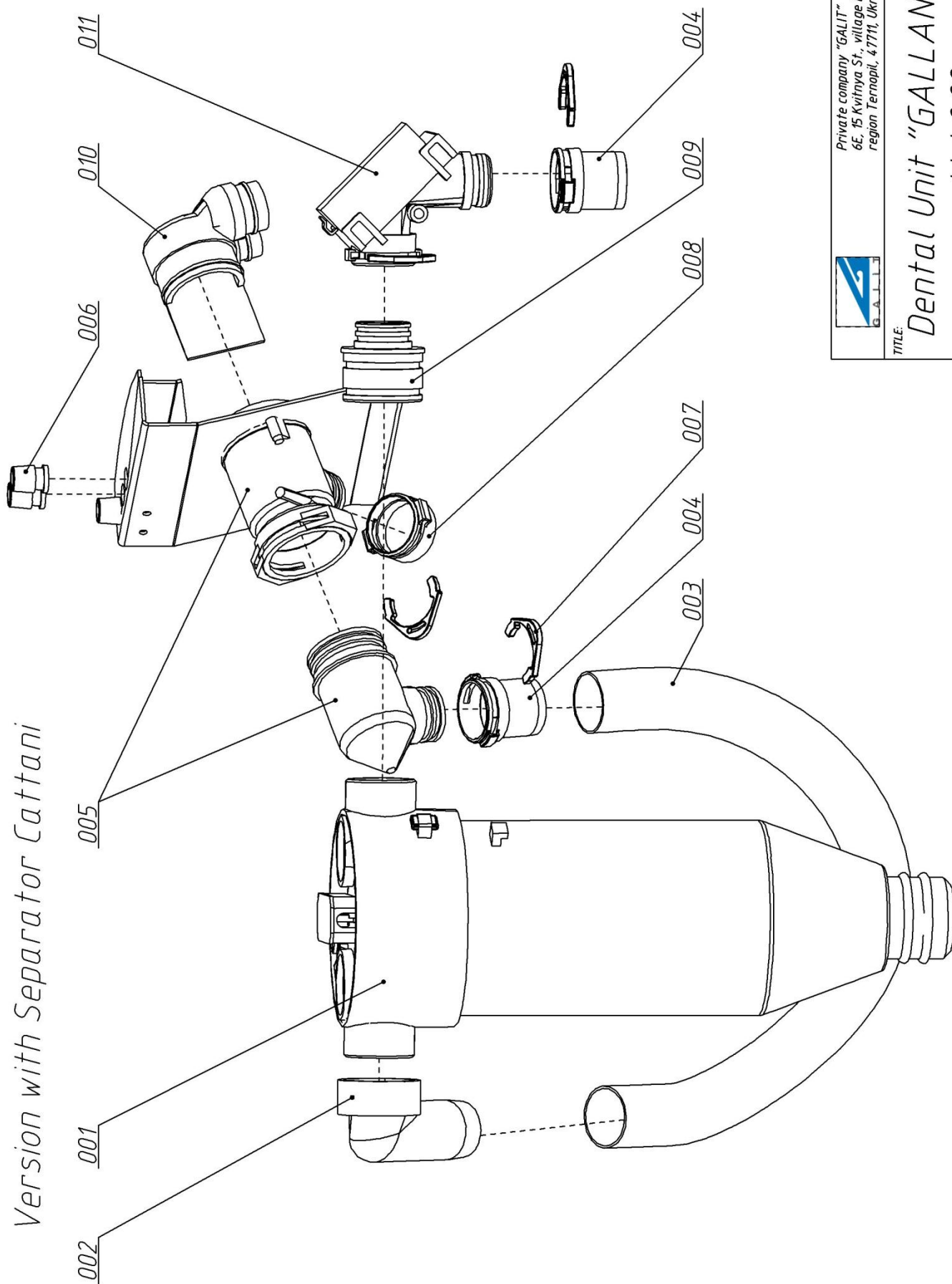


Private company "GALLANT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.20

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.21	001	Bracket	AEGA 216.00.00.019	
2.21	002	Separator CAS-1	7117-23	
2.21	003	Bushing	DFG 7.860.004	
2.21	004	Nut DIN 934 M4	6004-2	
2.21	005	Screw DIN 912 M4x10	5154010-2	
2.21	006	Screw DIN 912 M4x20	5154020-2	

Version with Separator CAS-1

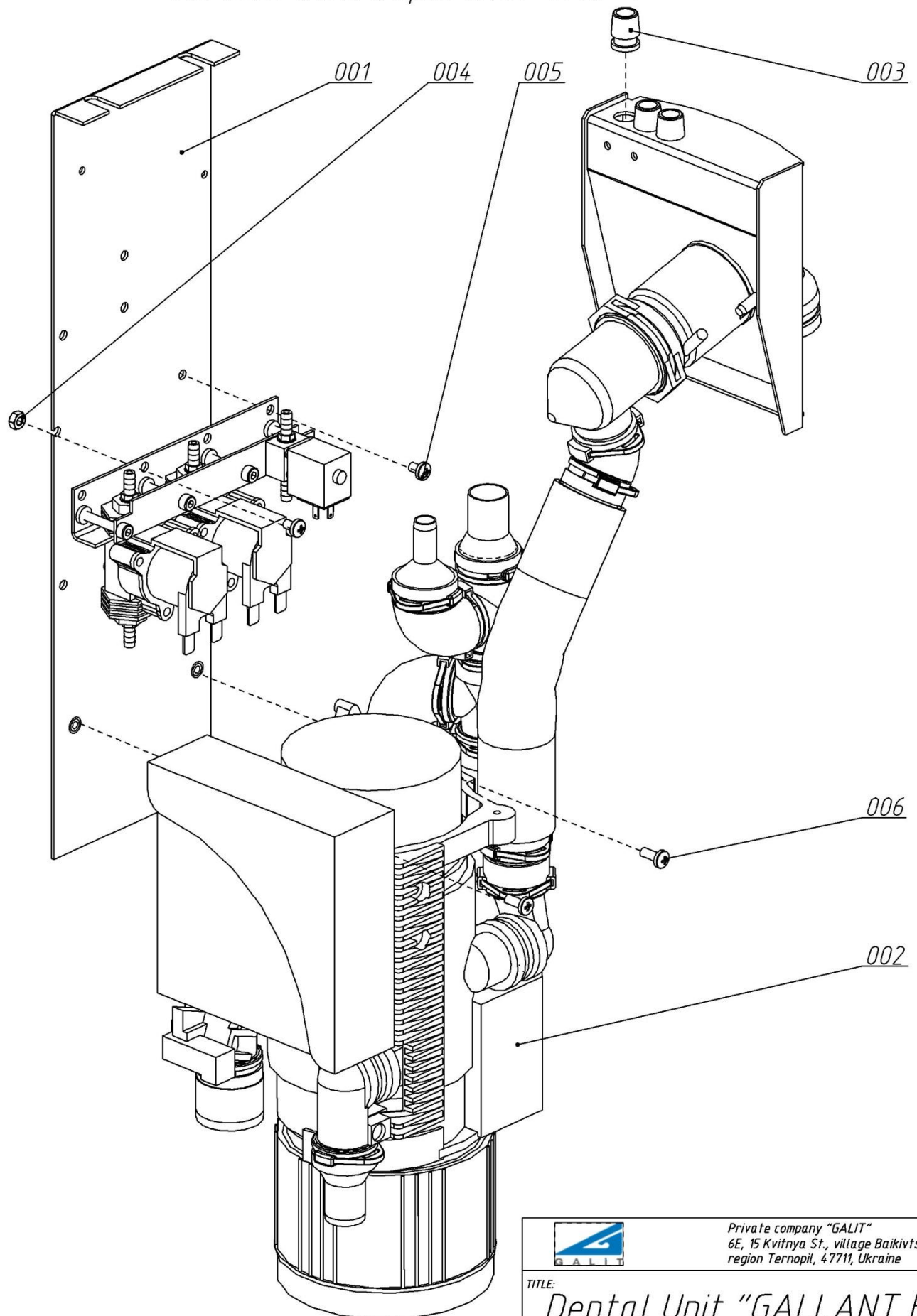

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine


TITLE:

Dental Unit "GALLANT F"
List 2.21

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.22	001	Secretion filter	0725-040-50	
2.22	002	Double connector	0725-041-05	
2.22	003	Blank	0700-700-10	
2.22	004	Securing ring	0700-700-01	
2.22	005	Elbow 45°	0700-700-05	
2.22	006	Socket*	0700-720-30	
2.22	007	Sleeve lignum Ø30		
2.22	008	Self tapping screw DIN 96 3x12	1023012-2	

* - It is possible to use Socket 0700-720-25.

Version with Separator CAS-1

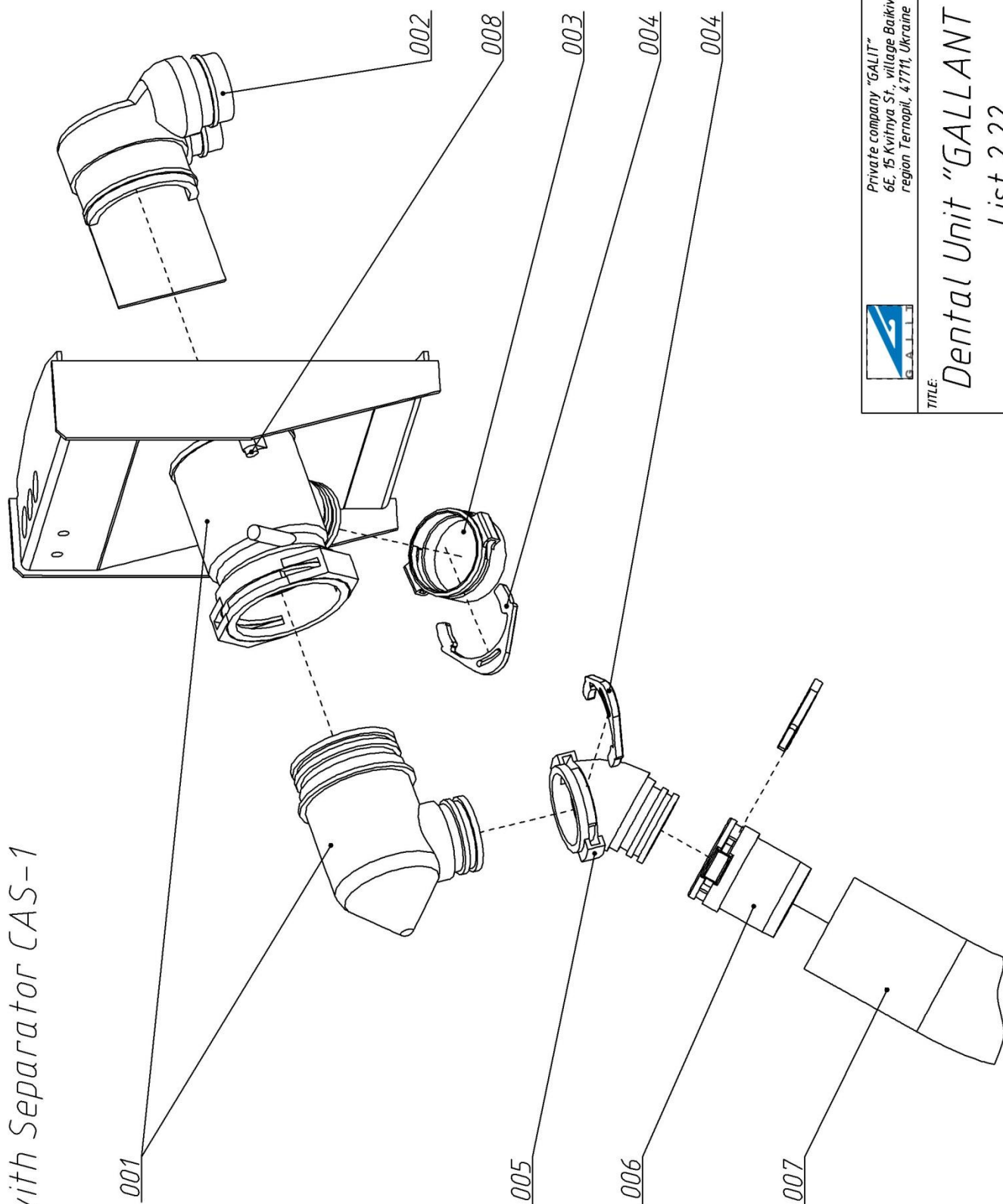

Private company "GALLANT"
6E, 15 Kvintya St., village Balkivtsi,
region Ternopil, 47711, Ukraine


TITLE:

Dental Unit "GALLANT F"
List 2.22

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
2.23	001	Separator CAS-1	7117-23	
2.23	002	Socket	0700-720-10	
2.23	003	Elbow 90°	0700-700-04	
2.23	004	Securing ring	0700-700-01	
2.23	005	Socket	0700-720-17	
2.23	006	Socket*	0700-720-30	
2.23	007	Socket	0700-720-18	
2.23	008	T-Joint	0700-700-02	
2.23	009	T-Joint	0700-700-08	
2.23	010	Double socket	0700-700-14	
2.23	011	Blank	0700-700-10	

* - It is possible to use Socket 0700-720-25.

Version with Separator CAS-1

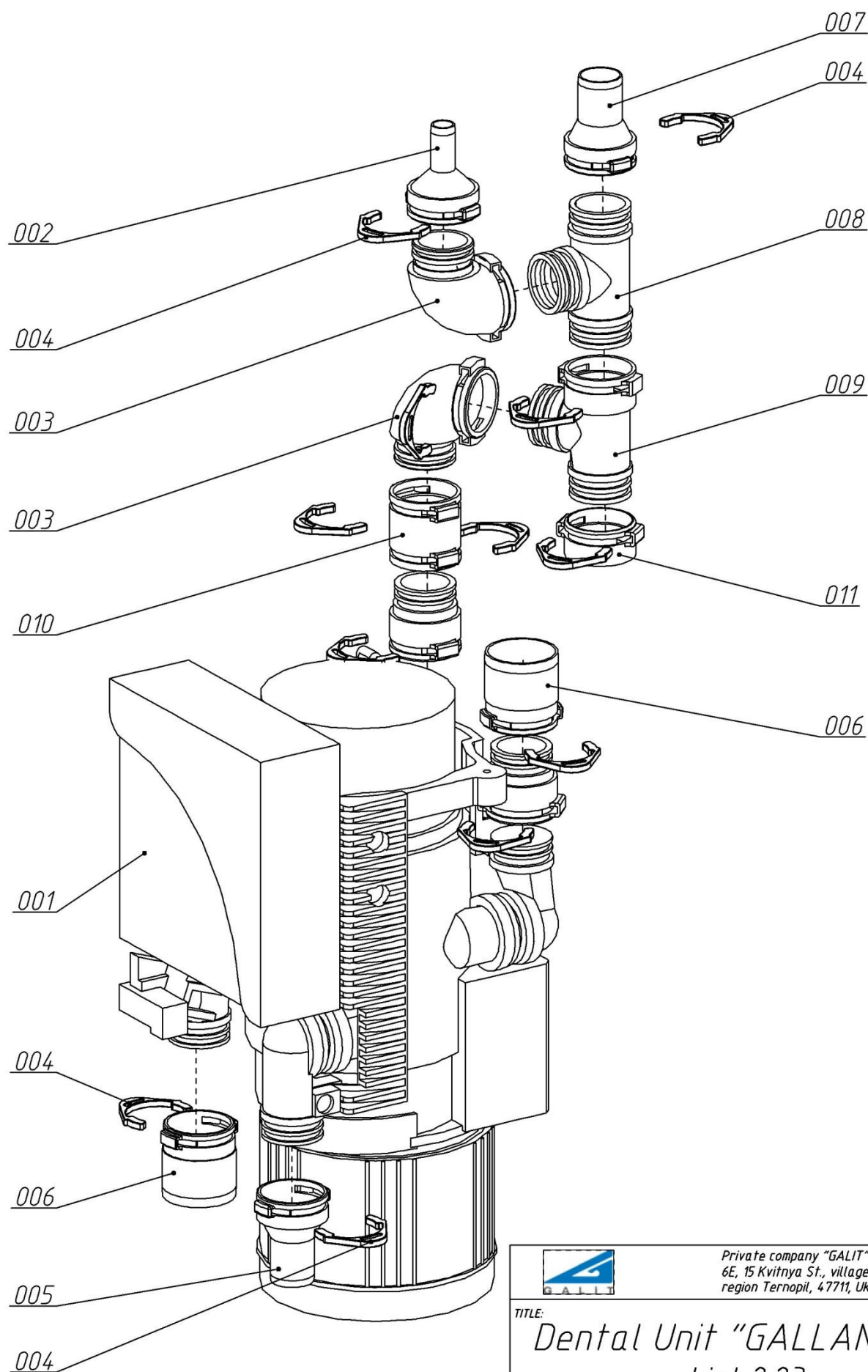


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:
Dental Unit "GALLANT F"
List 2.23

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.24	001	Bracket	AEGA 016.00.00.024	
2.24	002	Bracket	AEGA 216.05.00.001	
2.24	003	Double connector	0725-041-05	
2.24	004	Blank	0700-700-10	
2.24	005	Secretion filter	0725-040-00	
2.24	006	Place selector valve	7560-500-60	
2.24	007	Elbow 90°	0700-700-04	
2.24	008	Sleeve lignum Ø30*		
2.24	009	Spittoon bowle valve	7560-500-52	
2.24	010	Socket	0700-720-18	
2.24	011	Socket	0700-720-10	
2.24	012	T-Joint	0700-700-02	
2.24	013	Double socket	0700-700-14	
2.24	014	Double socket	0700-700-11	
2.24	015	Extension piece	0700-700-17	
2.24	016	Socket*	0700-720-30	
2.24	017	Securing ring	0700-700-01	
2.24	018	Mini regulating device	VAF-532-01	
2.24	019	Washer Ø5mm	FIF-607-01	
2.24	020	Fitting	FIF-617-01	
2.24	021	Screw DIN 912 M4x10	5154010-2	

* - It is possible to use Socket 0700-720-25 and Sleeve lignum Ø25.

Version with spittoon bowl valve

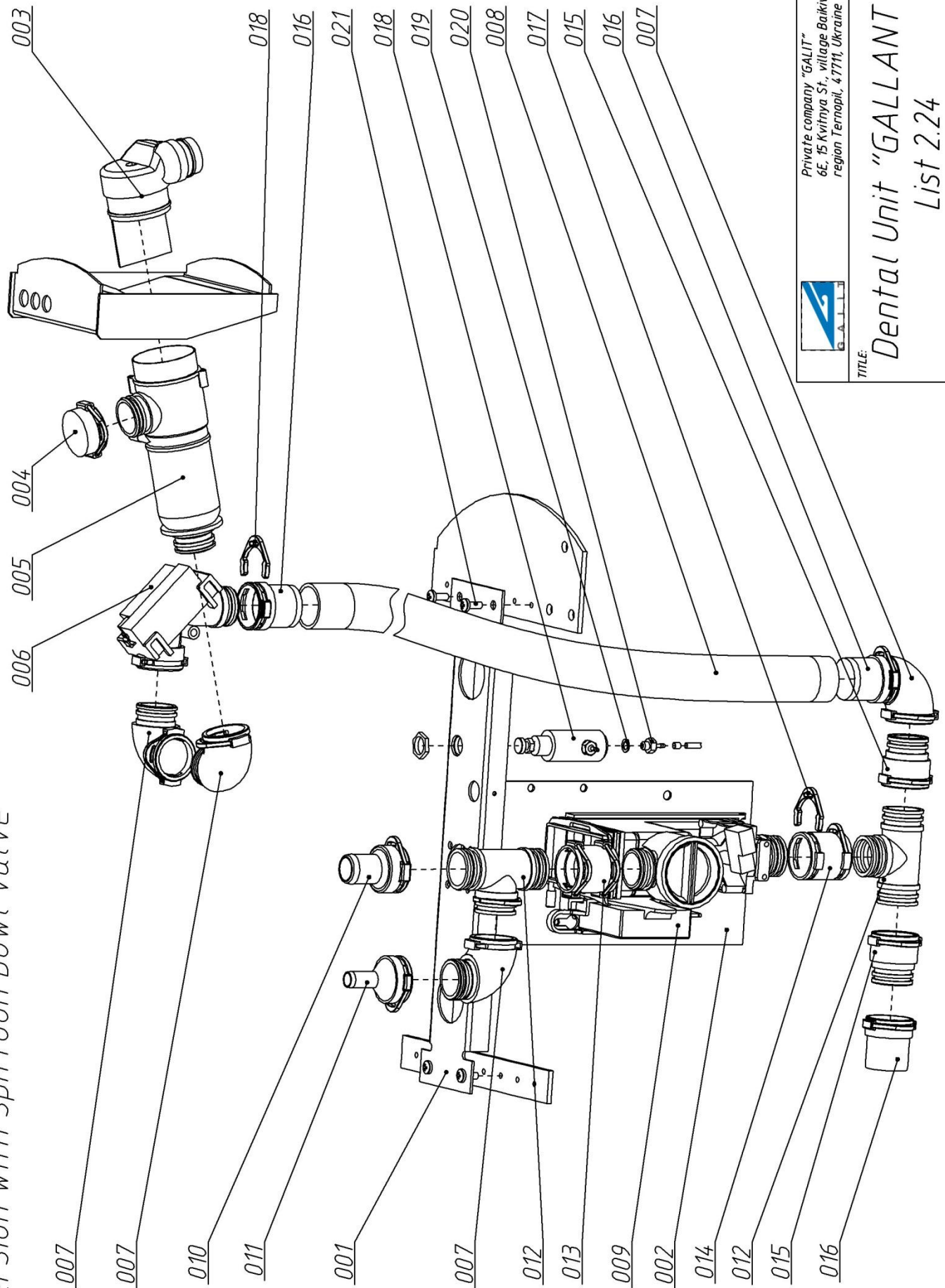

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine




TITLE:

Dental Unit "GALLANT F"

List 2.24

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.25	001	Cup	AEGA 116.75.00.000	
2.25	002	Cup	AEGA 116.77.00.000	
2.25	003	Cuspidor	AEGA 716.05.00.000	
2.25	004	Cup	AEGA 6.210.002	
2.25	005	Cramp	AEGA 916.00.0.013	
2.25	006	Clamp	GAL 268.00.00.001	
2.25	007	Stub	AEGA 216.00.00.011	
2.25	008	Cam	AEGA 8.360.001	
2.25	009	Upper plate	AEGA 8.076.015	
2.25	010	Plate	AEGA 7.840.003	
2.25	011	Screw DIN 913 M4x6	984006-P	
2.25	012	Screw DIN 912 M6x10	5156010-2	
2.25	013	Screw DIN 912 M8x20	5158020-2	
2.25	014	Screw DIN 965/963 M3x8	5M14008-2	
2.25	015	Nut DIN 934 M8	6008-2	
2.25	016	Washer DIN 125 8	7008-2	
2.25	017	Bushing	DU5025	
2.25	018	Washer	WC52	
2.25	019-01		RI07051	
2.25	019-02	Cuspidor screen with gray lid	VCF-353-02	

Cuspidor block

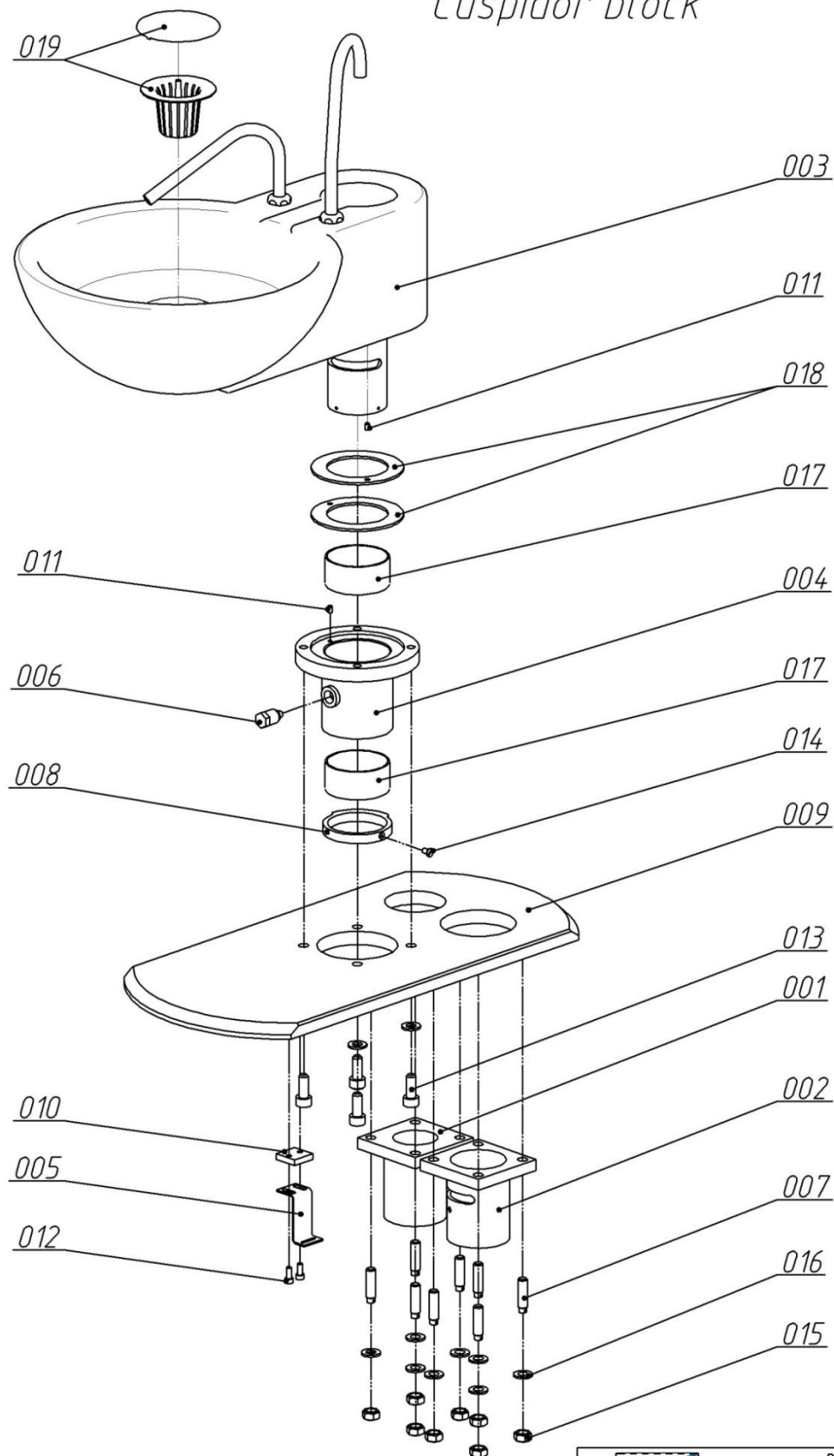


Private company "GALIT"
6E, 15 Kvintya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.25

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
2.26	001	Clamp	AEGA 8.090.097	
2.26	002	Jackplug	DB-9M	
2.26	003	Socket	DB-9F	
2.26	004	Fitting	1500 8/6-1/8	
2.26	005	Minifaucet	A202 1/8-1/8	
2.26	006	Fitting	1590 6/4	
2.26	007	Bushing	AEGA 8.227.001	
2.26	008	Screw	AEGA 216.00.00.009	
2.26	009	Fitting	1500 6/4-1/8	
2.26	010	Nut DIN 934 M3	6003-2	
2.26	011	Washer DIN 125 4	7004-2	
2.26	012	Nut DIN 934 M4	6004-2	

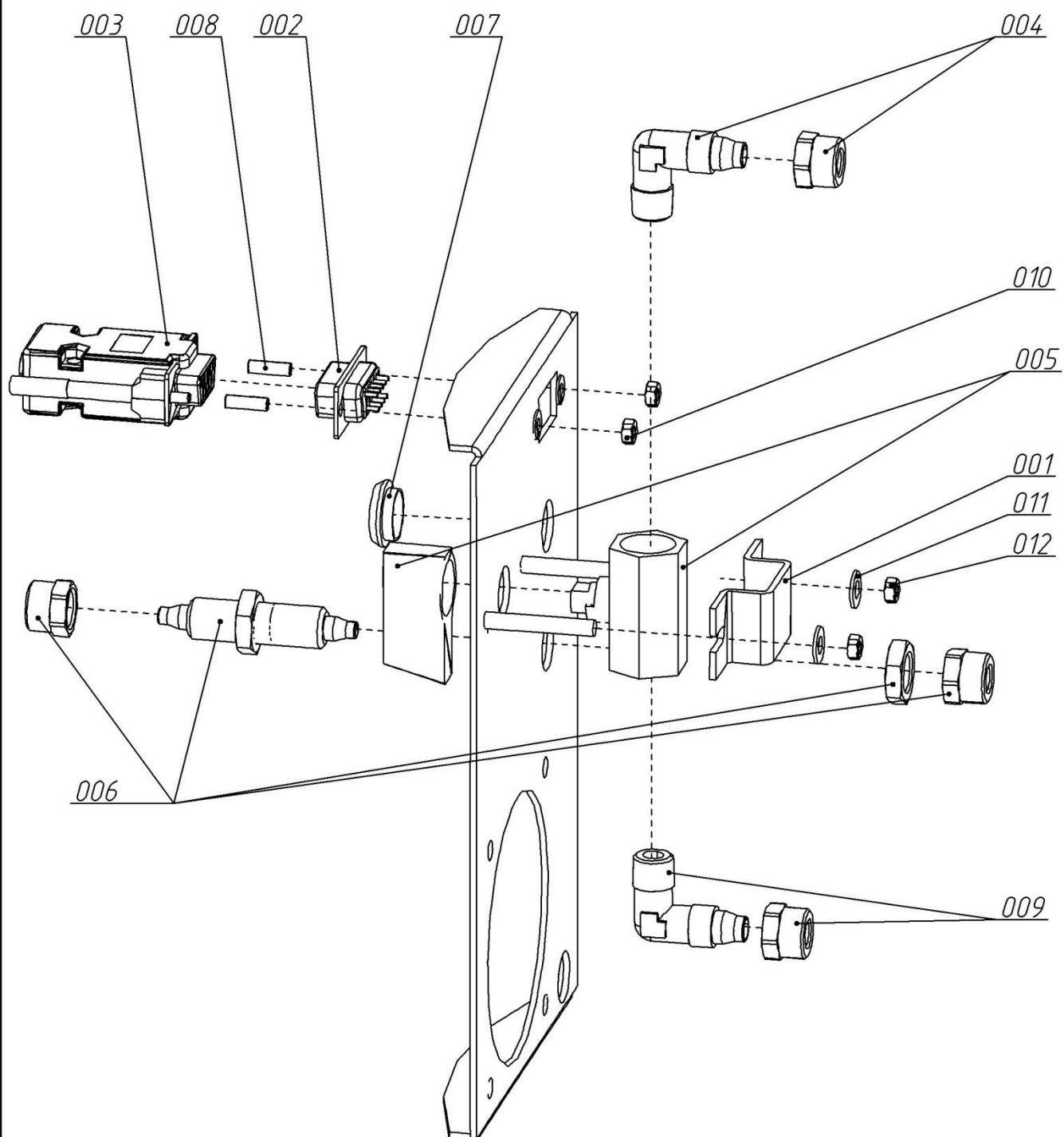

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT F"
List 2.26