


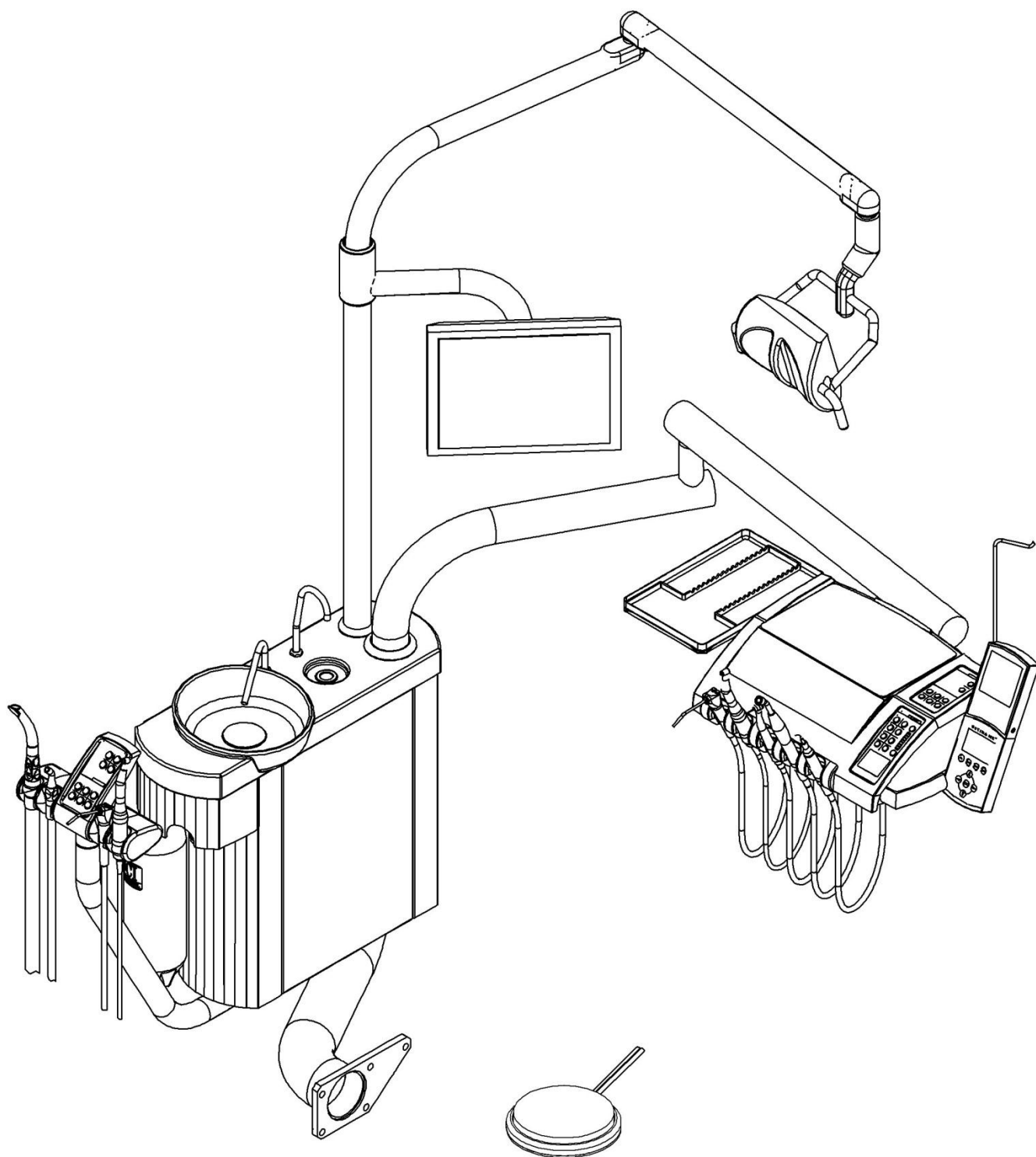


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.0

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.1	001-01	Light on pantograph arm EDI	306010300 (FARO)	
1.1	001-02	Light on pantograph arm ALYA	312000000 (FARO)	
1.1	001-03	Light on pantograph arm MAYA	320010300 (FARO)	
1.1	002	Monitor Holder	GAL 043.00.00.000	
1.1	003	Monitor	Neovo DR-17 (standard)	
1.1	004	Light post	AEGA 116.76.00.000	
1.1	005-01	Assistant table	AEGA 116.93.00.000	
1.1	005-02	Holder	AEGA 116.89.00.000	
1.1	006	Pantograph arm	AEGA 016.65.00.000	
1.1	007-01	Doctor table U	AEGA 127.00.00.000	
1.1	007-02	Doctor table L	AEGA 137.00.00.000	
1.1	008	Water unit		
1.1	009-01	Foot controller	FCF 251-01 (US Medlink)	
1.1	009-02	Foot controller P2L1	224900200 (FARO)	
1.1	009-03	Foot controller	AEGA 4.320.001	

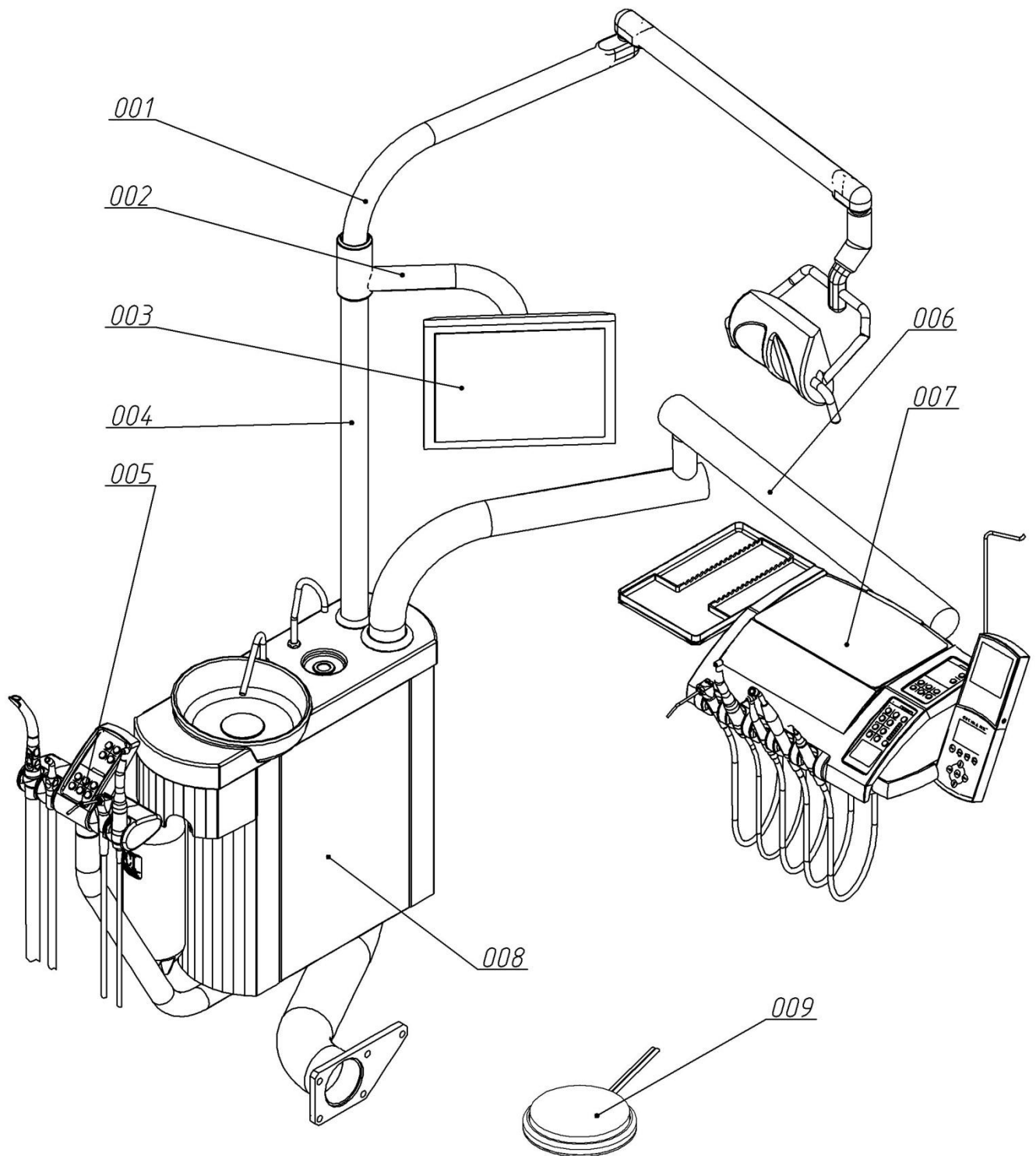

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.1

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.2	001	Panel	AEGA 116.00.00.079	
1.2	002	Switch	AE-C 1353ABNAB	
1.2	003	Enclosure	AEGA 116.00.00.078	
1.2	004	Enclosure	AEGA 116.00.00.081	
1.2	005	Cover	AEGA 116.00.00.083	
1.2	006	Enclosure	AEGA 116.00.00.080	
1.2	007-01	Frame	AEGA 116.80.00.000	
1.2	007-02	Frame	AEGA 116.79.00.000	
1.2	008	Washer	AEGA 116.00.00.084	
1.2	009	Screw DIN 912 M4x10	5154010-2	
1.2	010	Nut DIN 934 M4	6004-2	




Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:


Dental Unit "GALLANT H"

List 1.2

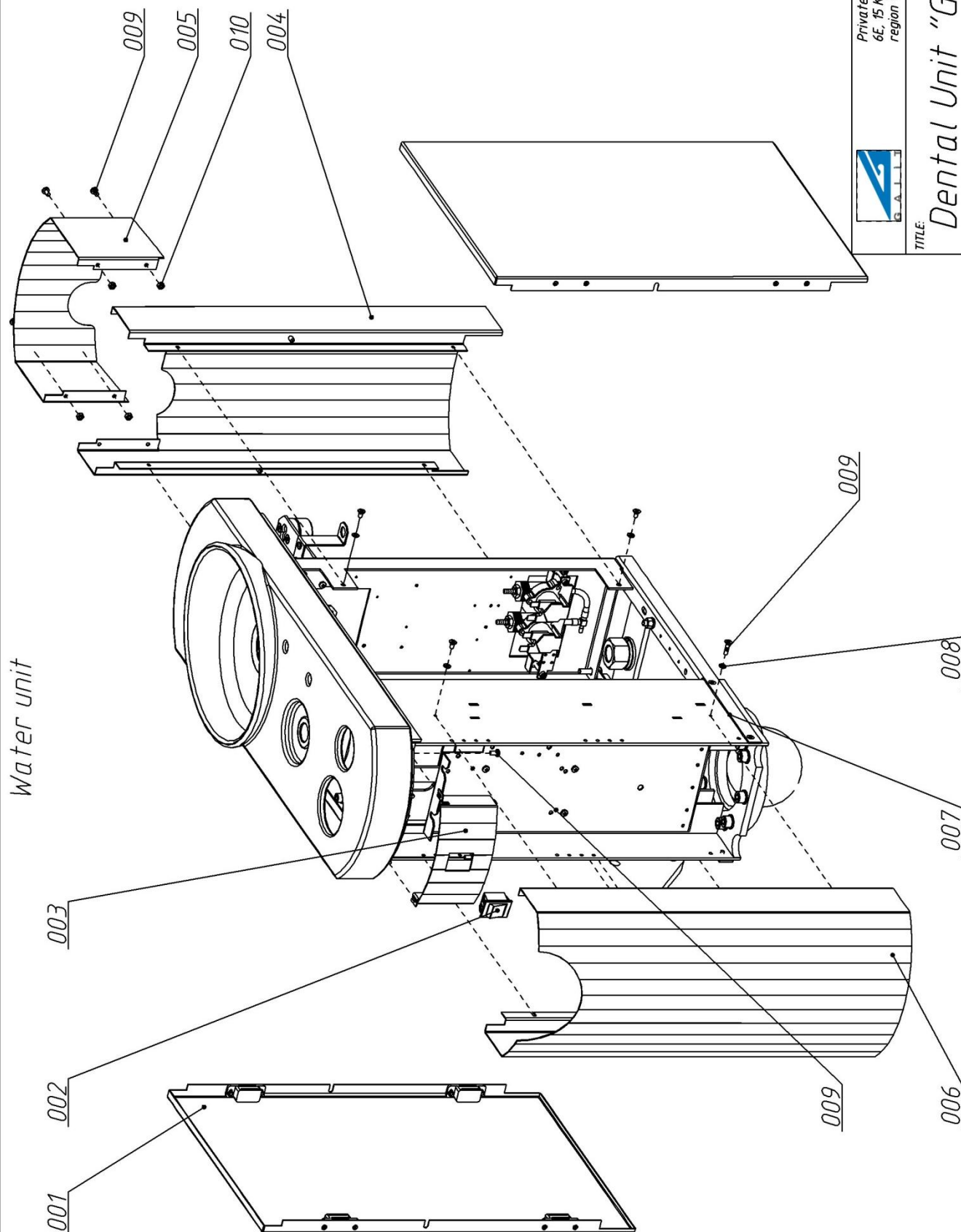

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.3	001	Panel	AEGA 116.00.00.079	
1.3	002	Magnet clip	246.26.741 HAFELE	
1.3	003	Screw DIN 965/963 M3x12	5M13012-2	
1.3	004	Nut DIN 934 M3	6003-2	
1.3	005	Washer DIN 125 3	7003-2	

Panel

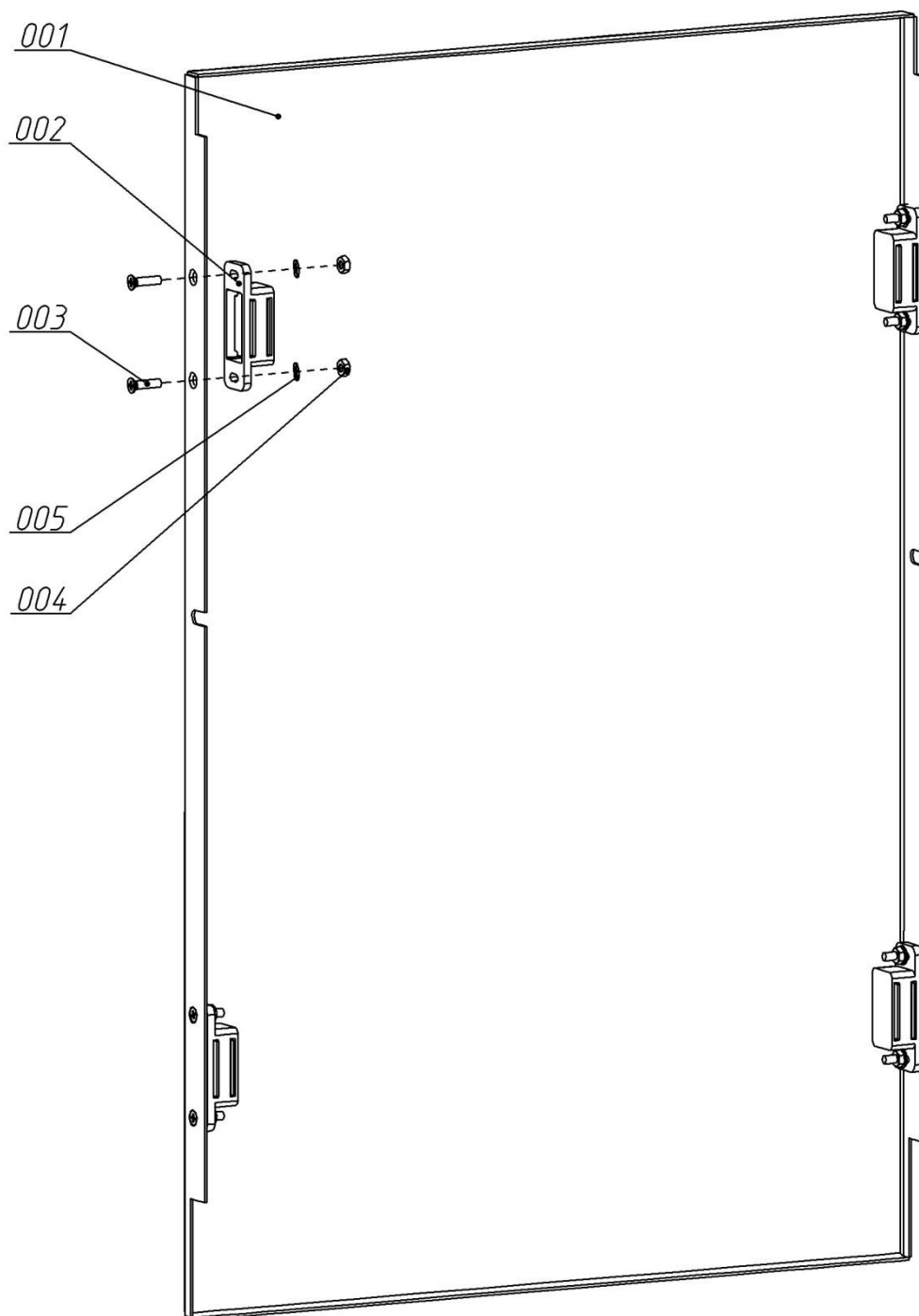


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.3

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.4	001	Enclosure	AEGA 116.00.00.080	
1.4	002	Finger	AEGA 116.00.00.066	
1.4	003	Nut DIN 934 M4	6004-2	

Enclosure

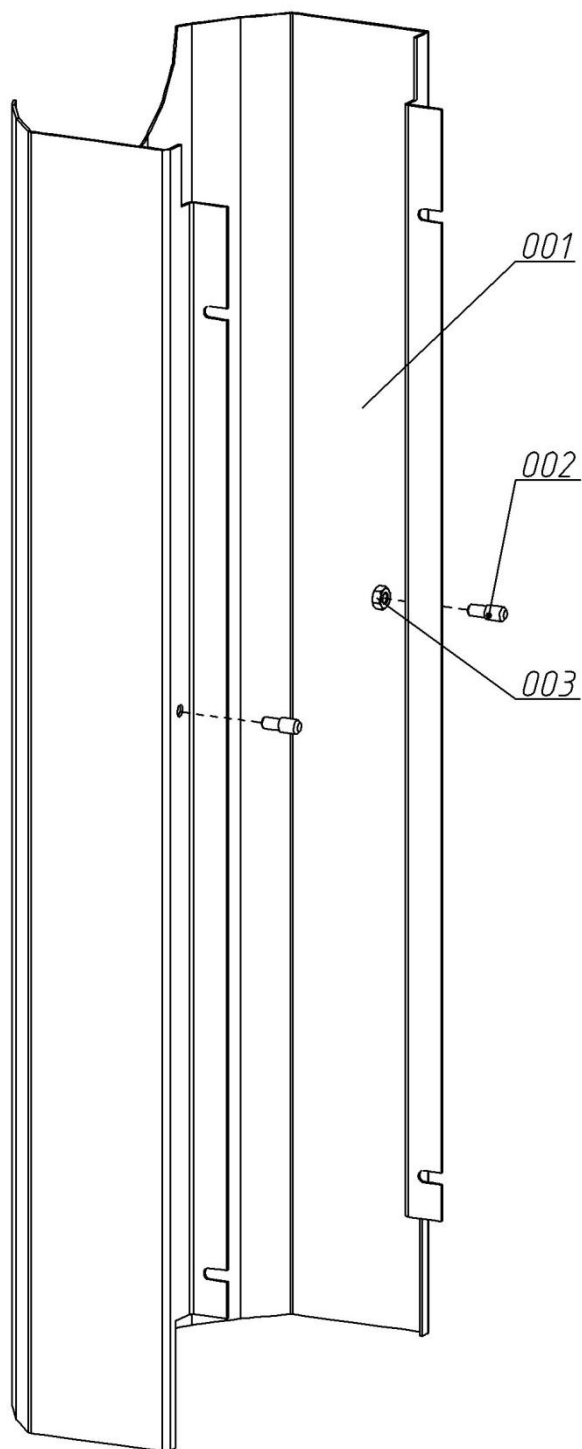


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.4

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.5	001	Bushing	AEGA 116.00.00.098	
1.5	002	Holder	AEGA 116.00.00.072	
1.5	003	Bush	BA8.226.314-24	
1.5	004	Adapter	AEGA 116.99.00.000	
1.5	005	Firm bottle 2 litter	ACF-543-02	
1.5	006	Washer DIN 125 5	7005-2	
1.5	007	Screw DIN 912 M5x30	5155030-2	
1.5	008	Screw DIN 912 M8x14	5158014-2	
1.5	009	Screw DIN 912 M8x25	5158025-2	
1.5	010	Washer DIN 125 8	7008-2	
1.5	011	Screw DIN 912 M6x20	5156020-2	
1.5	012	Nut DIN 934 M6	6006-2	
1.5	013	Washer DIN 125 6	7006-2	

Connection of cuspidor, assistant table and adapter to framework

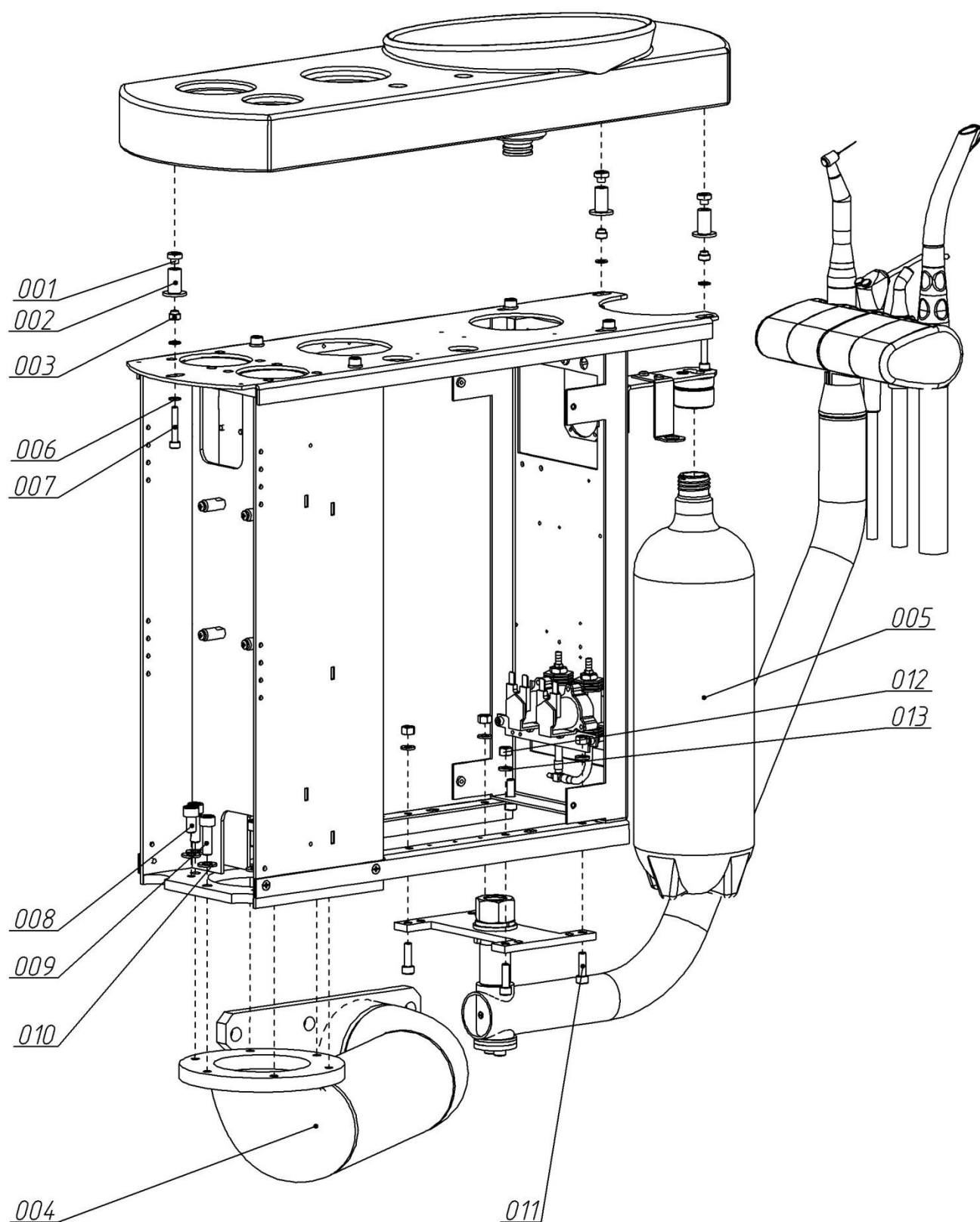


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.5

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.6	001	Cup	AEGA 116.75.00.000	
1.6	002	O-ring	AEGA 116.77.00.001	
1.6	003	Cup	AEGA 116.77.01.000	
1.6	004	Bearing	HK 40x16	
1.6	005	O-ring A47 GOST 13941-68	95PK2047	
1.6	006	Screw DIN 912 M8x25	5158025-2	
1.6	007	Washer DIN 125 8	7008-2	
1.6	008	Screw DIN 914 M6x6	986006-H	
1.6	009	Screw	AEGA 116.00.00.062-01	
1.6	010	Screw DIN 913 M6x10	986010-P	

Connection of Pantograf arm and Light post to framework

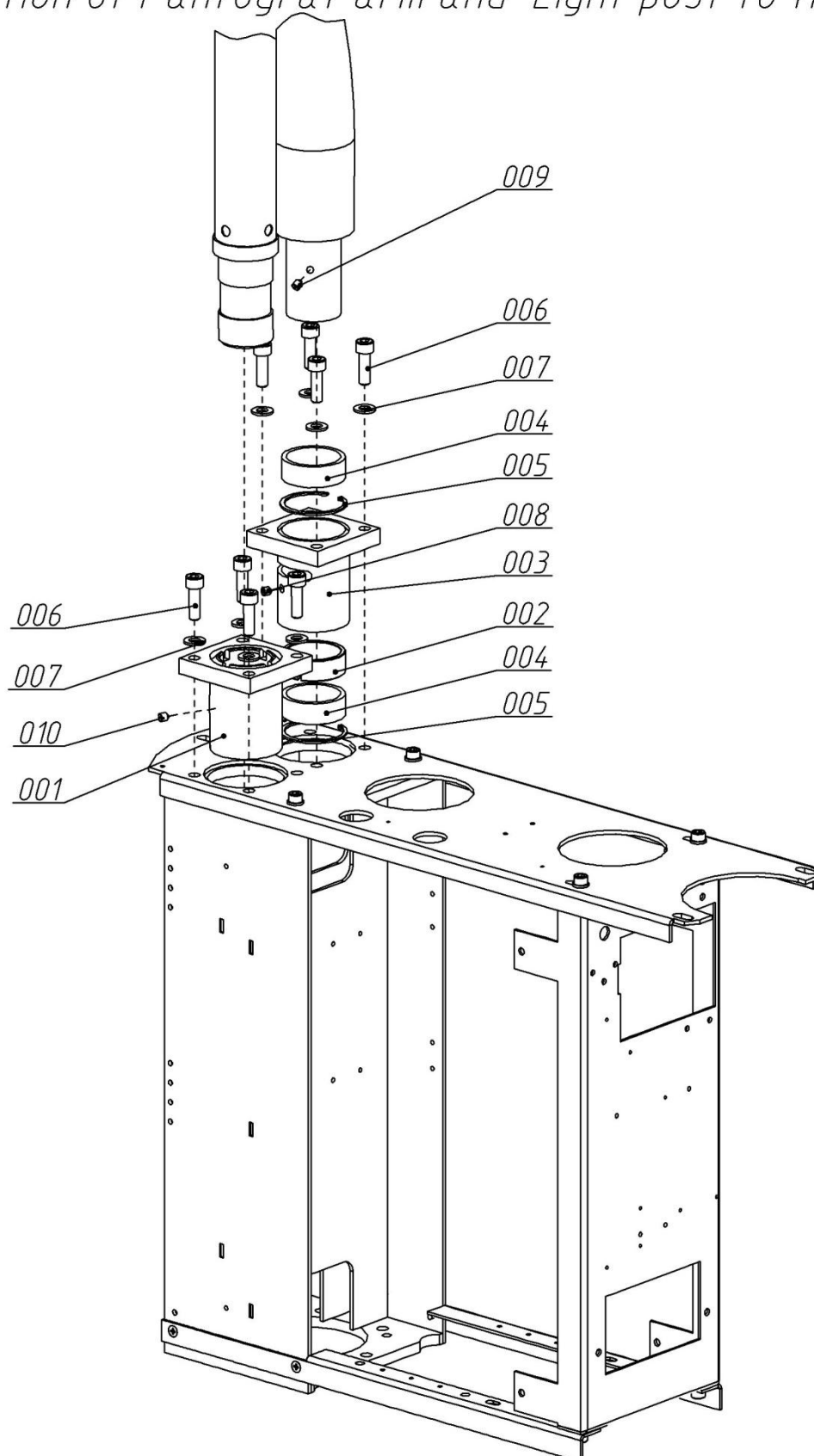


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.6

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.7	001	Bracket	AEGA 016.00.00.031	
1.7	002	Bracket	AEGA 116.00.00.070	
1.7	003	Filter-regulator	N108-D01-C01	
1.7	004	Filter-regulator	N108 D00	
1.7	005	Micro-regulator	M008 R00	
1.7	006	Electric-pneumatic switch	224331	
1.7	007	Fitting	2500 1/8	
1.7	008	Fitting	FIF-627-04	
1.7	009	Coupling	FIF-605-02	
1.7	010	Hose Ø4	1A-006-07	
1.7	011	Fitting	1500 8/6-1/8	
1.7	012	Fitting	1500 6/4-1/8	
1.7	013	Fitting	FIF-617-01 1/16	
1.7	014	Fitting	2070 1/8-1/8	
1.7	015	Fitting	2611 1/8	
1.7	016	Fitting	2531 1/8-M5	
1.7	017	Washer Ø5	FIF-607-01	
1.7	018	Fitting	FIF-618-01 1/8	
1.7	019	Screw DIN 7985 M4x8	5154010-2	
1.7	020	Coupling	FIF-605-01	
1.7	021	Hose Ø2	1A-013-07	

Filters

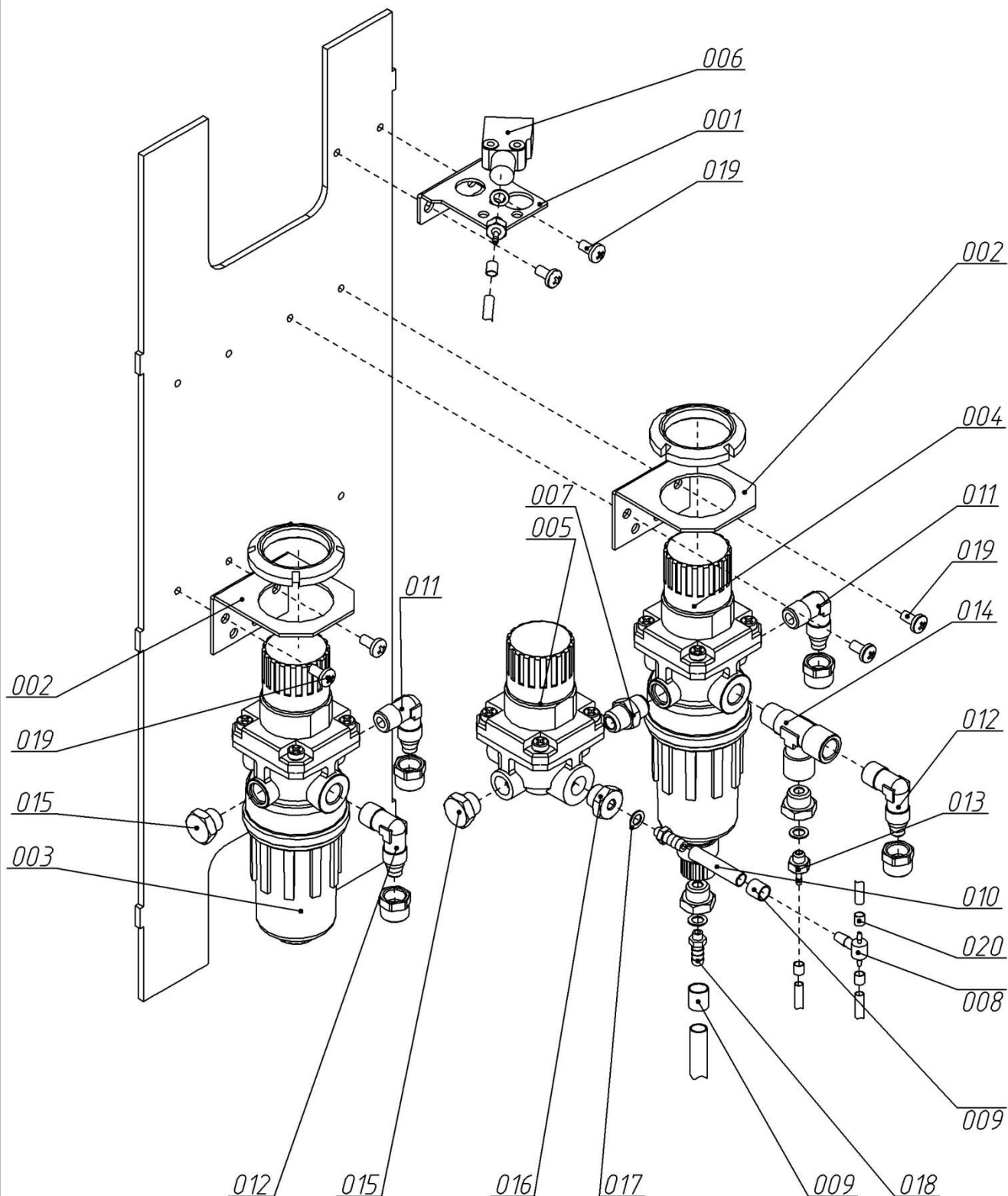


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.7

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
1.8	001	Holder	AEGA 116.86.00.000	
1.8	002	Strip	AEGA 116.00.00.076	
1.8	003	Fitting	FIF-618-01 1/8	
1.8	004	Coupling	FIF-605-02	
1.8	005	Fitting	FIF-627-02	
1.8	006	Hose Ø4	1A-006-06	
1.8	007	Fitting	2531 1/8-M5	
1.8	008	Screw DIN 912 M4 x 10	5154010-2	
1.8	009	Screw DIN 912 M4 x 25	5154025-2	
1.8	010	Nut DIN 934 M4	6004-2	
1.8	011	Solenoid valve RPE	2105NC24dc	
1.8	012	Washer Ø5	FIF-607-01	

Solenoid valves RPE

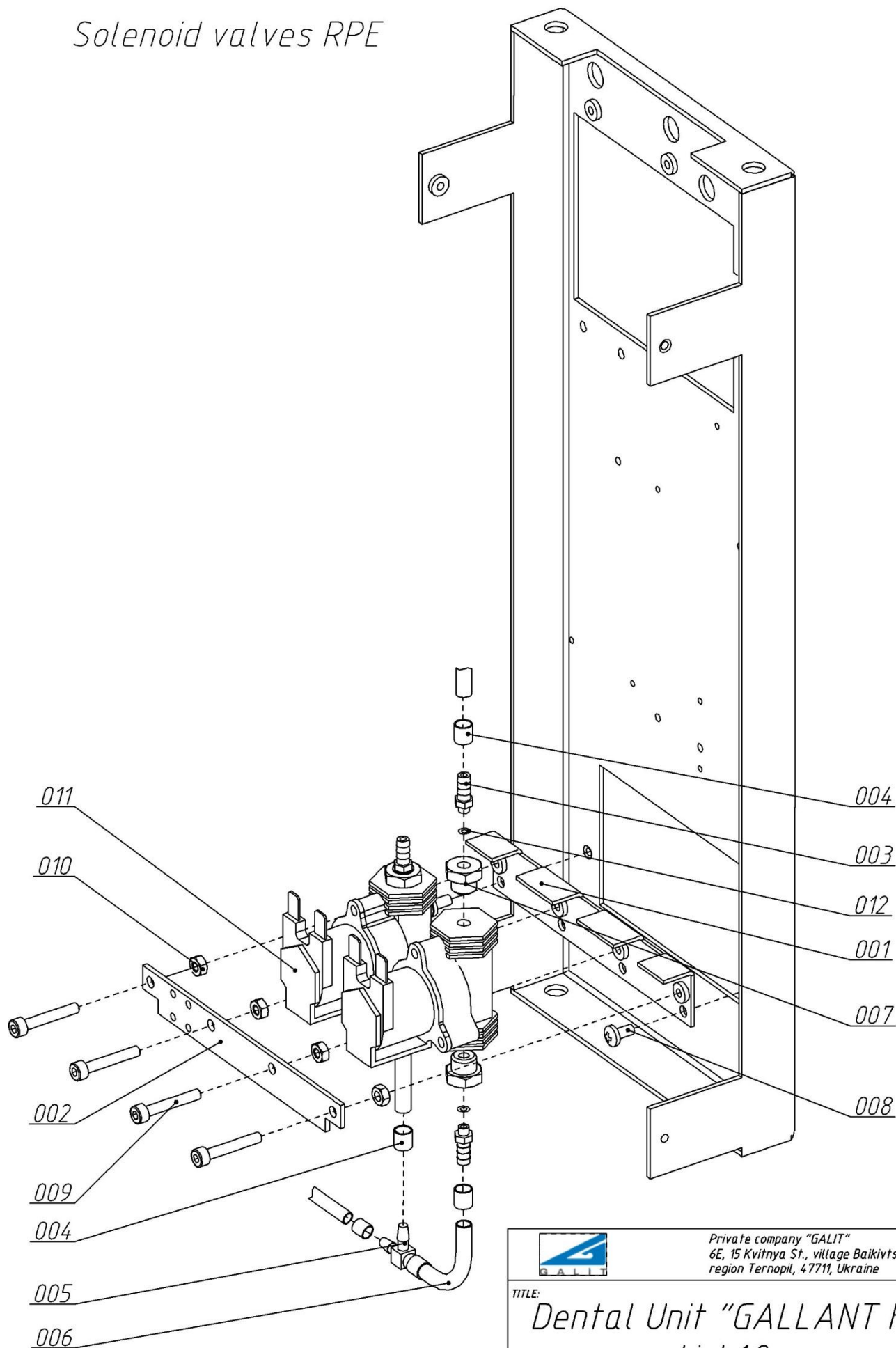


Private company "GALLIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.8

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.9	001	Bracket	AEGA 116.95.00.001	
1.9	002	Column	AEGA 116.00.00.067	
1.9	003	Interplanar Strut	RI-LCBS-TFM4-8	
1.9	004	Transformer	TST 300/041	
1.9	005	Controller	A 120 M	
1.9	006	Washer	AEGA 016.00.00.030	
1.9	007	Rubber washer	AEGA 116.95.00.002	
1.9	008	Fuse-holder	TLF-100-3	
1.9	009	Screw DIN 7985 M4x8	5M54008-2	
1.9	010	Washer DIN 125 6	7006-2	
1.9	011	Bolt DIN 933 M6x80	5M66080-2	
1.9	012	Screw DIN 7985 M3x16	5M53016-2	
1.9	013	Nut DIN 934 M6	6006-2	
1.9	014	Nut DIN 934 M4	6004-2	
1.9	015	Fuse		

Transformer electric parts

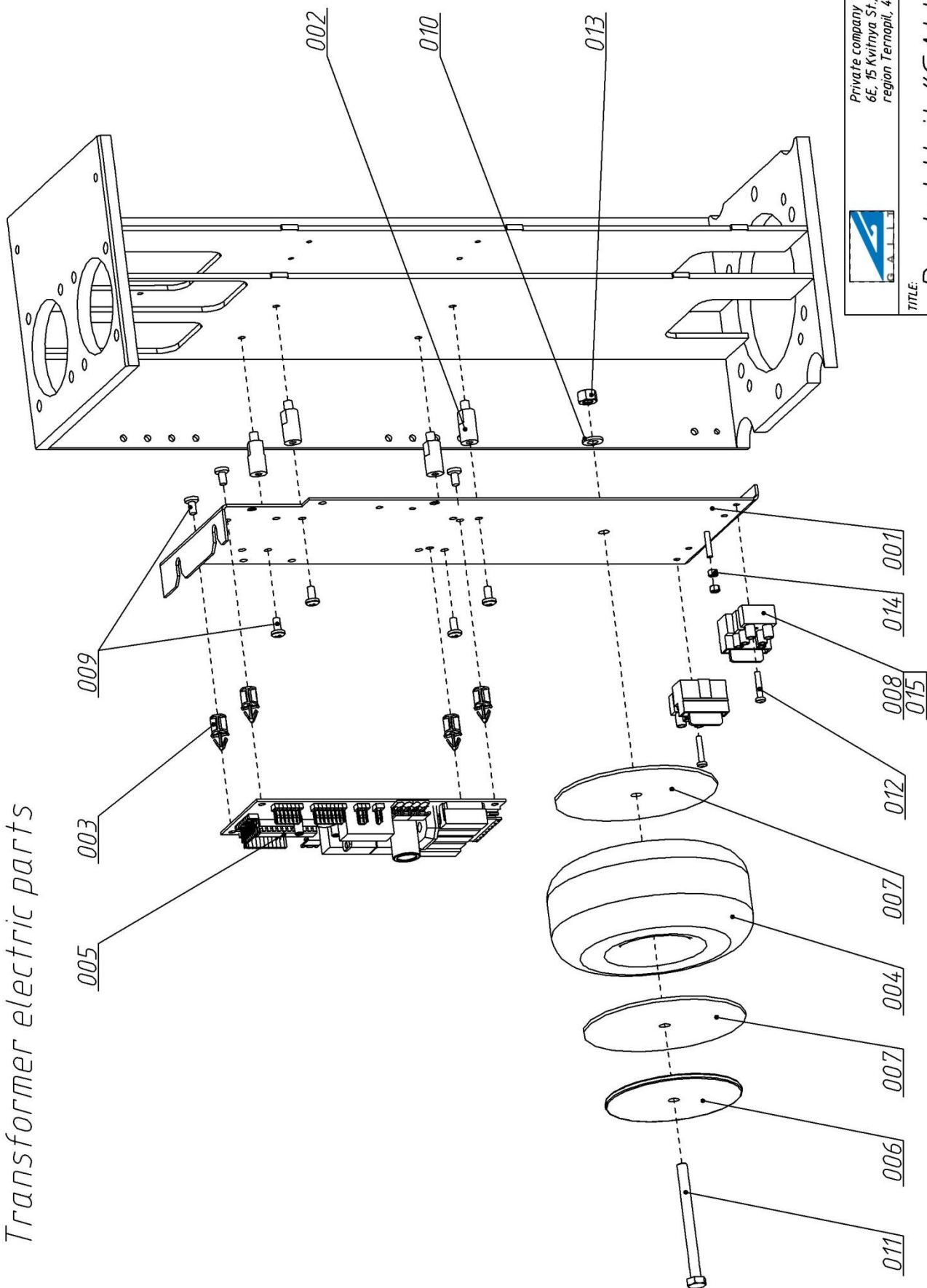

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine




TITLE:

Dental Unit "GALLANT H"
List 1.9

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.10	001	Pipe	AEGA 116.74.02.001	
1.10	002	Holder	AEGA 116.74.01.002	
1.10	003	Plug	AEGA 116.74.00.010	
1.10	004	Ring	AEGA 116.00.00.061-01	
1.10	005	Ring	AEGA 116.00.00.061	
1.10	006	O-ring	010-14-25 GOST 9833-74	
1.10	007	Pipe	AEGA 116.74.01.001	
1.10	008	Mouth	AEGA 916.00.00.010	
1.10	009	Washer	AEGA 116.74.01.005	
1.10	010	Nut	AEGA 116.74.00.009	
1.10	011	Nut	AEGA 116.74.01.004	
1.10	012	Securing ring	0700-700-01	
1.10	013	Elbow 30°	0700-700-07	
1.10	014	Socket	0700-720-18	
1.10	015	Pressing	AEGA 116.74.01.003	
1.10	016	Cuspidor	ITEM NEW	
1.10	017-01		RI07051	
1.10	017-02	Cuspidor screen with gray lid	VCF-353-02	

Cuspidor

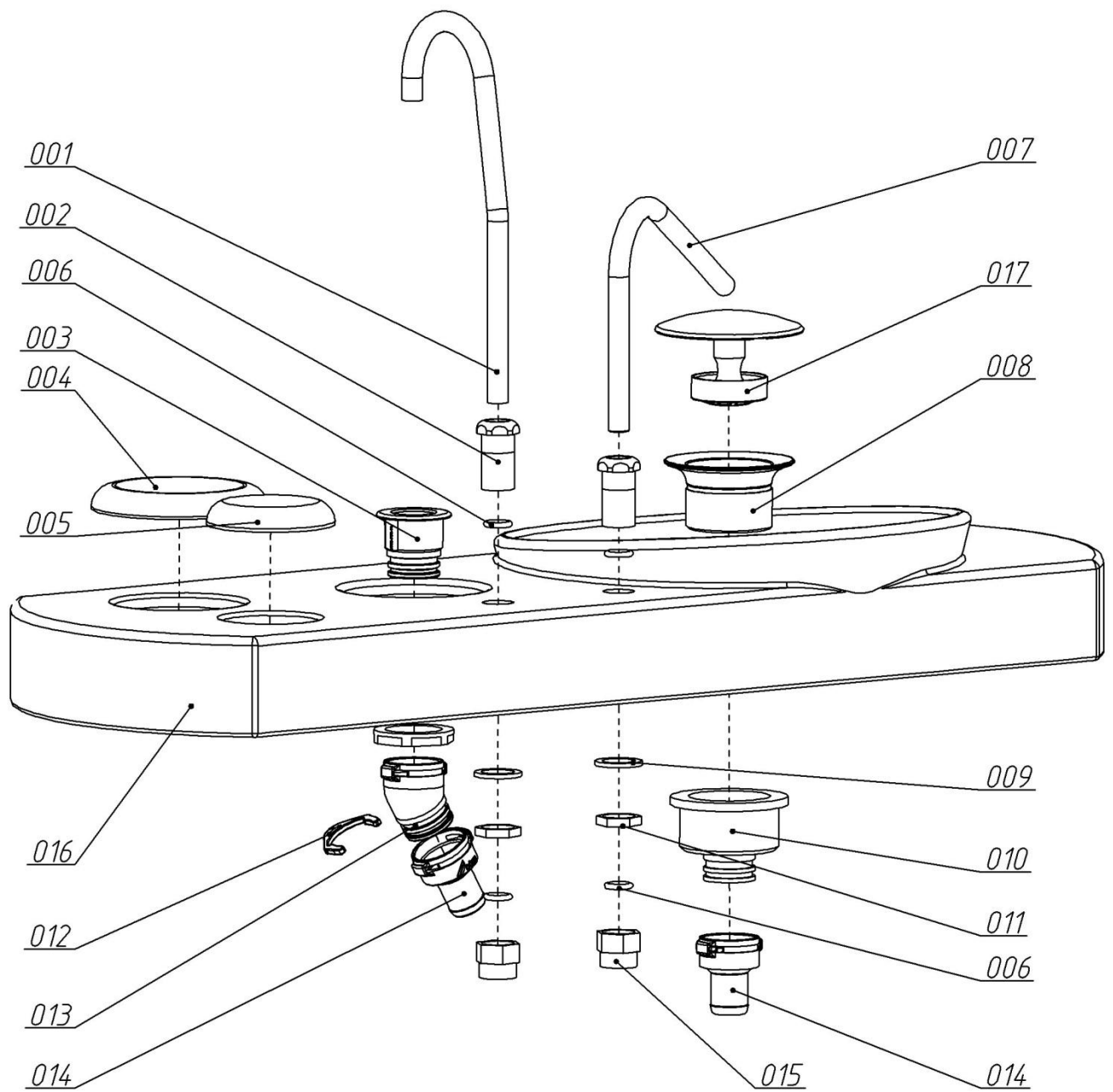


Private company "GALIT"
6E, 15 Kvintya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.10

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.11	001-01	Instruments holders and instruments	AEGA 116.89.00.000	
1.11	001-02	Assistant table	AEGA 116.93.00.000	
1.11	002	Washer	AEGA 116.78.00.004	
1.11	003	Assistant Table arm	AEGA 116.78.01.000	
1.11	004	Bracket	AEGA 116.78.00.001	
1.11	005	Axis	AEGA 116.78.00.002	
1.11	006	Washer	AEGA 116.78.00.003	
1.11	007	Nut DIN 934 M20	6020-2	
1.11	008	Spring washer DIN 7980 20	7P20-2	
1.11	009	Washer DIN 125 20	7O20-2	
1.11	010	Screw DIN 915 M6 x 8	986008-C	
1.11	011	Screw DIN 912 M6 x 12	5156012-2	

Assistant table

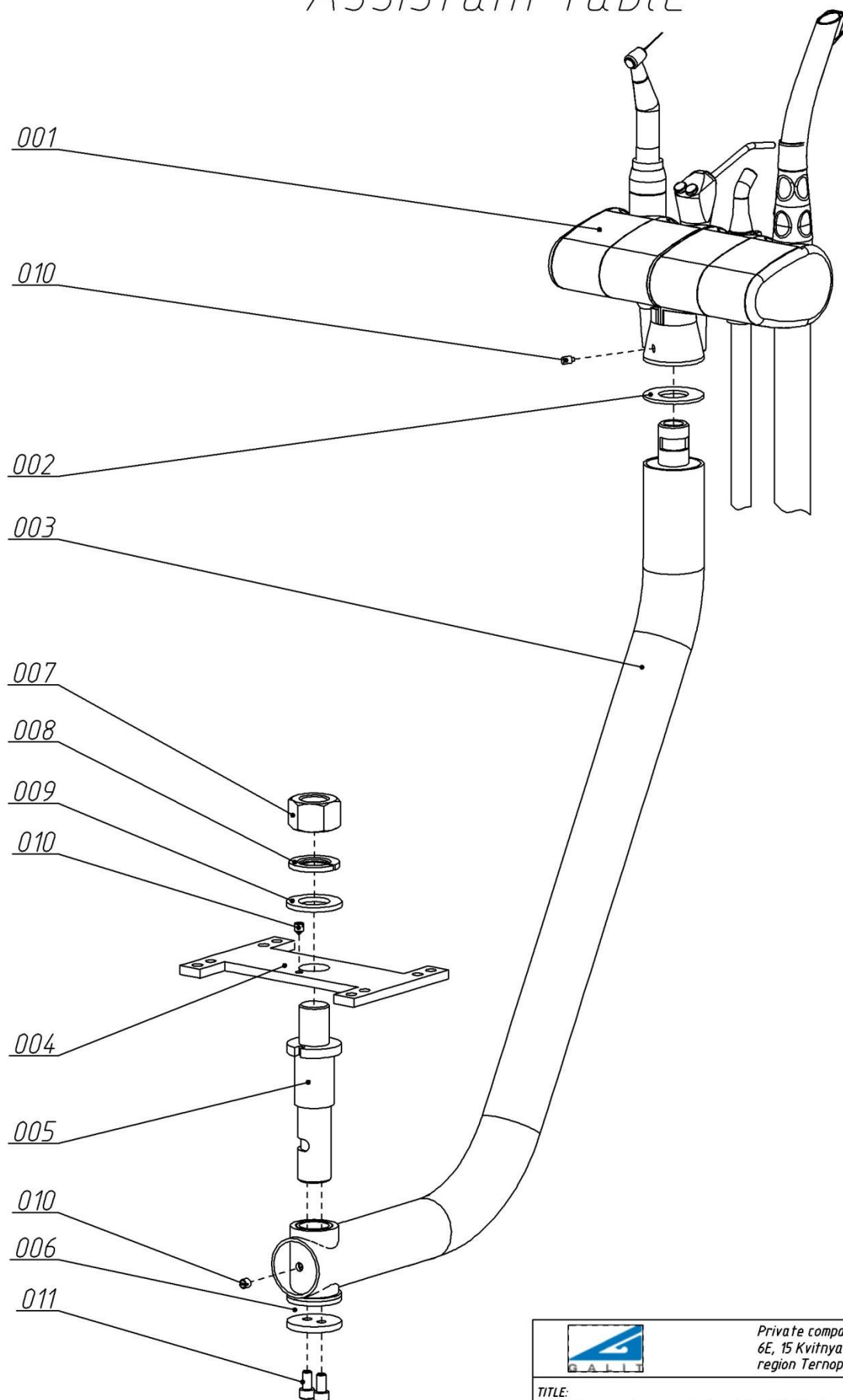


Private company "GALLIT"
6E, 15 Kvintya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.11

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.14	001	Bracket	AEGA 116.00.00.082	
1.14	002	Place selector valve	7560-500-60	
1.14	003	Secretion filter	0725-040-50	
1.14	004	Blank	0700-700-10	
1.14	005	Socket	0700-720-25	
1.14	006	Securing ring	0700-700-01	
1.14	007	Elbow 90°	0700-700-04	
1.14	008	Filter cover	0725-041-05	
1.14	009	Self tapping screw DIN 96 3x12	1023012-2	
1.14	010	Screw DIN 967 M4x8	5154010-2	

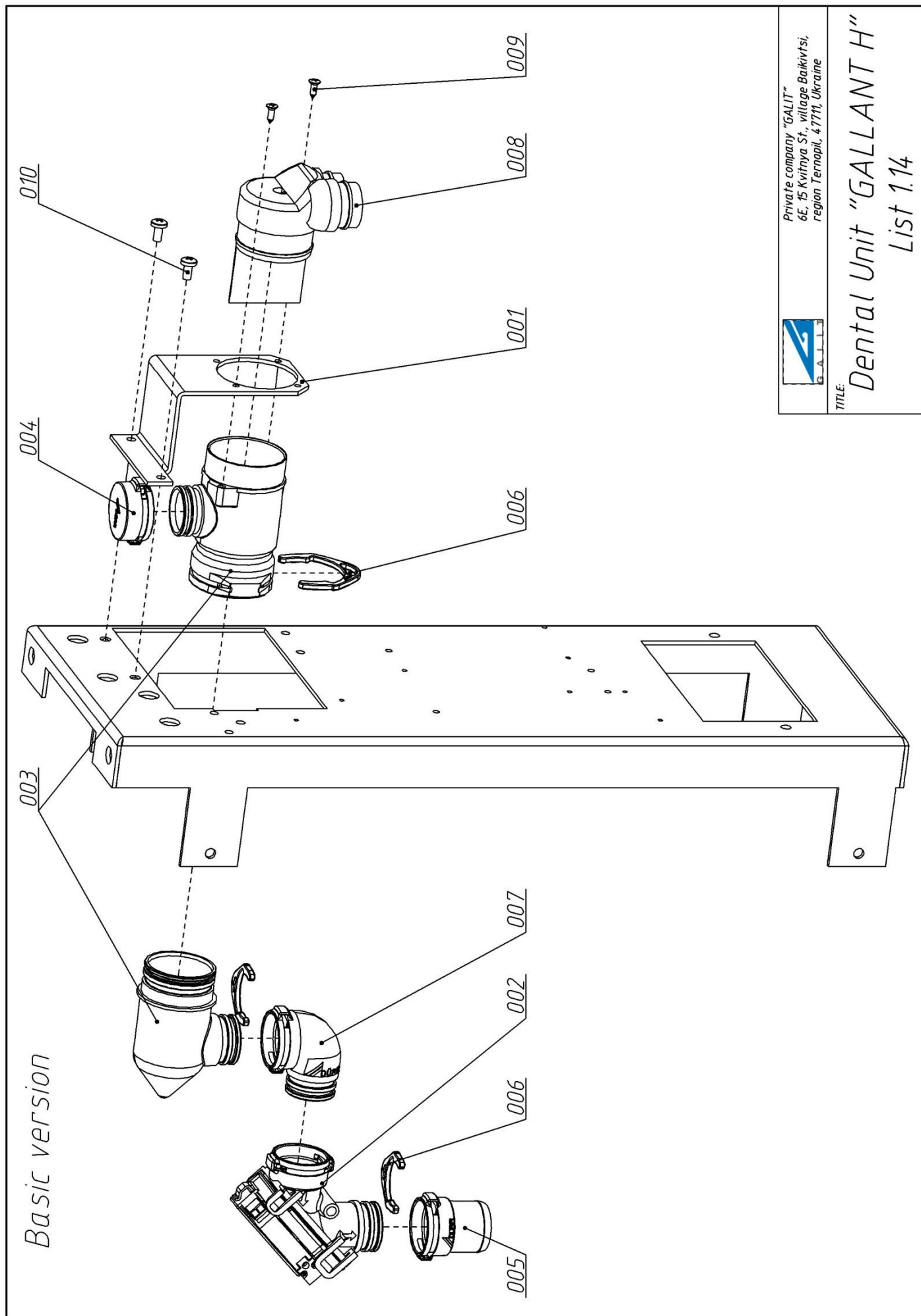

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine




TITLE:

Dental Unit "GALLANT H"

List 1.14

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
1.15	001	Plug	AEGA 116.74.00.010	
1.15	002	Nut	AEGA 116.74.00.009	
1.15	003	Elbow 30°	0700-700-07	
1.15	004	Socket	0700-720-18	
1.15	005	Grey Pipe	Ø17.5 Cattani	
1.15	006	Securing ring	0700-700-01	
1.15	007	Securing ring	0700-700-09	
1.15	008	Elbow 90°	0700-700-04	
1.15	009	Securing ring	0700-700-54	
1.15	010	Blank	0700-700-10	
1.15	011	T-joint	0700-700-08	
1.15	012	Y-joint	0700-700-03	
1.15	013	Double Socket	0700-700-14	
1.15	014	Coarse filter	0700-702-52	
1.15	015	Screw DIN 912 M3x30	5153030-2	

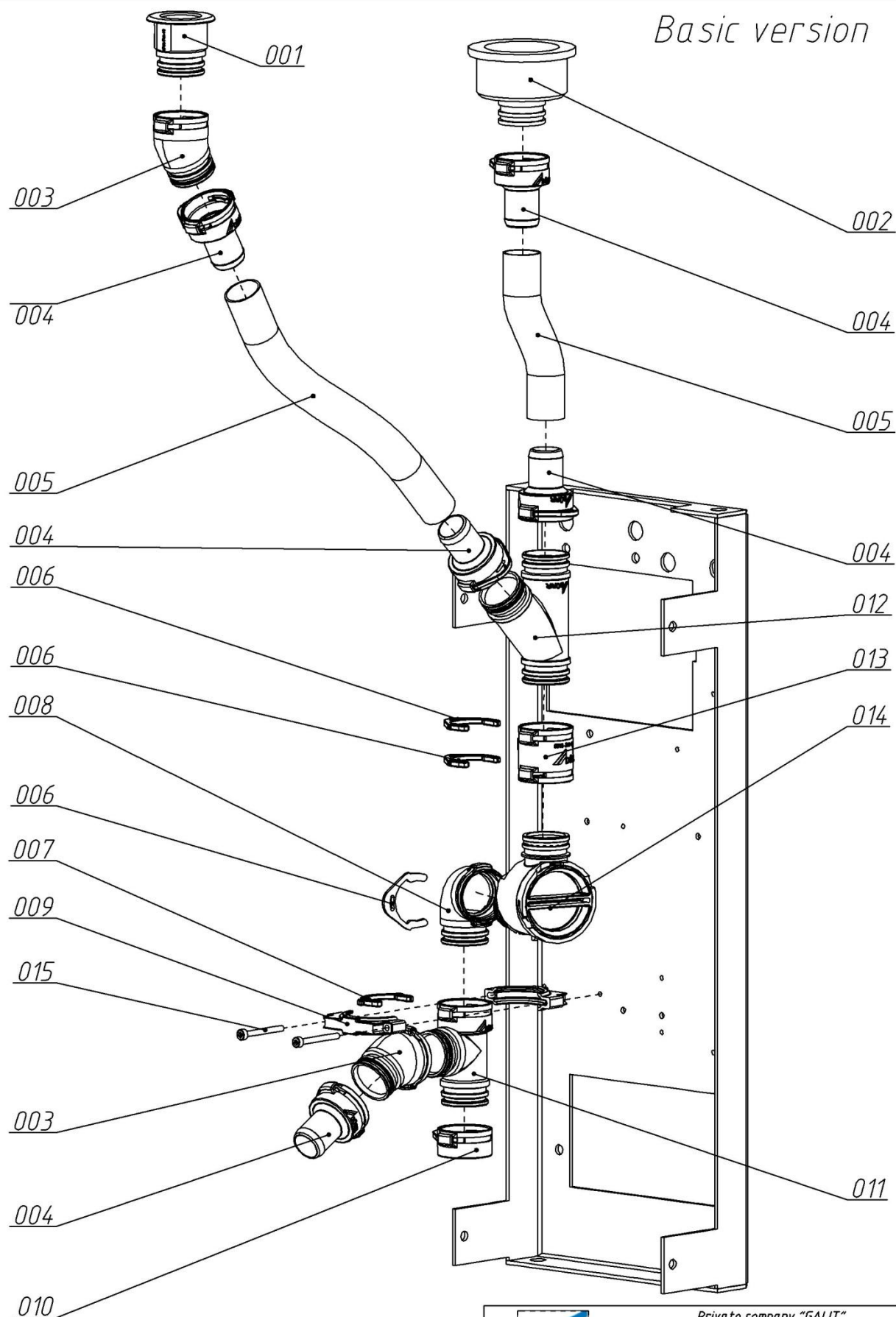


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.15

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.16	001	Holder	AEGA 116.00.00.074	
1.16	002	Holder	AEGA 116.00.00.073	
1.16	003	Fitting	FIF-617-01 1/16	
1.16	004	Pneumatic switch	VAF-451-03	
1.16	005	Fitting	2601 2 M5	
1.16	006	Plug	AEGA 9.313.001	
1.16	007	Fitting	2601 4.5 M5	
1.16	008	Hose Ø4	1A-006-07	
1.16	009	Screw DIN 912 M4x10	5154010-2	
1.16	010	Screw DIN 912 M5x10	5155010-2	
1.16	011	Coupling	FIF-605-01	
1.16	012	Coupling	FIF-605-02	
1.16	013	Screw DIN 7985 M4X8	5M54008-2	
1.16	014	O-ring	AEGA 116.00.00.097	
1.16	015	Washer Ø5	FIF-607-01	

Attachment of water supply elements

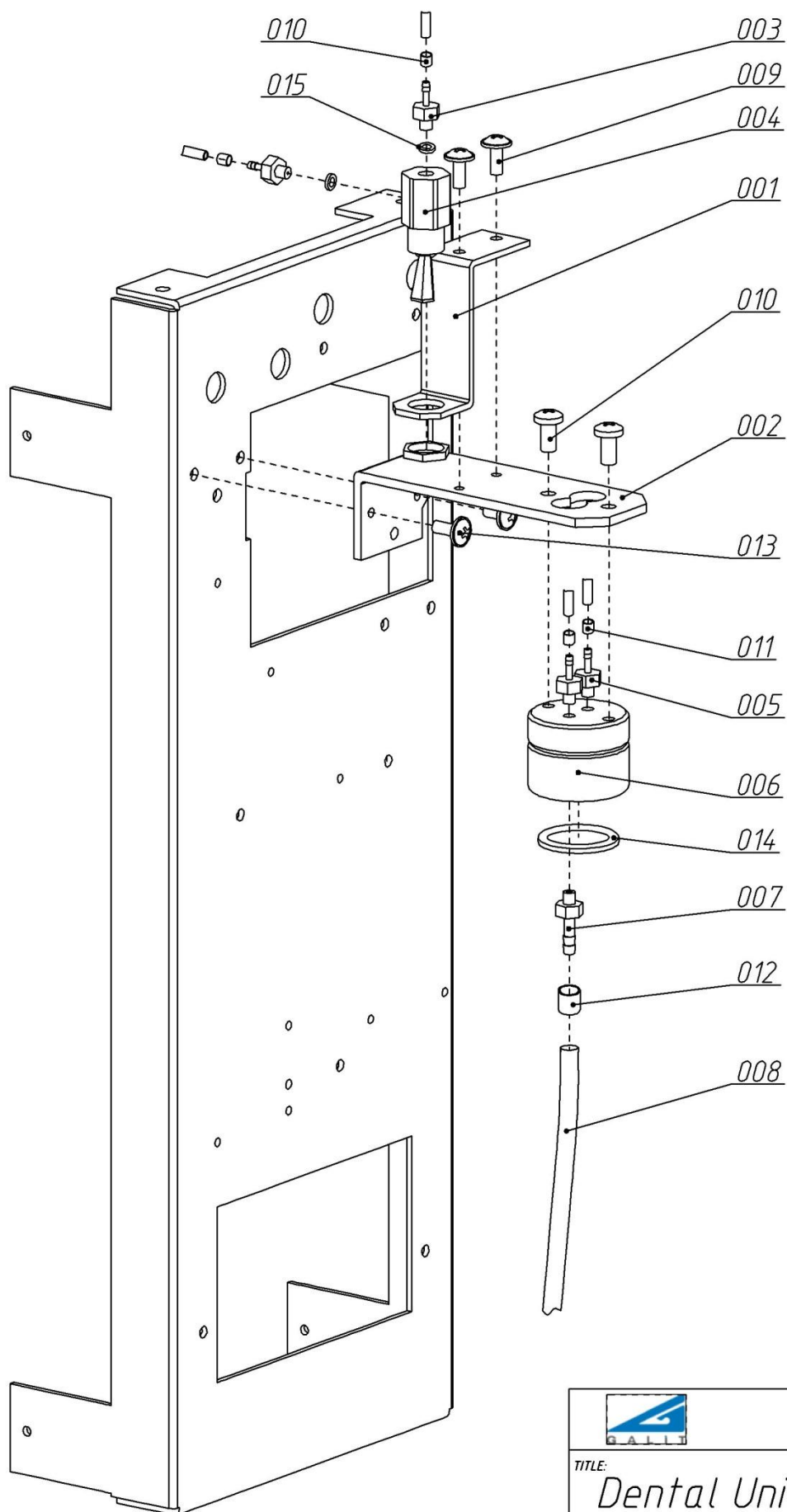


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.16

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
1.17	001	Bracket	AEGA 116.00.00.069	
1.17	002	Case	AEGA 116.96.00.000	
1.17	003	Socket	3923B	
1.17	004	Separator Cattani	020973	
1.17	005	Place selector valve	7560-500-60	
1.17	006	Socket	0700-720-25	
1.17	007	Securing ring	0700-700-01	
1.17	008	Plug	0700-700-21	
1.17	009	Washer DIN 125 4	7004-2	
1.17	010	Screw DIN 7985 M4x8	5M54008-2	

Version with Cattani Separator

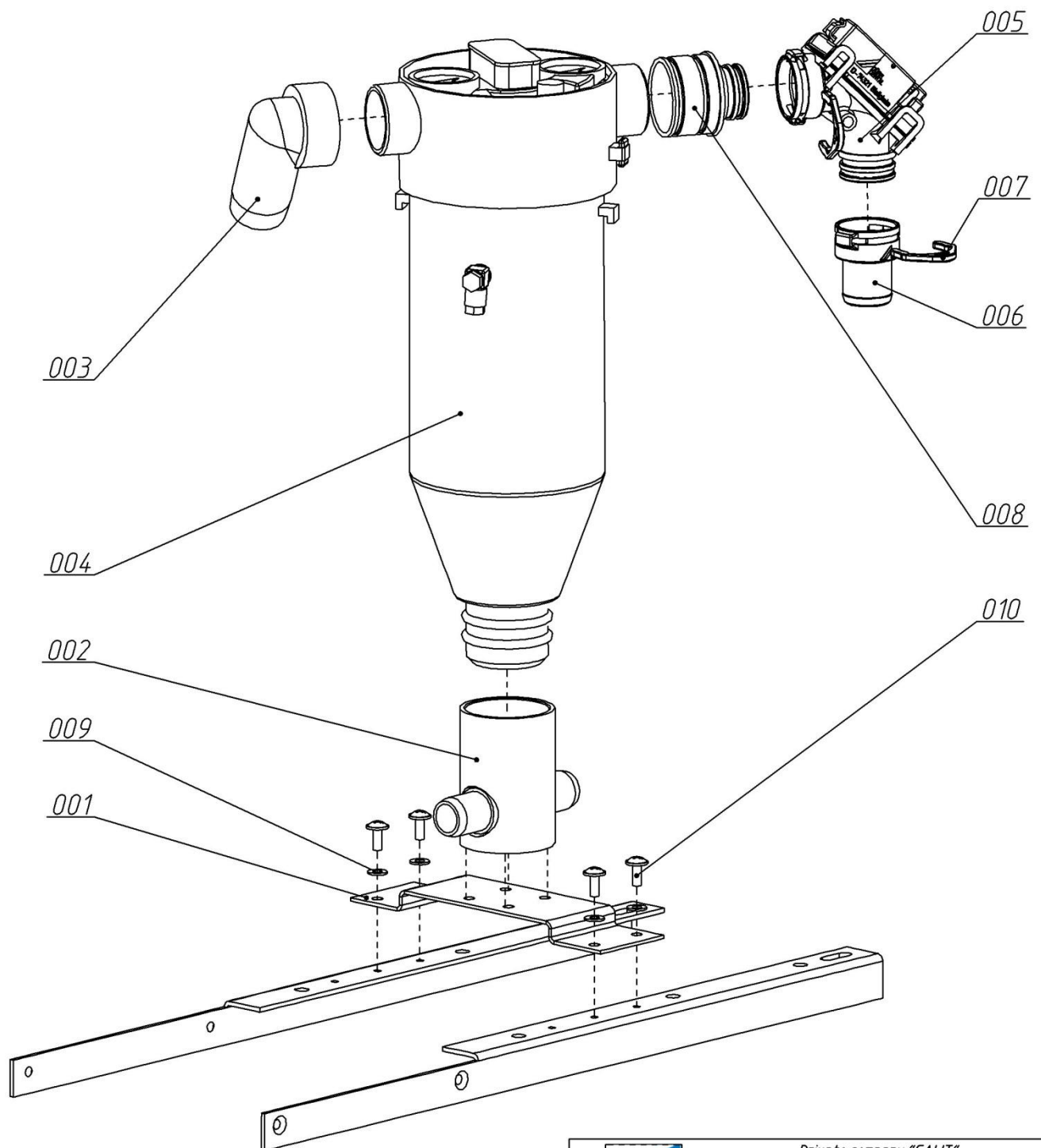


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.17

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
1.18	001	Nut	AEGA 116.74.00.009	
1.18	002	Securing ring	0700-700-01	
1.18	003	Socket	0700-720-18	
1.18	004	Grey Pipe	Ø17.5 Cattani	
1.18	005	Y-joint	0700-700-03	
1.18	006	Double Socket	0700-700-14	
1.18	007	Coarse filter	0700-702-52	
1.18	008	Elbow 90°	0700-700-04	
1.18	009	Securing ring	0700-700-09	
1.18	010	Clamp	0700-700-54	
1.18	011	T-joint	0700-700-08	
1.18	012	Blank	0700-700-10	

Version with Cattani Separator

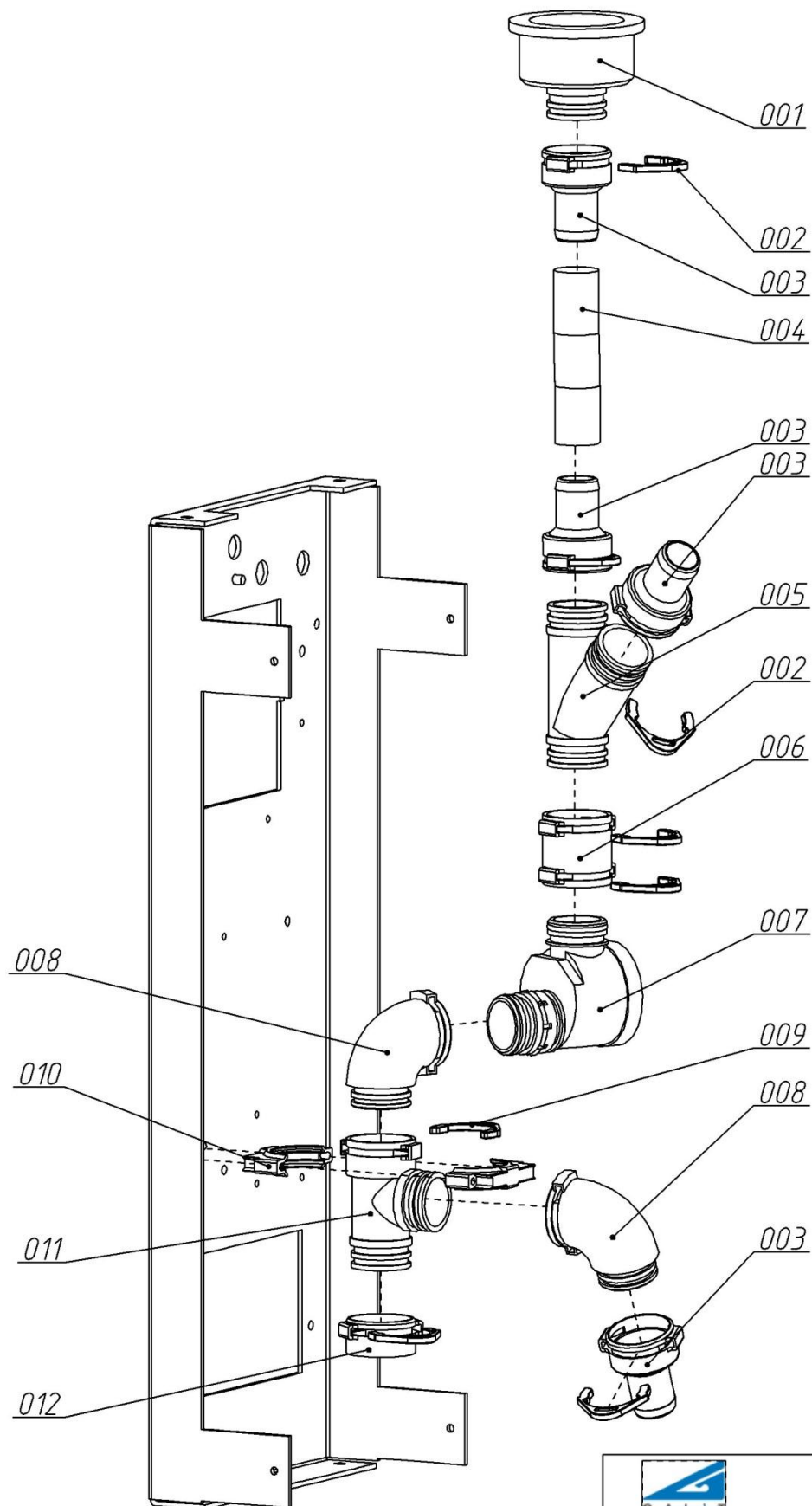


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.18

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.19	001	Bracket	AEGA 116.00.00.082	
1.19	002	Filter cover	0725-041-05	
1.19	003	Secretion filter	0725-040-50	
1.19	004	Blank	0700-700-10	
1.19	005	Securing ring	0700-700-01	
1.19	006	Socket	0700-720-25	
1.19	007	Screw DIN 912 M4x10	5154010-2	

Version with Cattani Separator and CAS-1

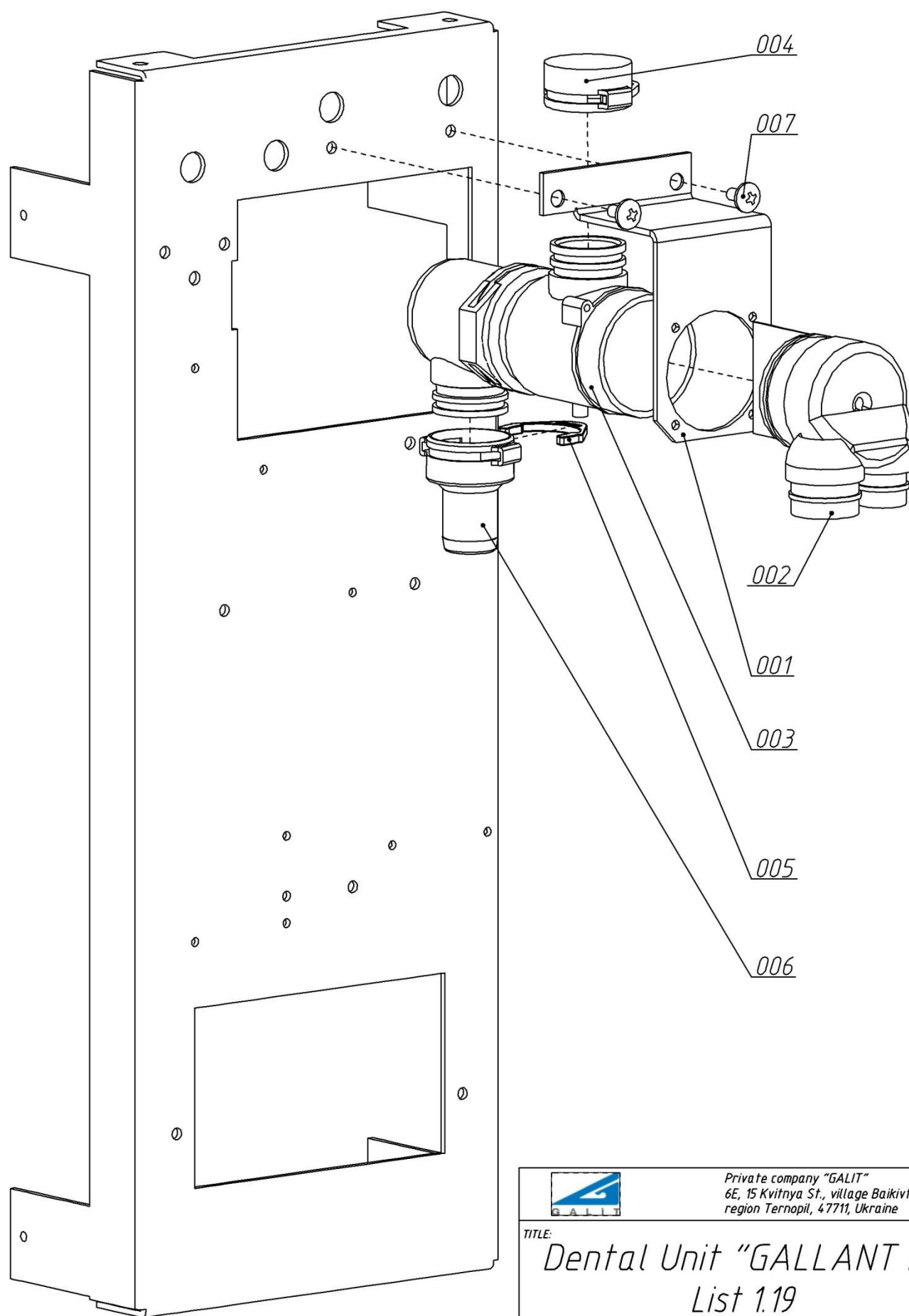


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:
Dental Unit "GALLANT H"
List 1.19

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.20	001	Plug	AEGA 116.74.00.010	
1.20	002	Nut	AEGA 116.74.00.009	
1.20	003	Bracket	AEGA 116.00.00.090	
1.20	004	Elbow 30°	0700-700-07	
1.20	005	Socket	0700-720-18	
1.20	006	Grey Pipe	Ø17.5 Cattani	
1.20	007	Elbow 45°	0700-700-05	
1.20	008	Elbow 90°	0700-700-04	
1.20	009-01	Separator CAS-1	7117-02	
1.20	009-02	Separator CAS-1 with selector switch	7117-23	
1.20	010	T-joint	0700-700-06	
1.20	011	Double Socket	0700-700-14	
1.20	012	Extension piece	0700-700-18	
1.20	013	Securing ring	0700-700-01	
1.20	014	Socket	0700-720-25	
1.20	015	Double connector	0700-700-13	
1.20	016	Coarse filter	0700-702-52	
1.20	017	Extension piece	0700-700-28	
1.20	018	Nut DIN 934 M4	6004-2	
1.20	019	Screw DIN 912 M4x20	5154020-2	

Version with Separator CAS-1

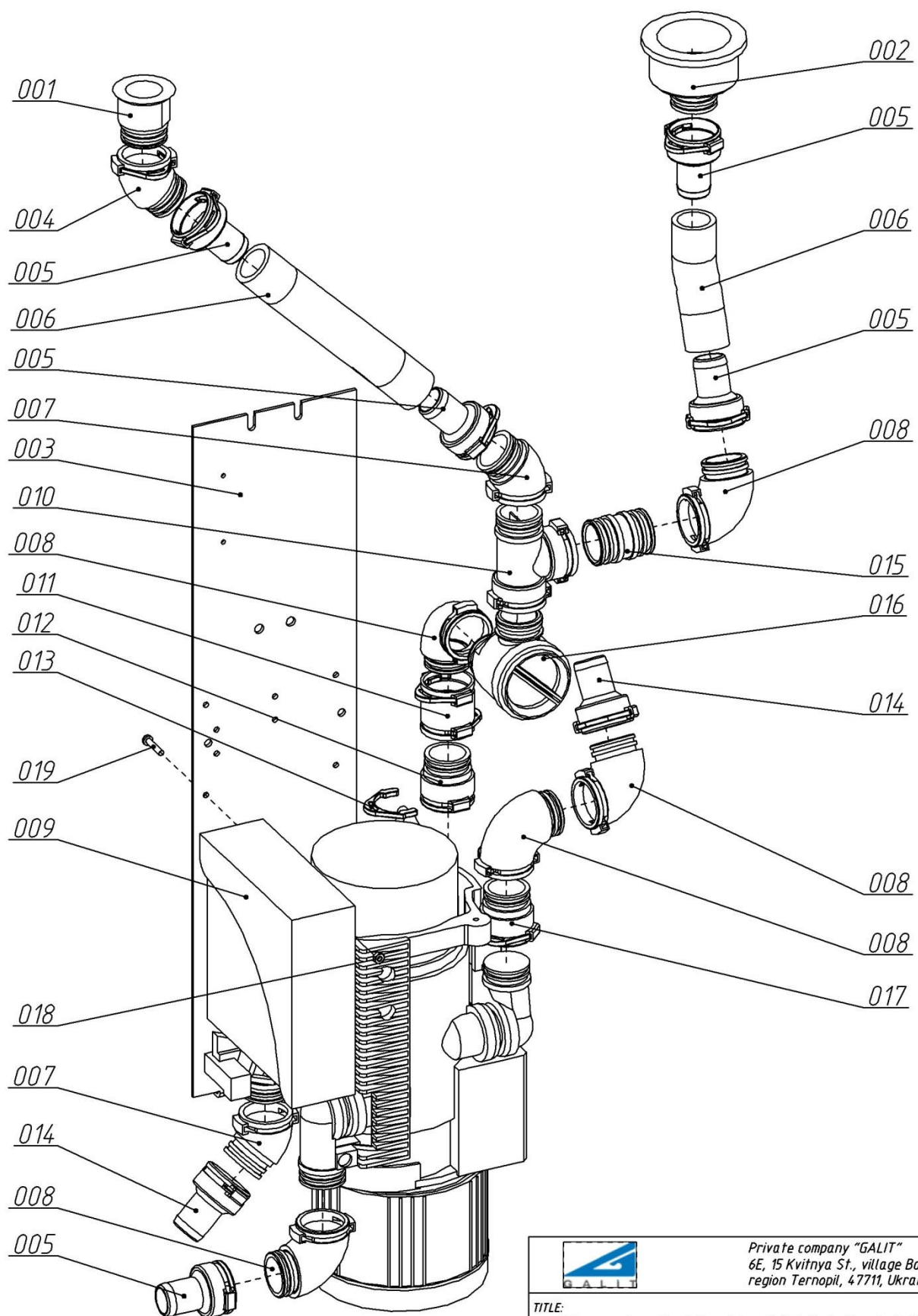


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.20

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.21	001	Plug	AEGA 116.74.00.010	
1.21	002	Nut	AEGA 116.74.00.009	
1.21	003	Bracket	AEGA 116.00.00.090	
1.21	004	Bracket	AEGA 116.00.00.082	
1.21	005	Secretion filter	0725-040-00	
1.21	006-01	Separator CS-1	7117-50	
1.21	006-02	Separator CS-1 with selector switch	7117-57	
1.21	007	Elbow 30°	0700-700-07	
1.21	008	Socket	0700-720-18	
1.21	009	Grey Pipe	Ø17.5 Cattani	
1.21	010	Socket	0700-720-25	
1.21	011	Sleeve Lignum Ø25		
1.21	012	Blank	0700-700-10	
1.21	013	Filter cover	0725-041-05	
1.21	014	Y-joint	0700-700-03	
1.21	015	Extension piece	0700-700-17	
1.21	016	Elbow 45°	0700-700-05	
1.21	017	T-joint	0700-700-06	
1.21	018	Securing ring	0700-700-01	
1.21	019	Double Socket	0700-700-14	
1.21	020	Elbow 90°	0700-700-04	
1.21	021	Coarse filter	0700-702-52	
1.21	022	T-joint	0700-700-08	
1.21	023	Screw DIN 912 M4x10	5154010-2	
1.21	024	Screw DIN 96 3x12	1023012-2	
1.21	025	Screw DIN 912 M4x20	5154020-2	
1.21	026	Nut DIN 934 M4	6004-2	

Version with Separator CS-1

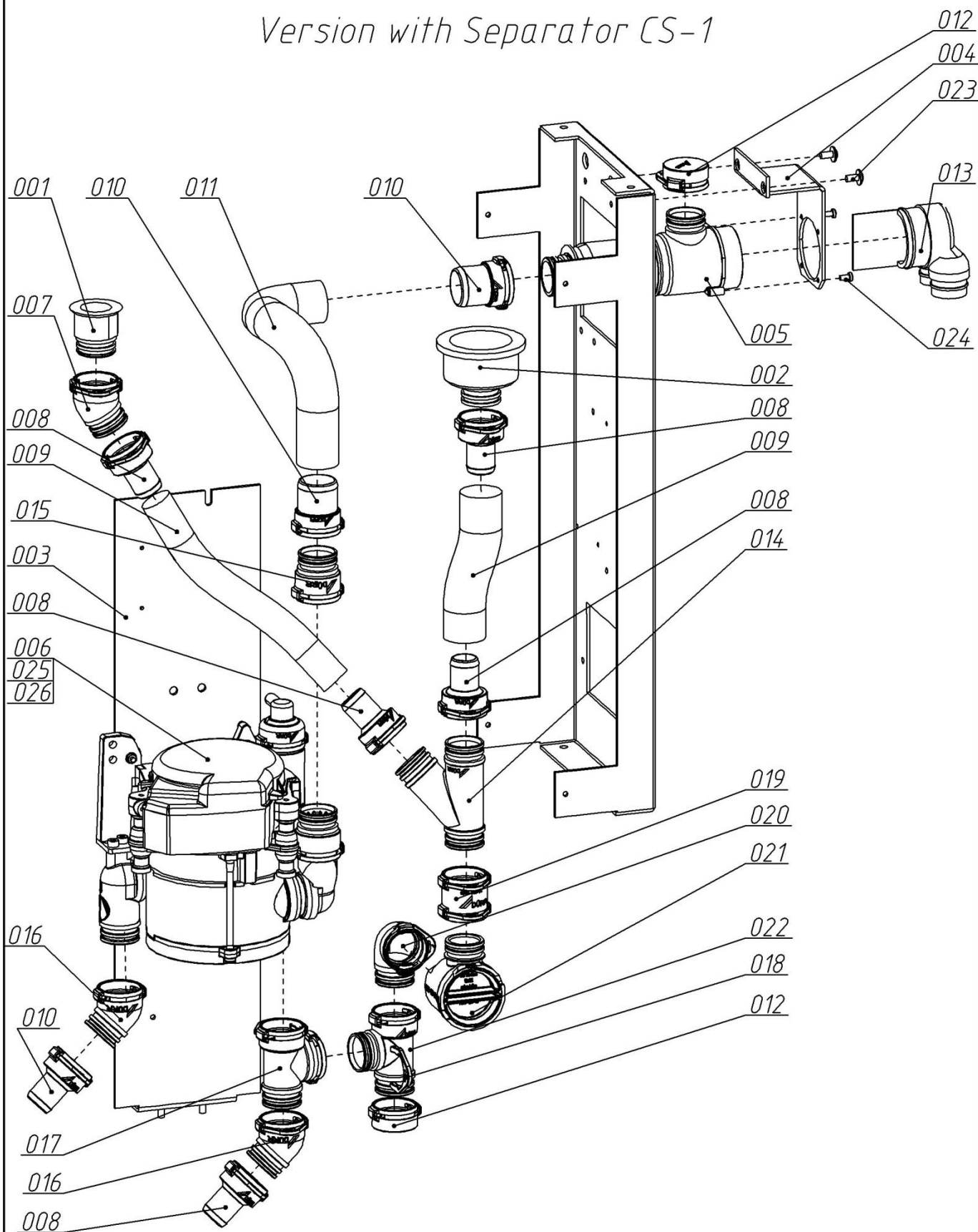


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.21

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.22	001	Plug	AEGA 116.74.00.010	
1.22	002	Nut	AEGA 116.74.00.009	
1.22	003	Bracket	AEGA 116.00.00.082	
1.22	004	Bracket	AEGA 116.00.00.090	
1.22	005	Spittoon bowl valve	7560-500-52	
1.22	006	Elbow 30°	0700-700-07	
1.22	007	Elbow 45°	0700-700-05	
1.22	008	Socket	0700-720-18	
1.22	009	Grey Pipe	Ø17.5 Cattani	
1.22	010	Securing ring	0700-700-01	
1.22	011	Socket	0700-720-25	
1.22	012	Secretion filter	0725-040-00	
1.22	013	Place selector valve	7560-500-60	
1.22	014	Filter cover	0725-041-05	
1.22	015	Sleeve Lignum Ø25		
1.22	016	Elbow 90°	0700-700-04	
1.22	017	Double connector	0700-700-13	
1.22	018	T-joint	0700-700-06	
1.22	019	Double Socket	0700-700-14	
1.22	020	Y-joint	0700-700-03	
1.22	021	Blank	0700-700-10	
1.22	022	Screw DIN 912 M4x10	5154010-2	
1.22	023	Screw DIN 96 3x12	1023012-2	
1.22	024	Screw DIN 912 M4x10	5154010-2	
1.22	025	Nut DIN 934 M4	6004-2	

Version with spittoon bowl valve

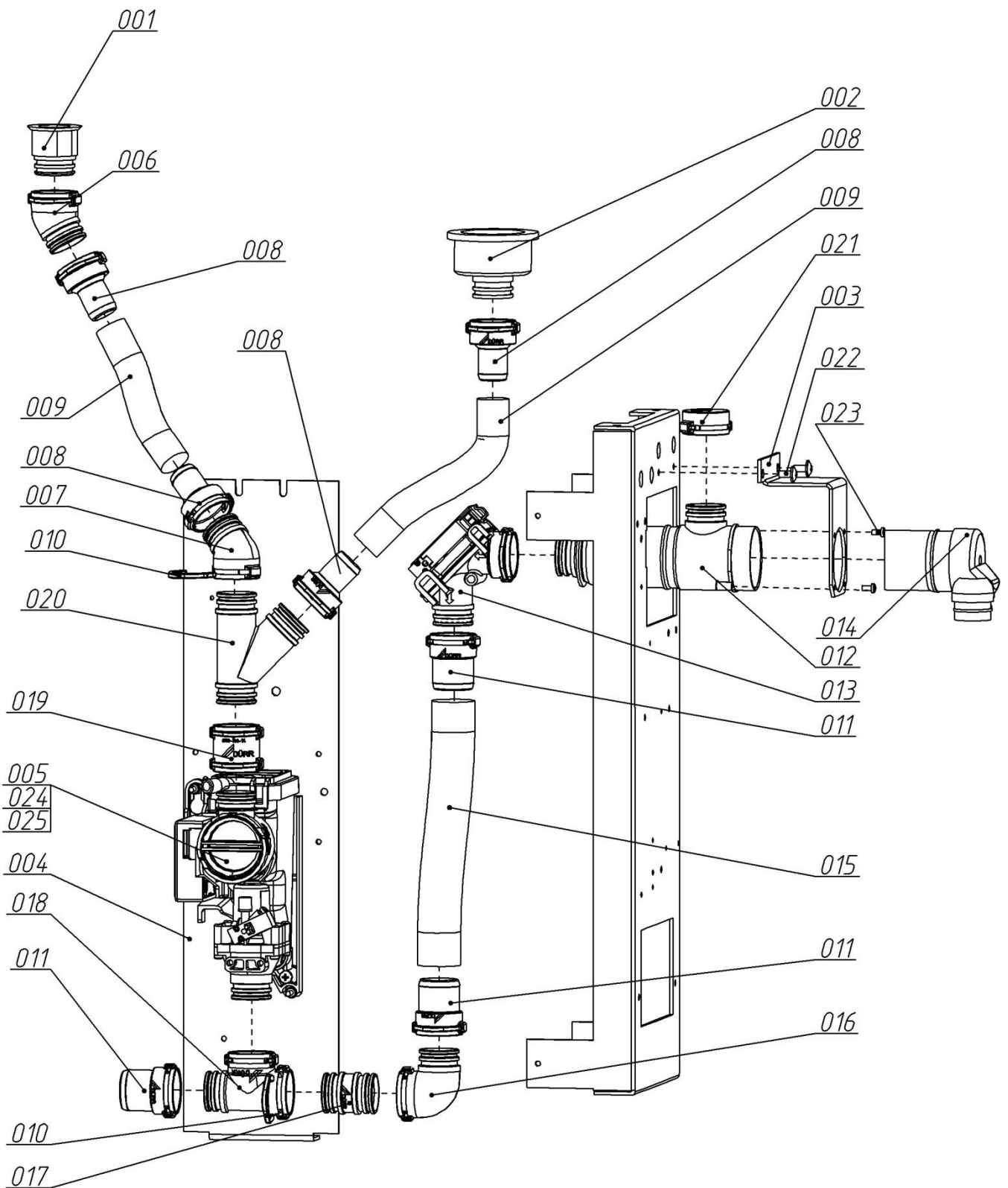


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.22

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.23	001	Plug	AEGA 116.74.00.010	
1.23	002	Nut	AEGA 116.74.00.009	
1.23	003	Bracket	AEGA 116.00.00.090	
1.23	004	Bracket	AEGA 116.00.00.082	
1.23	005	Bracket for Heavy particle collector		
1.23	006-01	Separator CS-1 without valve selection	7117-50	
1.23	006-02	Separator CS-1 valve selection	7117-57	
1.23	007	Heavy paticle collector		
1.23	008	T-joint	0700-700-06	
1.23	009	T-joint	0700-700-08	
1.23	010	Elbow 30°	0700-700-07	
1.23	011	Socket	0700-720-18	
1.23	012	boltttype clip	16-25 GOST 28191-89	
1.23	013	Grey Pipe	Ø17.5 Cattani	
1.23	014	Elbow 90°	0700-700-04	
1.23	015	Securing ring	0700-700-01	
1.23	016	Coarse filter	0700-702-52	
1.23	017	Blank	0700-700-10	
1.23	018	Secretion filter	0725-040-50	
1.23	019	Filter cover	0725-041-05	
1.23	020	Extension piece	0700-700-28	
1.23	021	Extension piece	0700-700-17	
1.23	022	Double Socket	0700-700-11	
1.23	023	Elbow 45°	0700-700-05	
1.23	024	Socket	0700-720-25	
1.23	025	boltttype clip	20-32 GOST 28191-89	
1.23	026	Sleeve Lignum Ø25		
1.23	027	Securing ring	0700-700-09	
1.23	028	Elbow 90°	7117-420-48	
1.23	029	T-joint	0700-700-02	
1.23	030	Double connector	0700-700-13	
1.23	031	Socket	0700-720-10	
1.23	032	Blank	0700-700-53	
1.23	033	Pipe Ø10mm		
1.23	034	Clip	9000-161-20	
1.23	035	Screw DIN 967 M4x8	5024008-2	
1.23	036	Screw DIN 912 M4x20	5154020-2	
1.23	037	Stand		
1.23	038	Nut DIN 934 M4	6004-2	

Version with Separator CS-1 and Heavy particle collector

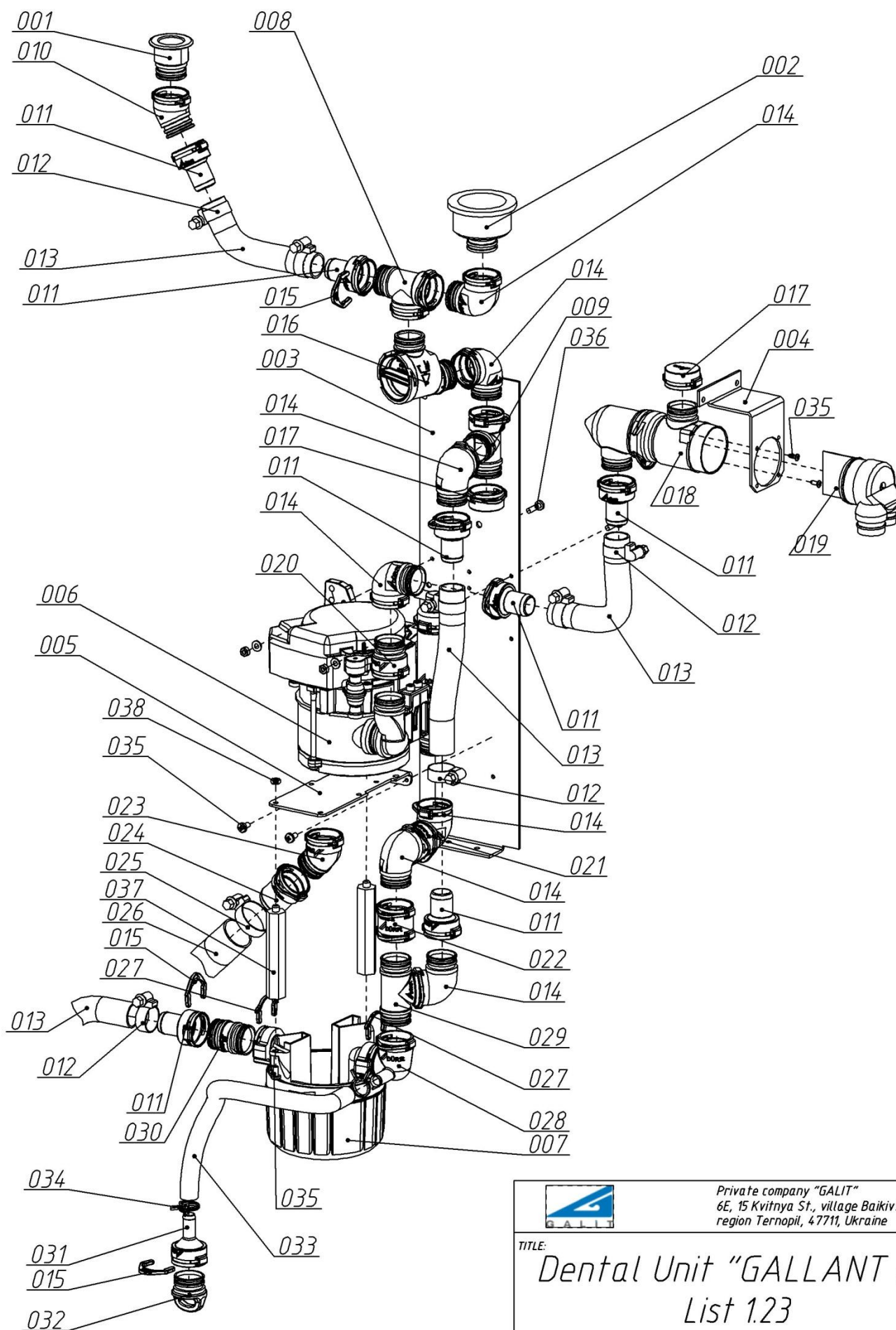


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.23

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.24	001	Stand	AEGA 116.80.01.000	
1.24	002	Bearer	AEGA 116.80.02.000	
1.24	003	Panel	AEGA 116.80.03.000	
1.24	004	Right angle piece	AEGA 116.80.00.001	
1.24	005	Left angle piece	AEGA 116.80.00.002	
1.24	006	Screw DIN 912 M6x12	5156012-2	
1.24	007	Screw DIN 965/963 M4x8	5M14008-2	
1.24	008	Washer DIN 125 6	7O06-2	
1.24	009	Washer DIN 7980 6	7P06-2	

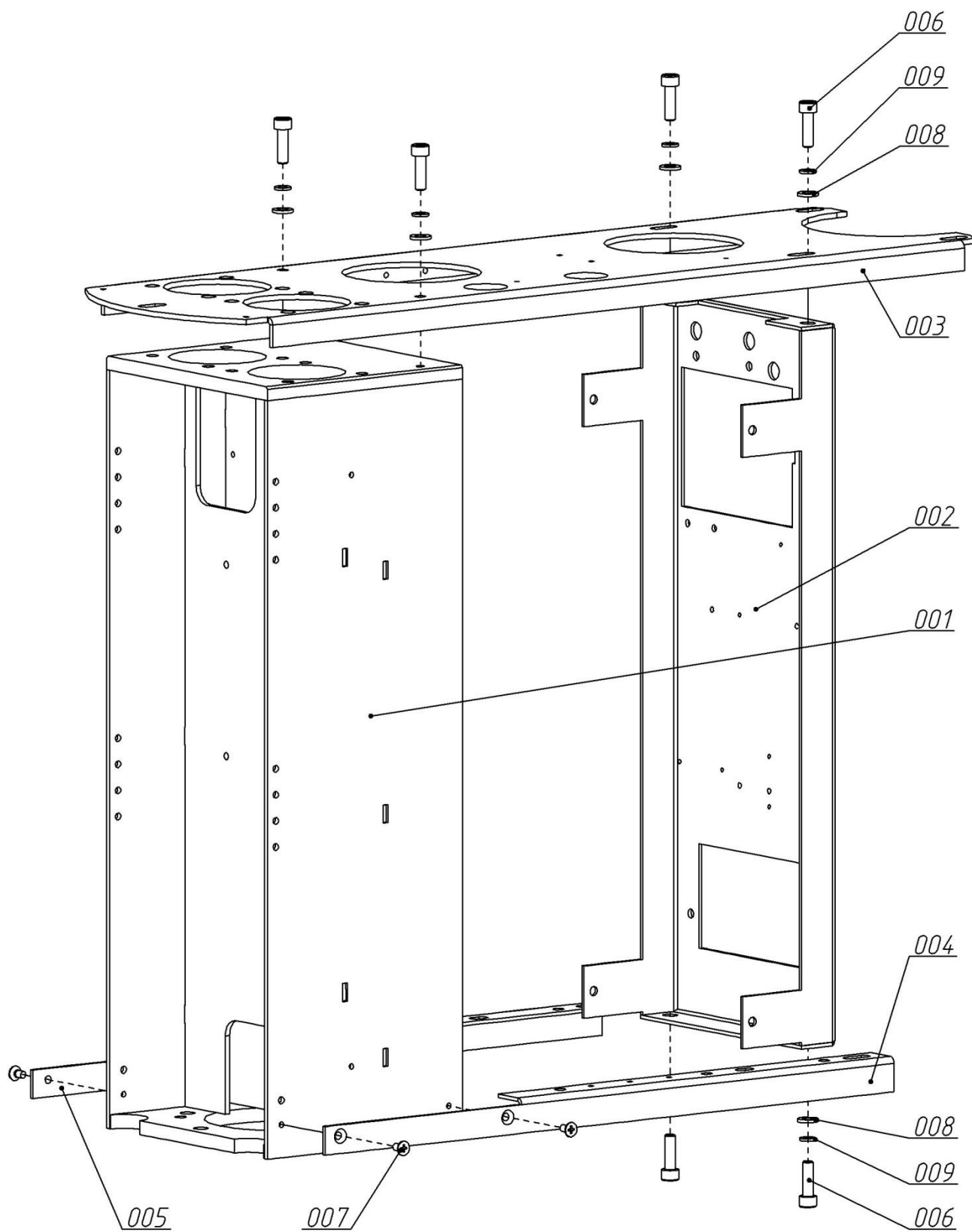


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.24


LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.25	001	Arm	GAL 043.01.00.000	
1.25	002	Axis	GAL 043.00.00.001	
1.25	003	Axis	GAL 043.00.00.002	
1.25	004	Ring	GAL 043.00.00.003	
1.25	005	Ball socket adjuster	GAL 043.00.00.004	
1.25	006	Holder	GAL 043.00.00.005	
1.25	007	Axis	GAL 043.00.00.006	
1.25	008	Nut	GAL 043.00.00.007	
1.25	009	plate	GAL 043.00.00.008	
1.25	010	Washer	GAL 043.00.00.009	
1.25	011	Washer	GAL 043.00.00.010	
1.25	012	Ring	GAL 043.00.00.011	
1.25	013	Blank	GAL 043.00.00.012	
1.25	014	Blank	GAL 043.00.00.012-01	
1.25	015	Screw DIN 914 M8x10	988010-H	
1.25	016	Screw DIN 915 M6x6	986006-C	
1.25	017	Screw DIN 915 M6x10	986010-C	
1.25	018	Screw DIN 965/963 M5x12	5M15012-2	

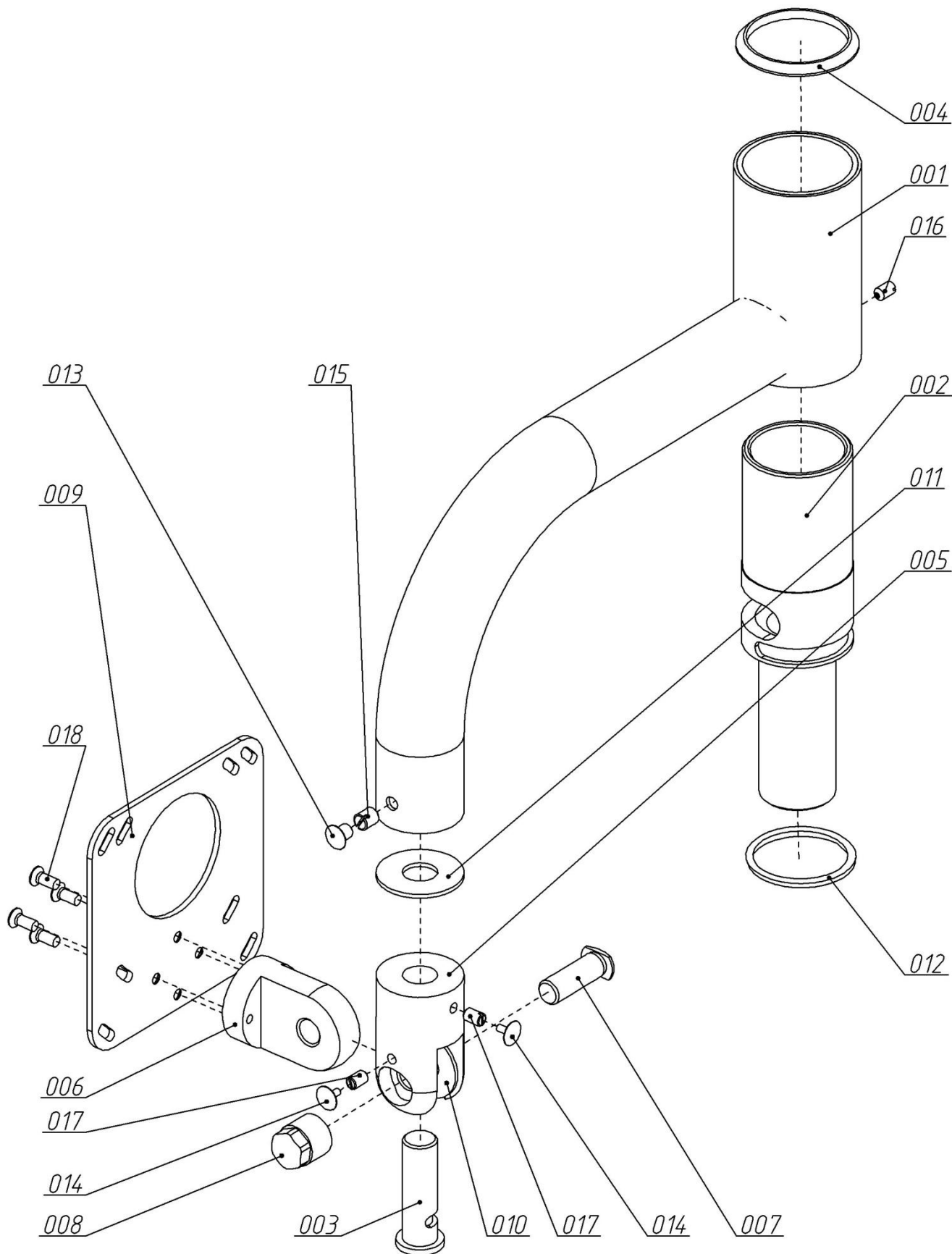

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:
Dental Unit "GALLANT H"
List 1.25

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.26	001	Bracket	AEGA 146.01.00.000	
1.26	002	Pipe	AEGA 146.00.00.001	
1.26	003	Bushing	AEGA 146.00.00.002	
1.26	004	Bushing	AEGA 146.00.00.003	
1.26	005	Cover	AEGA 146.00.00.004	
1.26	006	Cover	AEGA 146.00.00.005	
1.26	007	Axis	AEGA 146.00.00.007	
1.26	008	Screw DIN 915 M6x10	986010-C	
1.26	009	Screw DIN 965/963 M4x16	5M14016-2	
1.26	010	Stop ring A24 GOST 13940-68	95PK1024	

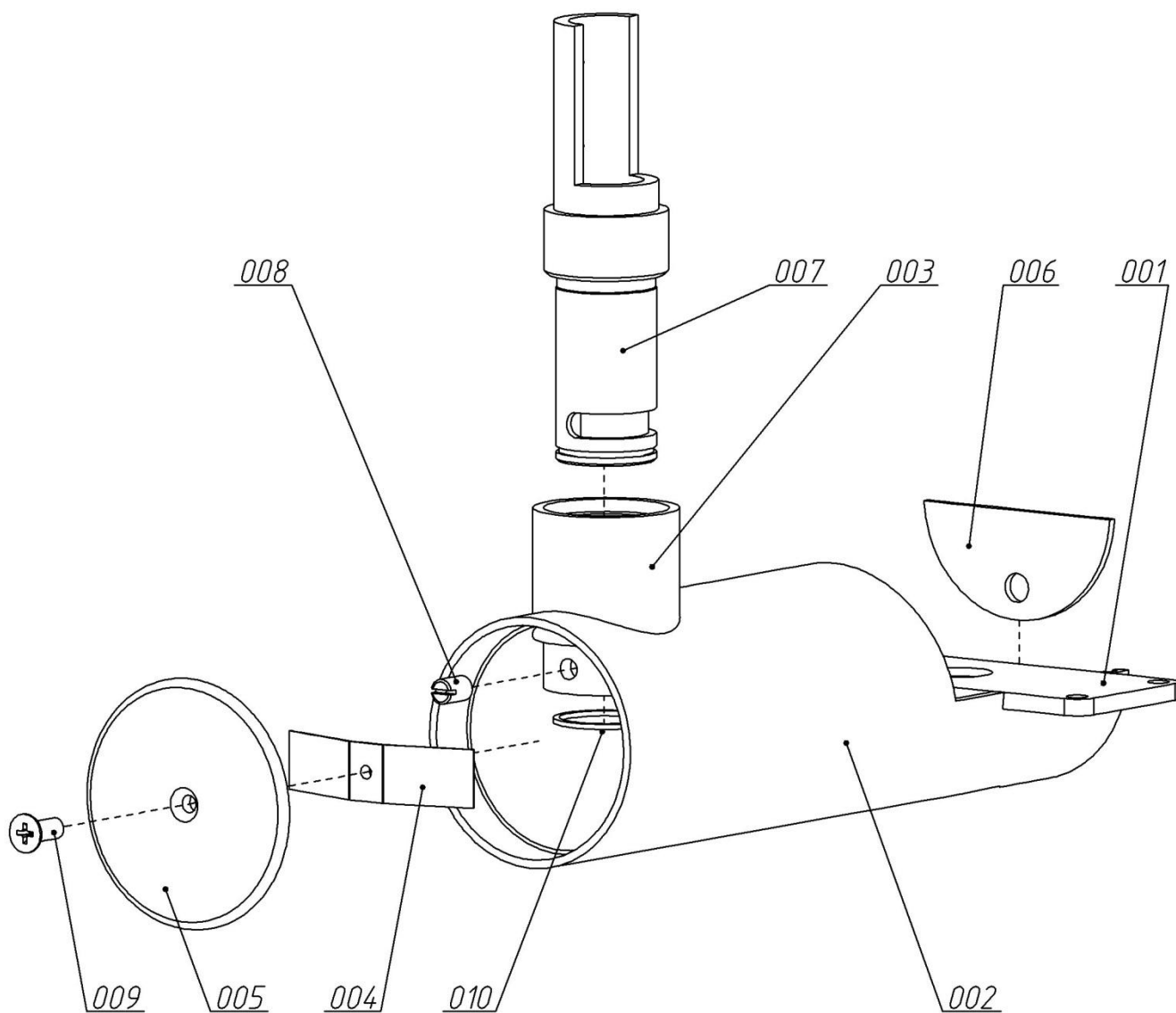


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.26

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.27	001	Bleed	AEGA 016.03.00.001-01	
1.27	002	Suction	AEGA 016.03.00.002	
1.27	003	Case	AEGA 016.03.00.003	
1.27	004	Trap	AEGA 016.03.00.004	
1.27	005	O-ring	2661 M5	
1.27	006	O-ring	2661 M6	
1.27	007	O-ring	2661 1/8	
1.27	008	Buckle	10-16 ГОСТ 28191-89	
1.27	009	Sleeve	FOOD 10+metallic fiber	
1.27	010	Fitting	2013 1/8	

Water bleed

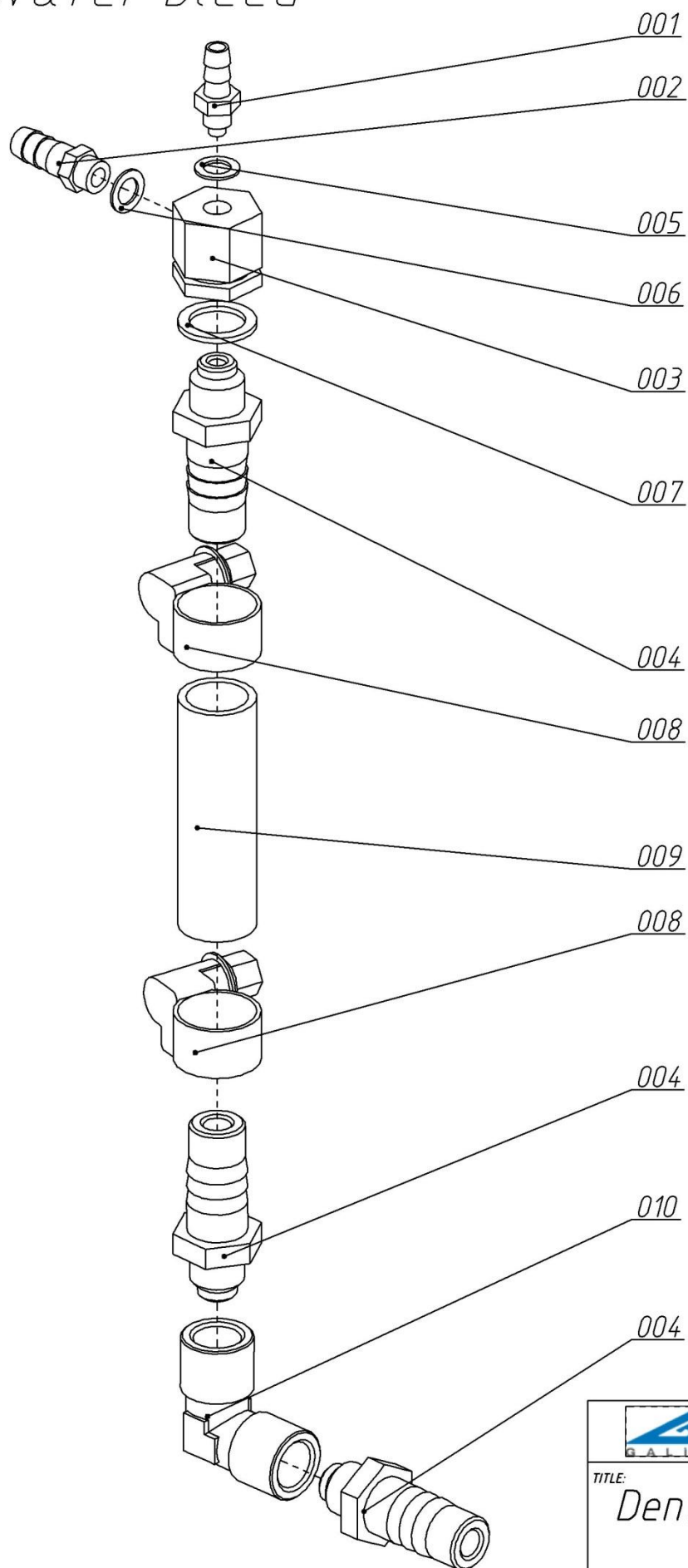


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.27

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.28	001	Bleed	AEGA 016.03.00.001-01	
1.28	002	Suction	AEGA 016.03.00.002	
1.28	003	Case	AEGA 016.03.00.003	
1.28	004	Trap	AEGA 016.03.00.004	
1.28	005	O-Ring	2661 M5	
1.28	006	O-Ring	2661 M6	
1.28	007	O-Ring	2661 1/8	

Air bleed

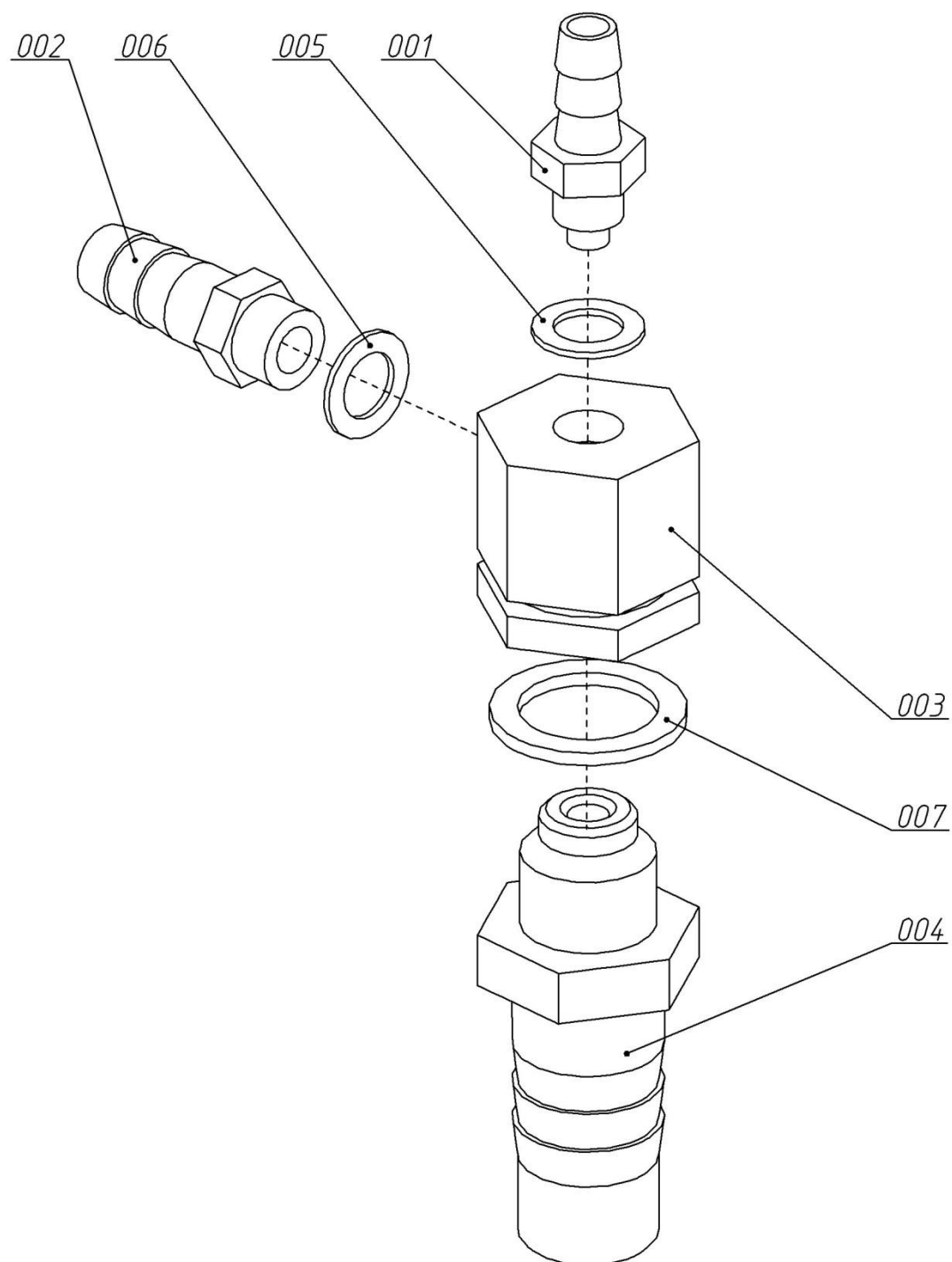


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.28

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
1.29	001	Grey Pipe	Ø17.5 Cattani (0.2 m.)	
1.29	002	Grey Pipe	Ø17.5 Cattani (0.5 m.)	
1.29	003	Grey Pipe	Ø17.5 Cattani (0.3 m.)	
1.29	004	Grey Pipe	Ø17.5 Cattani (0.1 m.)	
1.29	005	Grey Pipe	Ø17.5 Cattani (2.4 m.)	
1.29	006	Sleeve Lignum	Ø25 (0.4 m.)	
1.29	007	Sleeve Lignum	Ø25 (2.4 m.)	

Version with eco sepamatic duerr

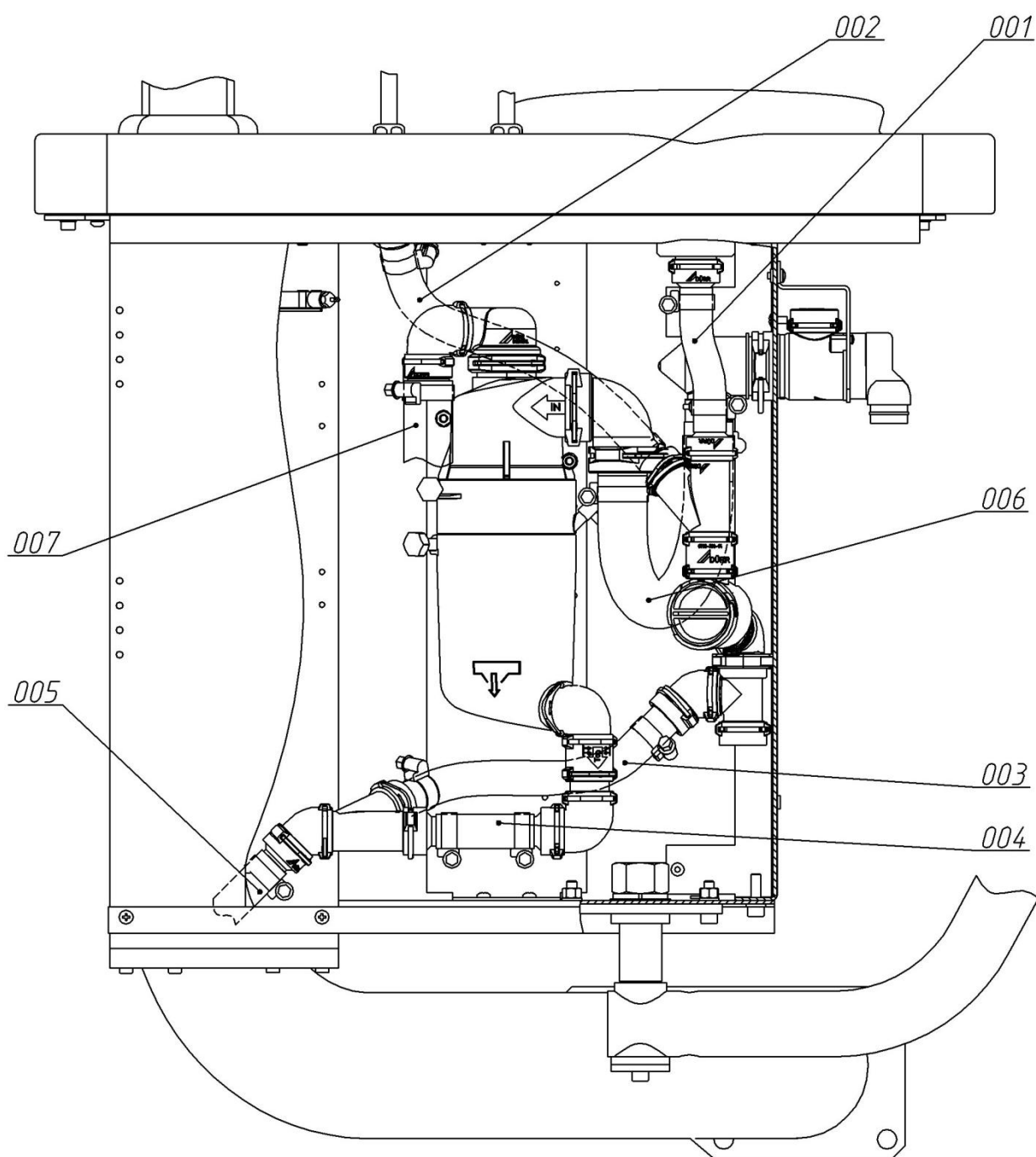


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.29

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.30	001	Bracket	AEGA 116.00.00.082	
1.30	002	Bracket	AEGA 116.00.00.090	
1.30	003	Nut DIN 934 M5	6005-2	
1.30	004	bolt type clip	16-25 GOST 28191-89	
1.30	005	bolt type clip	20-32 GOST 28191-89	
1.30	006	Washer DIN 125 5	7005-2	
1.30	007	Stud	M5x90	
1.30	008	Securing ring	0700-700-01	
1.30	009	Y-joint	0700-700-03	
1.30	010	Elbow 90°	0700-700-04	
1.30	011	Elbow 45°	0700-700-05	
1.30	012	Blank	0700-700-10	
1.30	013	Double connector	0700-700-13	
1.30	014	Double Socket	0700-700-14	
1.30	015	Socket	0700-720-18	
1.30	016	Socket	0700-720-25	
1.30	017	Securing ring	0700-800-01	
1.30	018	Elbow 90°	0700-800-04	
1.30	019	Elbow	0700-800-31	
1.30	020	Socket	0700-820-25	
1.30	021	Secretion filter	0725-040-50	
1.30	022	Filter cover	0725-041-05	
1.30	023	Eco sepamatic	7150-700-50	
1.30	024	Sleeve Lignum	Ø25 (0.4 m.)	
1.30	025	Sleeve Lignum	Ø25 (2.4 m.)	
1.30	026	Grey Pipe	Ø17.5 Cattani (2.4 m.)	

Version with eco sepamatic duerr

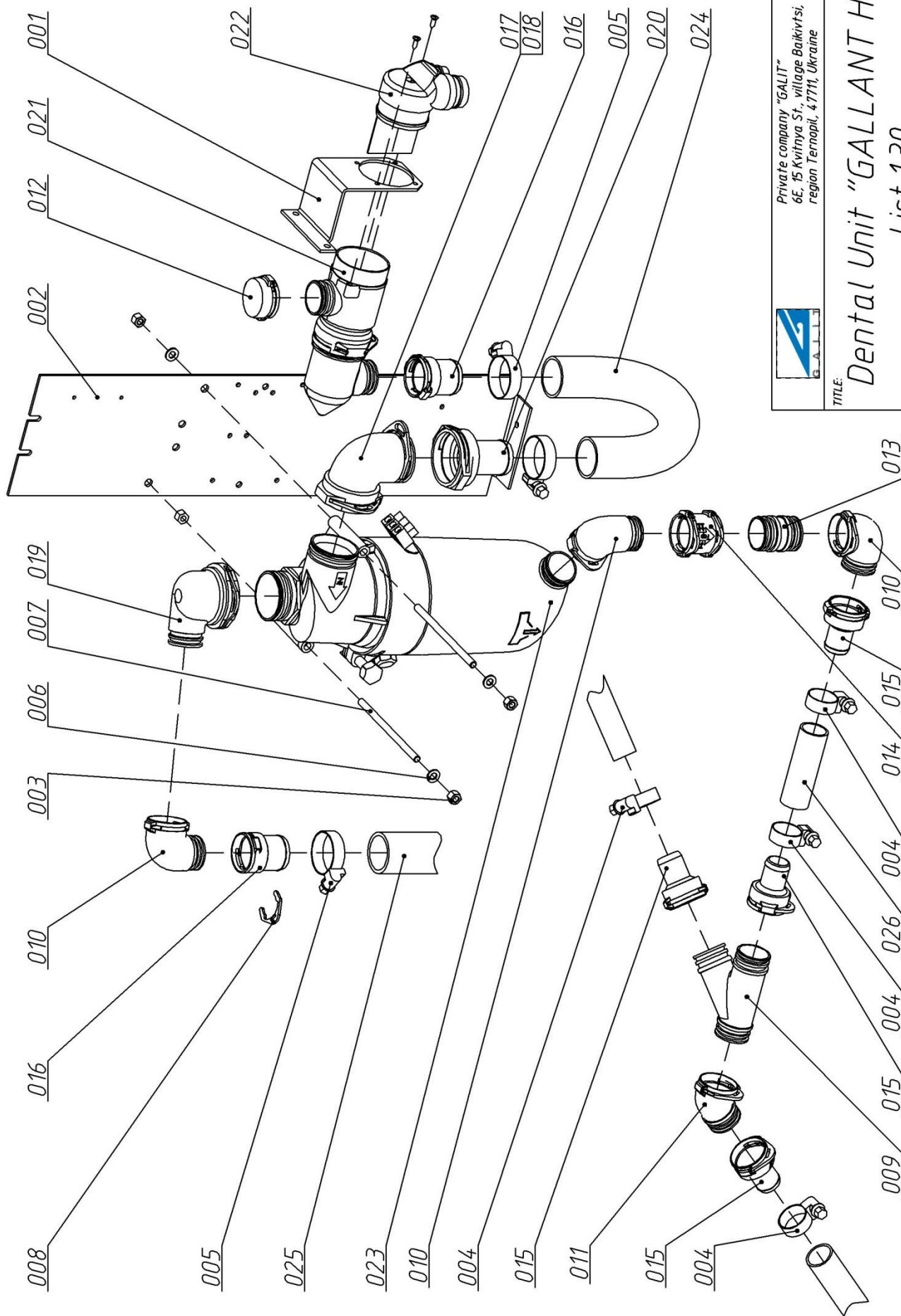

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine




TITLE:

Dental Unit "GALLANT H"

List 1.30

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.31	001	Plug	AEGA 116.74.00.010	
1.31	002	Nut	AEGA 116.74.00.009	
1.31	003	Screw DIN 912 M3x30	5153030-2	
1.31	004	bolt type clip	16-25 GOST 28191-89	
1.31	005	Securing ring	0700-700-01	
1.31	006	Y-joint	0700-700-03	
1.31	007	Elbow 90°	0700-700-04	
1.31	008	Elbow 45°	0700-700-05	
1.31	009	T-joint	0700-700-08	
1.31	010	Securing ring	0700-700-09	
1.31	011	Blank	0700-700-10	
1.31	012	Double Socket	0700-700-14	
1.31	013	Clamp	0700-700-54	
1.31	014	Coarse filter	0700-702-52	
1.31	015	Socket	0700-720-18	
1.31	016	Grey Pipe	Ø17.5 Cattani (0.1 m.)	
1.31	017	Grey Pipe	Ø17.5 Cattani (0.2 m.)	
1.31	018	Grey Pipe	Ø17.5 Cattani (0.3 m.)	
1.31	019	Grey Pipe	Ø17.5 Cattani (0.5 m.)	
1.31	020	Grey Pipe	Ø17.5 Cattani (2.4 m.)	

Version with eco sepamatic duerr

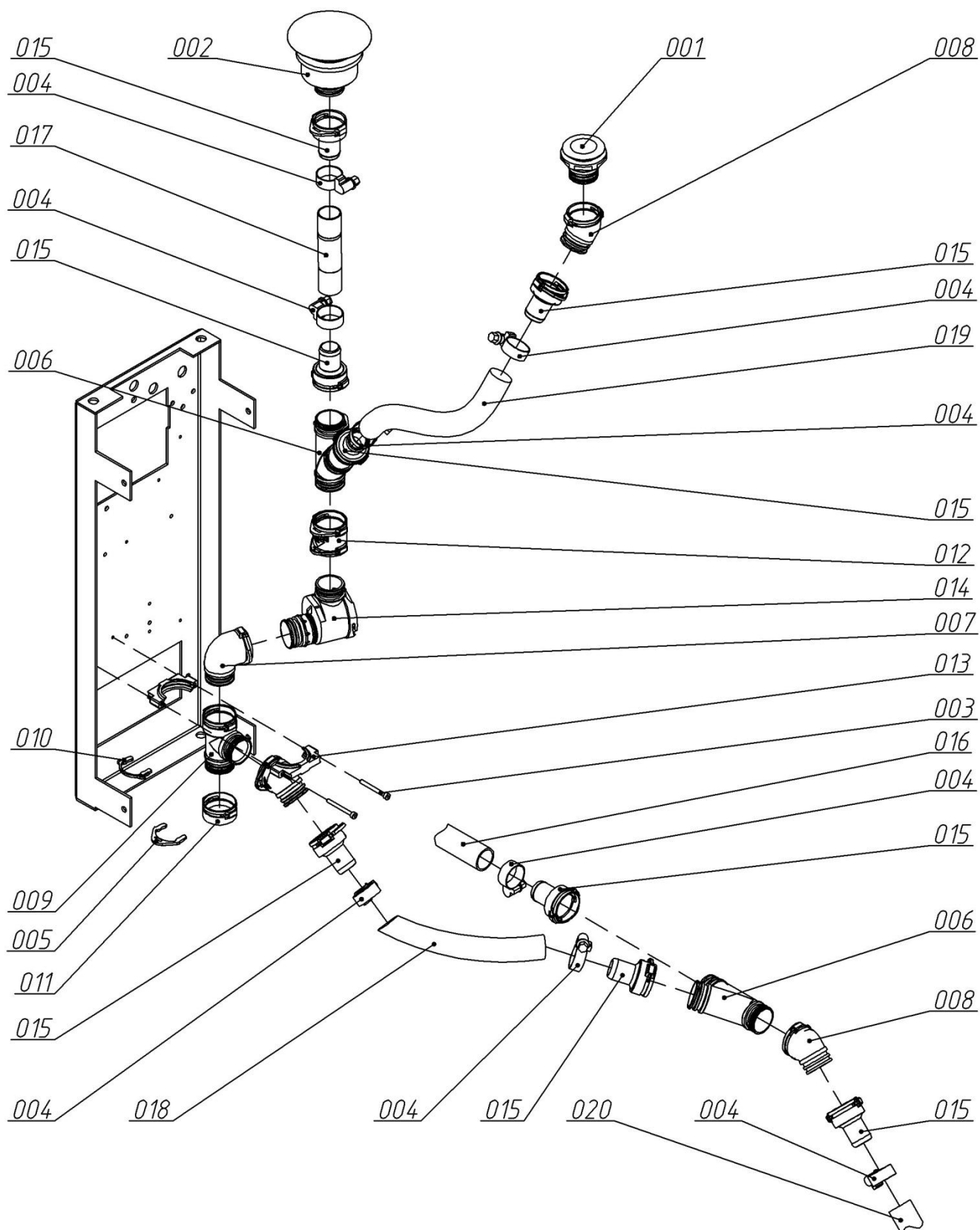


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.31

LIST	Nº	DESCRIPTION	CODE KD	CODE GALIT
1.32	001	Grey Pipe	Ø17.5 Cattani (0.2 m.)	
1.32	002	Grey Pipe	Ø17.5 Cattani (0.5 m.)	
1.32	003	Grey Pipe	Ø17.5 Cattani (0.3 m.)	
1.32	004	Grey Pipe	Ø17.5 Cattani (0.1 m.)	
1.32	005	Grey Pipe	Ø17.5 Cattani (2.4 m.)	
1.32	006	Sleeve Lignum	Ø25 (0.4 m.)	
1.32	007	Sleeve Lignum	Ø25 (2.4 m.)	

*Version with eco sepamatic duerr
with selector switch*

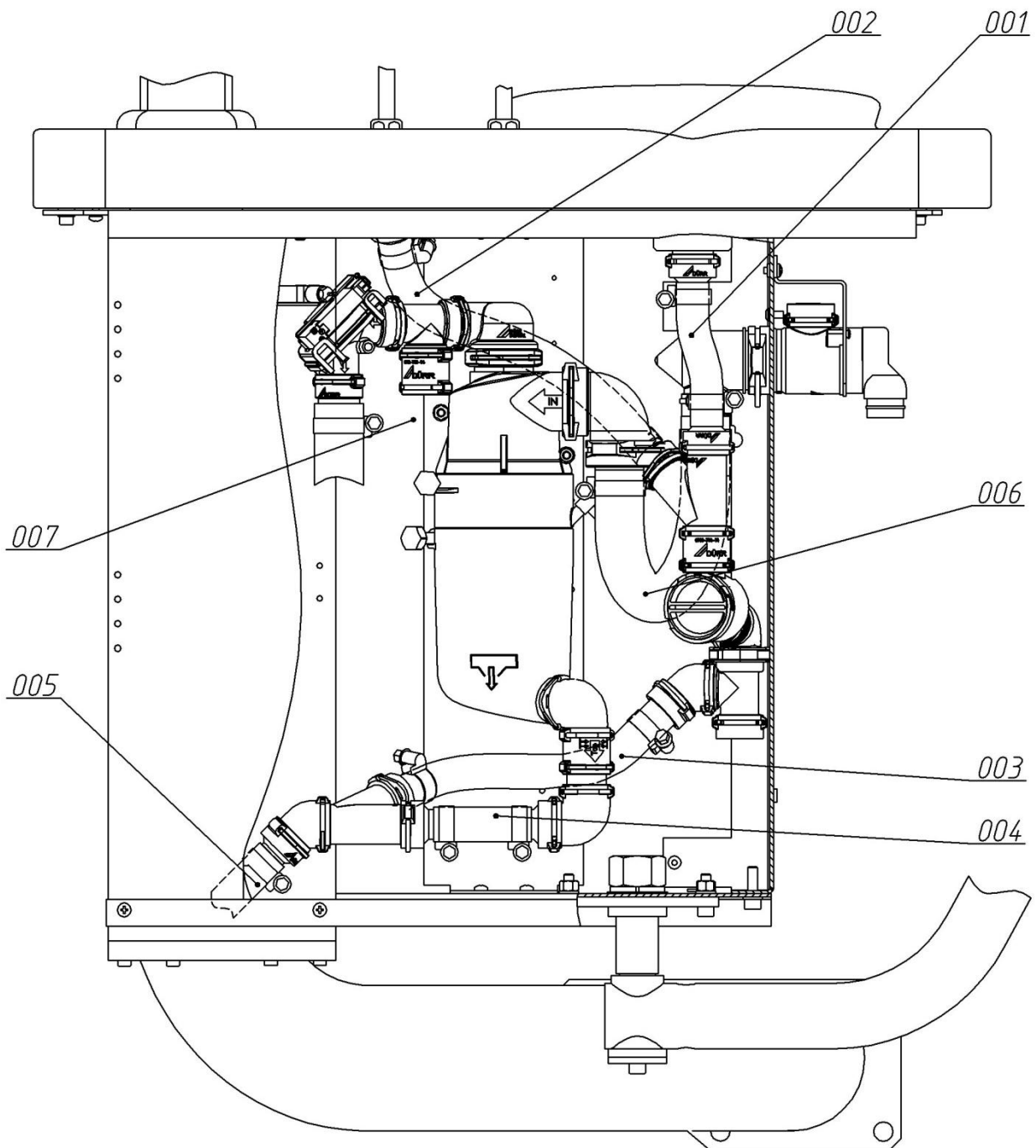

Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

*Dental Unit "GALLANT H"
List 1.32*

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.33	001	Bracket	AEGA 116.00.00.082	
1.33	002	Bracket	AEGA 116.00.00.090	
1.33	003	Nut DIN 934 M5	6005-2	
1.33	004	bolt type clip	16-25 GOST 28191-89	
1.33	005	bolt type clip	20-32 GOST 28191-89	
1.33	006	Washer DIN 125 5	7005-2	
1.33	007	Stud	M5x90	
1.33	008	Securing ring	0700-700-01	
1.33	009	Y-joint	0700-700-03	
1.33	010	Elbow 90°	0700-700-04	
1.33	011	Elbow 45°	0700-700-05	
1.33	012	T-joint	0700-700-08	
1.33	013	Blank	0700-700-10	
1.33	014	Double connector	0700-700-13	
1.33	015	Double Socket	0700-700-14	
1.33	016	Socket	0700-720-18	
1.33	017	Socket	0700-720-25	
1.33	018	Securing ring	0700-800-01	
1.33	019	Elbow 90°	0700-800-04	
1.33	020	Elbow	0700-800-31	
1.33	021	Socket	0700-820-25	
1.33	022	Secretion filter	0725-040-50	
1.33	023	Filter cover	0725-041-05	
1.33	024	Eco sepamatic	7150-700-50	
1.33	025	Place Selector valve	7560-500-60	
1.33	026	Sleeve Lignum	Ø25 (0.4 m.)	
1.33	027	Sleeve Lignum	Ø25 (2.4 m.)	
1.33	028	Grey Pipe	Ø17.5 Cattani (2.4 m.)	

The diagram is an exploded view of the Dental Unit 'GALLANT H'. It shows the main body of the unit, a large U-shaped water supply pipe, and various smaller components like elbows, tees, and fittings. Each part is labeled with a number in a box, indicating its assembly sequence. The numbers are arranged in a grid-like fashion around the diagram. The main body of the unit is shown in the center, with the U-shaped pipe to its right. The exploded view shows the assembly of the water supply system, including the main body, the U-shaped pipe, and various fittings. The diagram is a technical drawing, likely a patent or a technical manual, showing the assembly of the dental unit.

Private company "GALIT"
6E, 15 Kvintya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE: **Dental Unit "GALLANT H"**
List 133

Private company "GALIT"
6E, 15 Kvitrnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine


TITLE:

Dental Unit "GALLANT H"

List 1.33

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.34	001	Plug	AEGA 116.74.00.010	
1.34	002	Nut	AEGA 116.74.00.009	
1.34	003	Screw DIN 912 M3x30	5153030-2	
1.34	004	bolt type clip	16-25 GOST 28191-89	
1.34	005	Securing ring	0700-700-01	
1.34	006	Y-joint	0700-700-03	
1.34	007	Elbow 90°	0700-700-04	
1.34	008	Elbow 45°	0700-700-05	
1.34	009	T-joint	0700-700-08	
1.34	010	Securing ring	0700-700-09	
1.34	011	Blank	0700-700-10	
1.34	012	Double Socket	0700-700-14	
1.34	013	Clamp	0700-700-54	
1.34	014	Coarse filter	0700-702-52	
1.34	015	Socket	0700-720-18	
1.34	016	Grey Pipe	Ø17.5 Cattani (0.1 m.)	
1.34	017	Grey Pipe	Ø17.5 Cattani (0.2 m.)	
1.34	018	Grey Pipe	Ø17.5 Cattani (0.3 m.)	
1.34	019	Grey Pipe	Ø17.5 Cattani (0.5 m.)	
1.34	020	Grey Pipe	Ø17.5 Cattani (2.4 m.)	

*Version with eco sepamatic duerr
with selector switch*

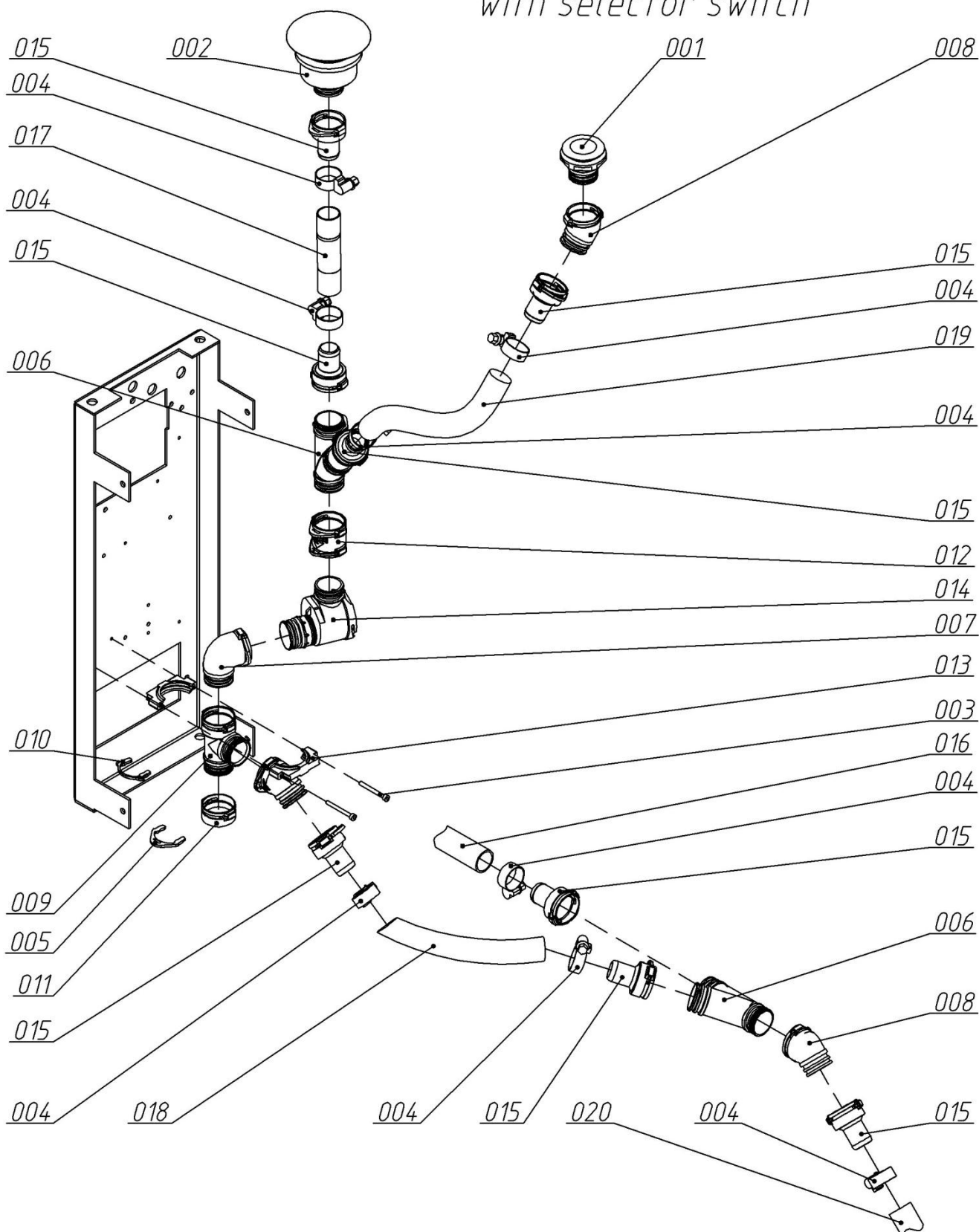


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

*Dental Unit "GALLANT H"
List 1.34*

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.35	001	Screw	AEGA 116.00.00.062-01	
1.35	002	O-ring	AEGA 116.77.00.001	
1.35	003	Bearing	HK 40x16	
1.35	004	O-ring A47 GOST 13941-68	95PK2047	
1.35	005	Screw DIN 914 M6x6	986006-H	
1.35	006	Screw DIN 913 M6x10	986010-P	

Connection of Pantograf arm and Light post to framework

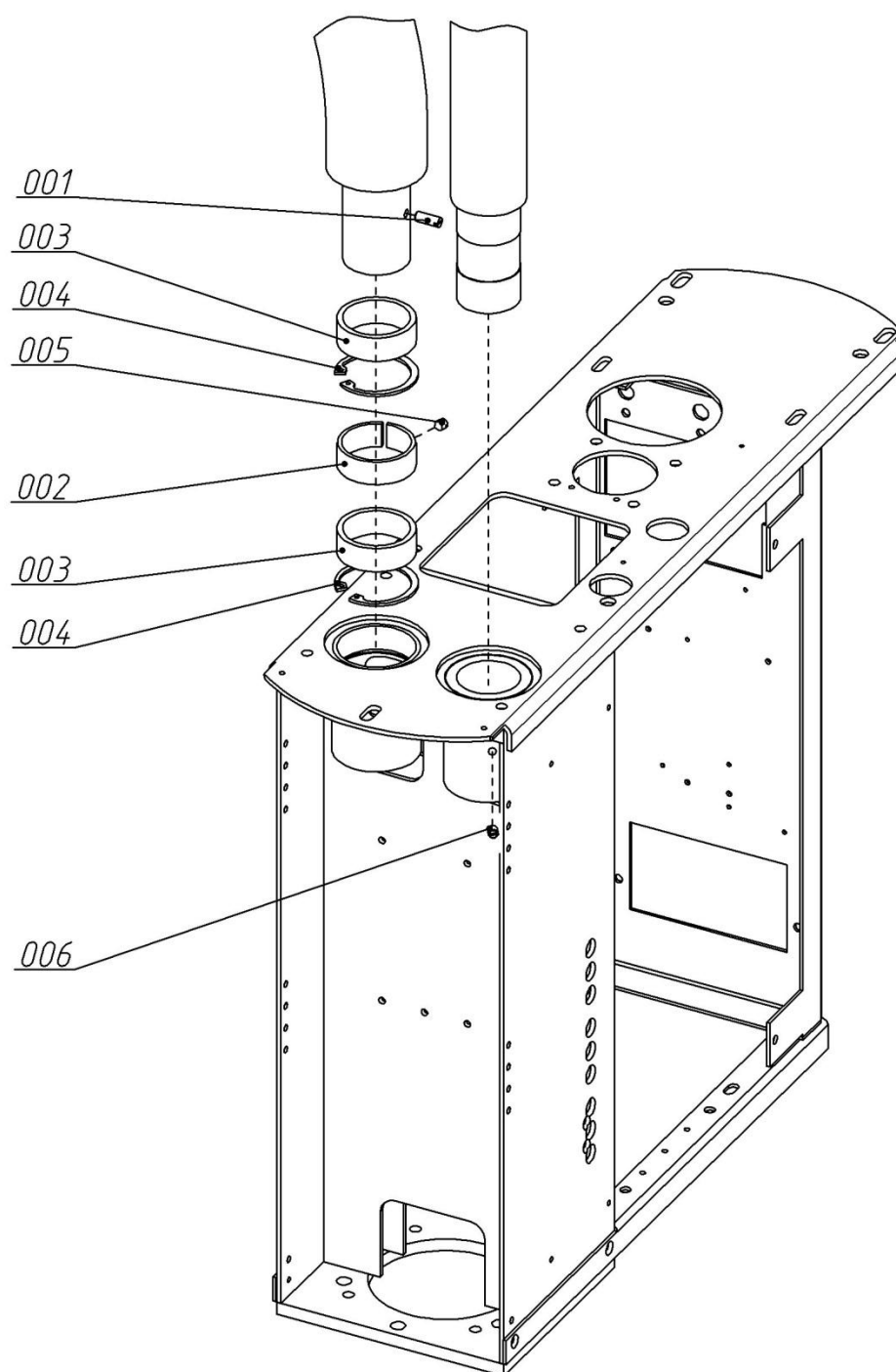


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.35

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.36	001	Stand	AEGA 116.79.01.000	
1.36	002	Bearer	AEGA 116.80.02.000	
1.36	003	Panel	AEGA 8.050.045	
1.36	004	Right angle piece	AEGA 116.80.00.001	
1.36	005	Left angle piece	AEGA 116.80.00.002	
1.36	006	Screw DIN 912 M6x12	5156012-2	
1.36	007	Screw DIN 965/963 M4x8	5M14008-2	
1.36	008	Screw DIN 965/963 M6x10	5M16010-2	

Frame

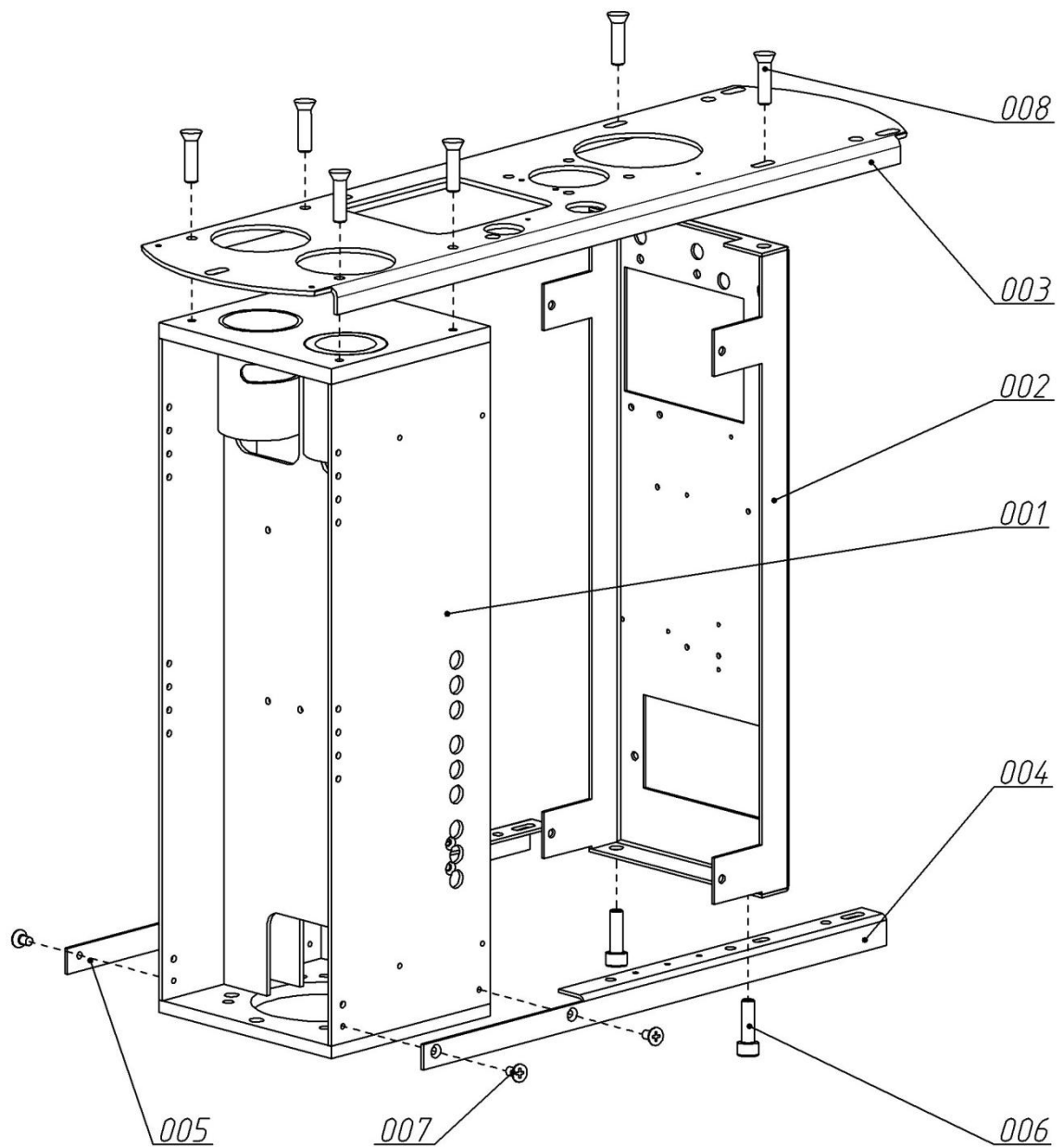


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.36

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.37	001	Bracket	AEGA 8.090.092	
1.37	002	Bracket	AEGA 116.00.00.090	
1.37	003	Screw DIN 7985 M4x8	5M54008-2	
1.37	004	Screw DIN 7985 M5x12	5M55012-2	
1.37	005	Nut DIN 934 M5	6005-2	
1.37	006	Washer DIN 7980 4	7P04-2	
1.37	007	Washer DIN 7980 5	7P05-2	
1.37	008	Washer DIN 125 4	7O04-2	
1.37	009	Washer DIN 125 5	7O05-2	
1.37	010	Water heater	45315	
1.37	011	Fitting	1500 6/4-1/8	
1.37	012	Fitting	2520 1/8-1/8	

Water heater

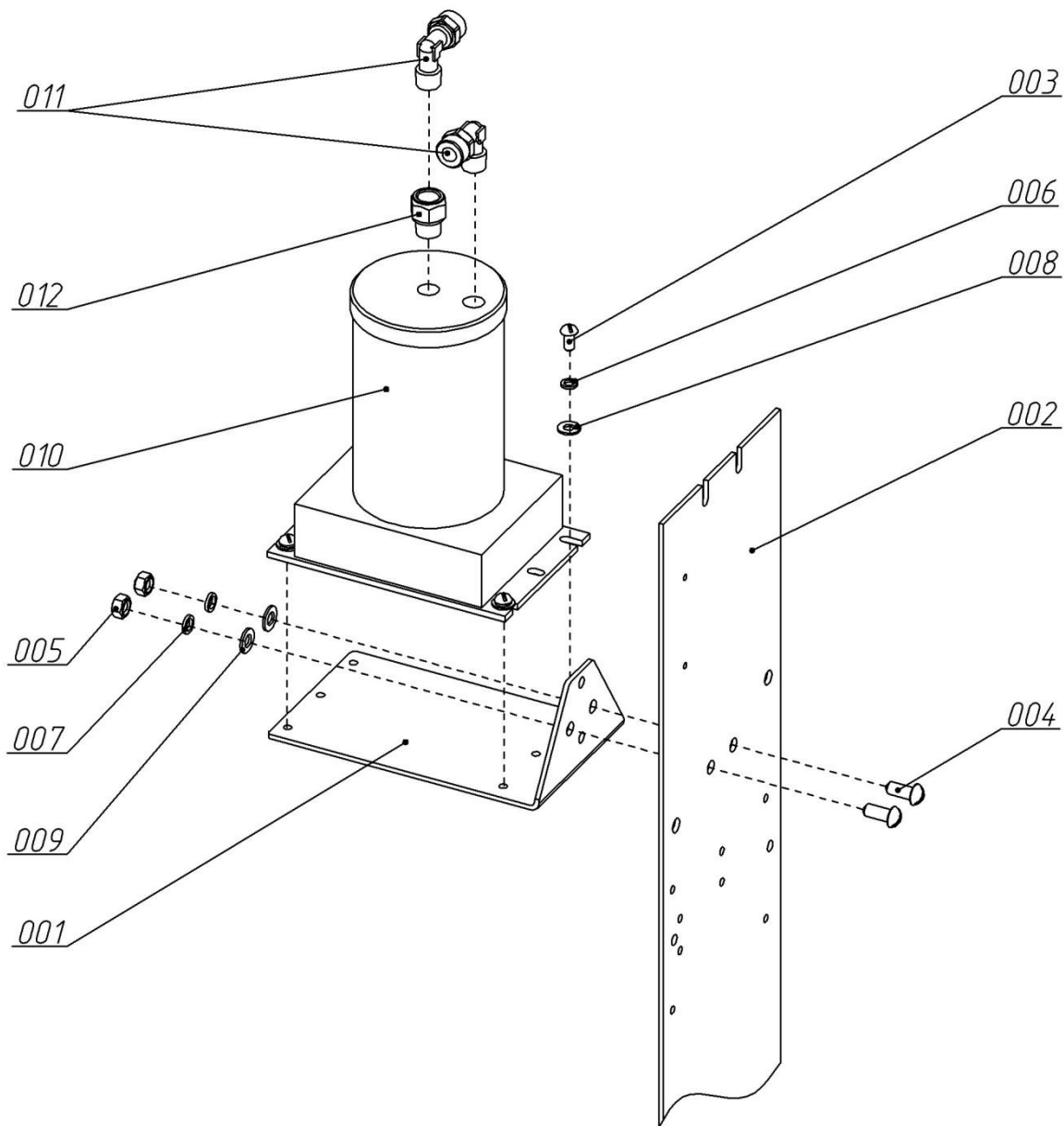


Private company "GALIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.37

LIST	№	DESCRIPTION	CODE KD	CODE GALIT
1.38	001	Cup	AEGA 6.210.002	
1.38	002	Cuspidor	AEGA 716.05.00.000	
1.38	003	Stud	AEGA 216.00.00.011	
1.38	004	Clamp	GAL 268.00.00.001	
1.38	005	Upper plate	AEGA 8.076.016	
1.38	006	Screw DIN 913 M4x6	984006-P	
1.38	007	Screw DIN 912 M8x20	5158020-2	
1.38	008	Washer DIN 125 8	7008-2	
1.38	009	Bushing	DU5025	
1.38	010	Washer	WC52	
1.38	011-01		RI07051	
1.38	011-02	Cuspidor screen with gray lid	VCF-353-02	

Cuspidor block

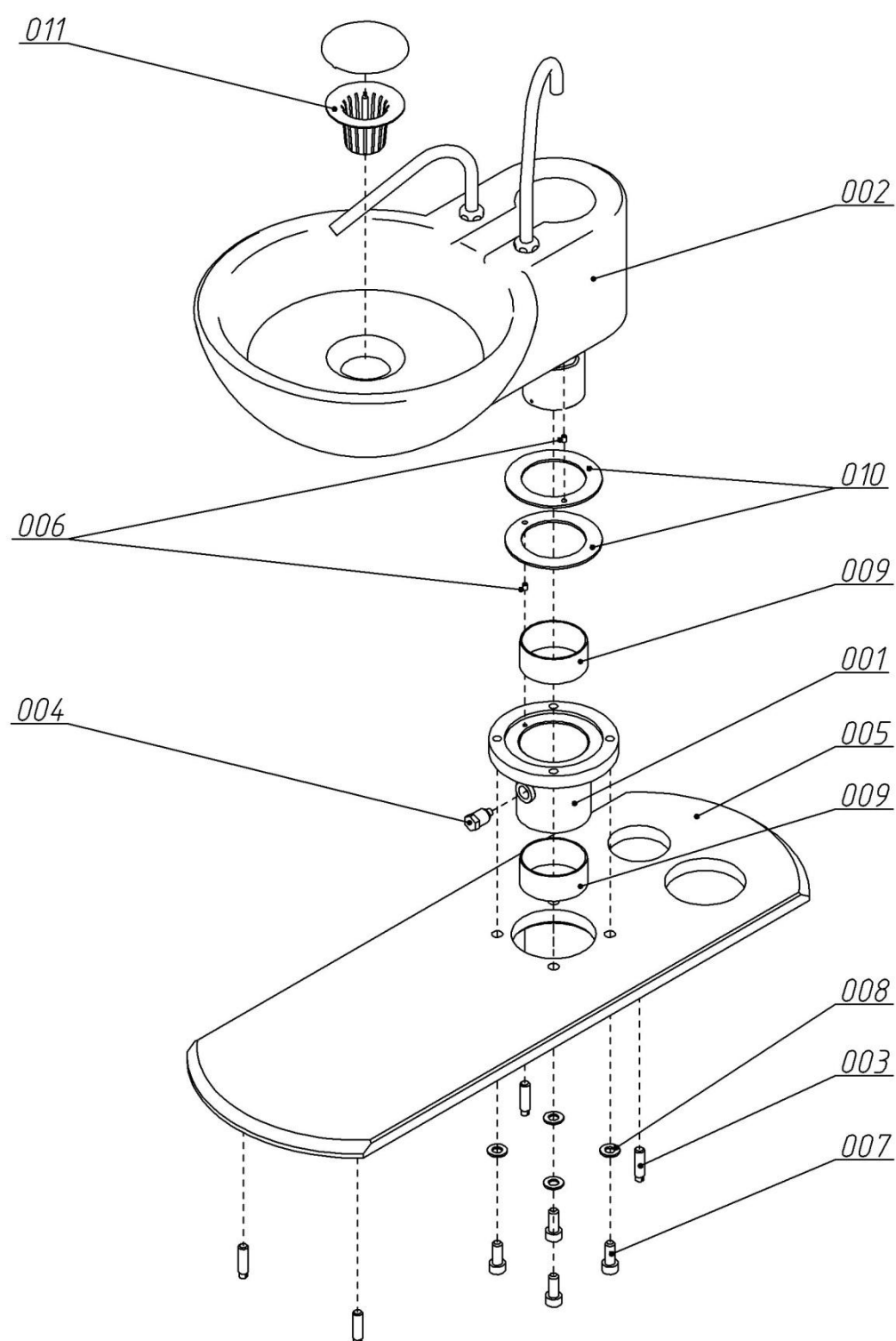

Private company "GALLIT"
6E, 15 Kvitnya St., village Baikivtsi,
region Ternopil, 47711, Ukraine

TITLE:

Dental Unit "GALLANT H"
List 1.38